

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-A allele ^{1,2}	ser ³																																		
*01:01:01:01-01:22, 01:01:24-01:31, 01:01:33-01:01:47, 01:01:49-01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:03:03, 01:09:01:01-01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303:01-01:306, 01:308N-01:313, 01:315-01:341, 01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:400N, *01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:396Q, 01:403, 01:418N	A1, Null, -																																		
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307, 01:342, 01:401	-				4																						26								
*01:01:56	-	1			4								12														26					30			
*01:04:01:01N-01:04:01:02N	Null	1			4																						26					30	31		
*01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1			4																														
*01:07, 01:95, 01:289	A1, -	1			4													17									26								
*01:12, 01:19, 01:200, 01:244	A1, -	1																										27							
*01:13, 01:176	A1, -	1			4							11		13		15											26		28						
*01:14, 01:391	A1, -	1																																	
*01:15N	Null	1			w																														
*01:17, 01:302	A1, -	1			4																						26		28						
*01:21, 01:126	A1, -	1			4																						26	27							
*01:26, 01:192	A1, -	1			4							11															26								
*01:28, 01:299	-	1			4							11				15											26								
*01:43	-	1			4							10															26								
*01:51	-	1			4				8			10															26								
*01:121	-	1			4															19							26								
*01:134	-	1			4																						26				29				
*01:136, 01:380:01:01-01:380:01:02	-	1			4							11															26		28						
*01:194	-	1			4								13		15												26		28						
*01:229	-	1			4							11			15												26				29				
*01:298	-	1			4					9								17									26								
*01:301Q	-	1			4	5		7																			w								
*01:344	-	1			4				8																										
*01:360	-	1			4																										27				
*01:361N	Null	1			4								12														26								
*01:381	-	1			4				8																		26								
*01:427, 36:02 ⁴	A36, -	1		3										14									22	23											
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:155, 02:157:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:1049, 02:1051-02:1089	A2, Low A2, -, A203, A210, Null, -																																		
				2																															

Negative Control

[illegible]

*03:34, 03:62:01-01-03:62:01:02, 03:263, 03:447, 03:451	-		3									13														27								
*03:41, 03:382	-		3																23						27									
*03:43, 03:82, 03:186	-		3									13	15	17											27	28								
*03:50	-		3				8					13													28									
*03:63	-		3							11		13							23						27									
*03:75, 03:177	-											13							23						28									
*03:88	-		3	4						11									23						27									
*03:89:01	-		2	3																					27									
*03:89:02, 03:108, 03:172, 03:198	-		3																						27	29								
*03:95, 03:343	-											13	14	16					23						28									
*03:152, 03:273	-		3	5										17											27									
*03:187	-											13							22						28									
*03:219, 03:429	-		3											17										27	28									
*03:231:01	-		3									13	14												28									
*03:246, 03:282	-		3									13				19									27	28								
*03:248	-		3									13													27	28		31						
*03:330N	Null		3																						27	28	29							
*03:347	-					5	7					13							23						28									
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*03:377	-			3										13										23			28							
*03:395	-	1		3										13	15												27	28						
*11:01:01-01-11:02:07, 11:04-11:05, 11:07:01, 11:09, 11:12:02-11:16, 11:19, 11:22-11:24:02, 11:26-11:27, 11:29-11:30, 11:32:01-11:34, 11:36-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208:02N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q-11:281, 11:283-11:289, 11:291-11:305, 11:307-11:311, 11:313Q-11:324, 11:327-11:347N, 11:349-11:365N, 11:367-11:371, 11:373-11:380, 11:382N-11:403N, 11:405-11:438	A11, Null, -				4						11																							
*11:03:01-01-11:03:01:02, 11:08, 11:20, 11:25:01-11:25:03, 11:31, 11:35, 11:60, 11:158, 11:175, 11:209, 11:348, 11:366, 11:381	A11, -			4							11																27							
*11:06, 11:18, 11:312	A11, -			4							11																	29						
*11:07:02, 11:372	-			4							11						19																	
*11:10, 11:269	A11, -			4			8				11																							
*11:11	-			4							11	12														26								
*11:12:01	A11			4							11																	28						
*11:17, 11:40, 11:223	-			4						10	11																							
*11:21N	Null			4							11																			30	31			
*11:94, 11:112, 11:211, 11:290	-			4							11															26								
*11:113, 11:162	-			4							11			15																				
*11:116, 11:140, 11:199:01-11:199:02, 11:222	-			4							11	13																						
*11:130	-		3	4							11		14						23															
*11:139	-			4	5	7					11																							
*11:155	-			4							11								22									28						
*11:183	-			4			8				11																	27						
*11:191	-			4			8				11																					31		
*11:199:03	-			4							11	13						20		22														
*11:226	-			4							11								22								26	28						
*11:271	-			4						10																	26	28						
*11:282	-	1		4							11		15																					
*11:306	-			4							11		16													25								
*11:325, 24:555 ⁴	-			4	5						11																							
*11:326	-			4							11																26	28						
*11:404	-			4							11																					31		
*23:01:01-01-23:06, 23:08N-23:10:02, 23:12, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:98, 23:100-23:103N, 23:106N-23:112, 23:114-23:127, 23:129	A23, Null, -				5	6																												
*23:07N	Null				5	6																									30	31		
*23:11N	Null				5	6							16																					
*23:13	A23				5	6																							28					
*23:14:01-23:14:02, 23:104-23:105, 24:71, 24:315, 24:392, 24:527 ⁴	A23, A24, -				5	6	7																											
*23:57	-				5														21															
*23:64	-				5	6								17																				
*23:66, 23:99, 23:128, 24:14:01-01-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, 24:291, 24:296, 24:304, 24:316, 24:324, 24:412, 24:481, 24:515, B*53:72, C*04:01:03 ⁴	A24, Null, Cw4, -				5																													
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control

[illegible]

*26:05	A26						8				13						23				30												
*26:07:01-26:07:02	A26							10			13						24				30												
*26:09, 26:91	A26, -						8			10	12																						
*26:16	A26						7			10		13									30												
*26:19	-		4							10		13	14								30												
*26:22	A26						8			10		13	14			21					30												
*26:30	A26						8					13					23	24			29 30												
*26:54	-						8			10	12	13			20						30												
*26:65	-						8			10		13					24				30												
*26:72, 26:184	-		4							10		13									30												
*26:75	-						8			10											30												
*26:78	-						8				11						23				29												
*26:92	-										11	13					24				29 30												
*26:120	-	1					8			10		13									30												
*26:122	-						8			10		13			20						30												
*26:152	-						8			10	11	13									30												
*26:177	-										11	13			18		21	23			29 30												
*26:193	-						8			10	11	13									29 30												
*26:198	-						8			10		13	14								30												
*26:204:01:01-26:204:01:02	-		4				8			10																							
*26:207	-									10		13			18				25		30												
*26:217	-						8			10	12	13									30												
*26:227	-						8					13					23				29 30												
*26:229	-						8				11	12	13			20	23				29 30												
*29:01:01:01-29:01:14, 29:02:01:01-29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40-29:47, 29:50-29:65, 29:68-29:73, 29:75-29:79, 29:81-29:97, 29:99-29:104, 29:107-29:118, 29:120-29:140, 29:142-29:154, 29:157-29:158, 29:160-29:163, 29:165-29:171	A29, Null, -											14						25		28													
*29:01:15, 29:119	-											14								28													
*29:07, 29:49	A29, -			5	6							14						25		28													
*29:13, 29:39	-											14		17				25		28													
*29:14, 29:172	-											14	16					25		28													
*29:19	-											14						25		29													
*29:20, 29:34, 29:66, 29:74, 29:106	-											14						25															
*29:28	-						8					14						25		28													
*29:30, 29:141	-											14																					
*29:48	-											14		18				25		29													
*29:67, 29:156	-			4								14						25		28													
*29:80	-											14				21		25		28													
*29:98	-											14		17				25															
*29:105, 29:164	-								10			14		18				25		28													
*29:155	-											14				21				28													
*29:159	-											14						25		28	29												
*30:01:01:01-30:01:07, 30:01:09-30:01:21, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48-30:49, 30:52-30:54, 30:56, 30:58-30:59N, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:88, 30:91-30:98, 30:102, 30:104, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:126, 30:128-30:131, 30:134-30:138, 30:141-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:164-30:165, 30:167-30:168, 30:170-30:173, 30:176, 30:178N-30:181, 30:184Q-30:185, 30:187, 30:190-30:192, 30:194-30:197N, 30:199, 30:201-30:203, 30:205	A30, Null, -											15						25		28													
*30:01:08, 30:18, 30:41, 30:60	A30, -											15								28													
*30:02:01:01-30:03, 30:07, 30:10, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85, 30:100-30:101Q, 30:107-30:108, 30:119, 30:121N, 30:124, 30:127, 30:133, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156-30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198, 30:200, 30:204N, 30:206	A30, - A30, Null, -											15						25															
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*30:04:01:01-30:04:04, 30:06, 30:09:01:01-30:09:01:02, 30:29, 30:77, 30:90, 30:99, 30:105, 30:117, 30:139, 30:155, 30:160	A30, -													15												25		27					
*30:08:01-30:08:02	A30, -			4										15												25			28				
*30:12:01-30:12:02, 30:31, 30:152, 30:166, B*07:260, C*12:328 ^d	A30, -													15																			
*30:13, 30:44	A30, -		2													15										25				29			
*30:16	A30		2													15										25			28	29			
*30:17, 30:183	A30, -															15										25		27	28				
*30:22	A30		2													15										25							

Negative Control

*30:46	-	2										13	15					25	27	29													
*30:55	-											13	15							28													
*30:89	-	3											15	17				25	27	28													
*30:103	-												15						27														
*30:125, 30:140	-								11				15					25															
*30:132N	Null												15					25		28	30	31											
*30:150	-	4											15					25	27														
*30:163	-				7								15					25		28													
*30:177	-												15	17				25															
*30:186	-											13	15					25		28													
*31:01:02:01-31:02:02, 31:05-31:07, 31:10-31:20, 31:22-31:23, 31:26-31:27, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:88, 31:90-31:108, 31:110-31:114, 31:116-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:199, 31:201N-31:219	A31, Null, -												16					25															
*31:03	A31										12	14	16					25	27														
*31:04:01:01-31:04:02	A31, -									12	14	16					24	25	27														
*31:08, 33:53 ⁴	A31, A24, -			5									16	18				25															
*31:09, 31:28	A31, -												16																				
*31:21	-												16	17				25															
*31:24-31:25	-										13	16						25															
*31:29	-			5	6								16					25															
*31:35	-											15	16	17				25															
*31:41	-												16			21		25															
*31:62	-												16			22		25															
*31:85	-			6									16					25															
*31:89, 31:115	-												16					25		28													
*31:109	-												16	18				25															
*31:123	-									12	14	16					24	25															
*31:176, 31:179	-												16					25	27														
*31:200	-												16					25	26														
*32:01:01:01-32:02, 32:07-32:14, 32:16, 32:18-32:23, 32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:48N, 32:50-32:59, 32:61-32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:115, 32:117N-32:127, 32:129-32:151, 32:153-32:155, 32:157-32:169, 32:171-32:173	A32, Null, -								9						17			25															
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -														17			25															
*32:04	A32, A3	3						9							17				27														
*32:05, 32:79	A32, -			5									16	17	18			25															
*32:06	A32							9								21		25															
*32:15	A32					8	9								17	18		25															
*32:17	-						9								17			25		28													
*32:24, 32:49, 32:116	-						9								17																		
*32:26:01-32:26:02, 32:152	-						9				13				17			25															
*32:42	-						9				14				17			25															
*32:64	-			4			9								17			25															
*32:72	-				6		9								17			25															
*32:93	-						9								17	18		25															
*32:128	-						9						16	17				25															
*32:156	-	1					9								17			25															
*32:170	-				5	6	9								17			25															
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*33:01:01:01-33:01:18, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:162, 33:164-33:183, 33:186, 33:188-33:189, 33:191-33:234	A33, Null, -																	18								25							
*33:13, 33:48	-									10								18								25							
*33:19	-				5	7												18							25								
*33:22	-																	18		21					25								
*33:24	-																	18			23				25			29					
*33:49, 33:144	-																	18							25		27						
*33:51	-					8												18							25								
*33:61, 33:190	-											13						18							25								
*33:119	-					7												18							25								
*33:125, 33:131	-														16			18							25		28						
*33:163, 33:185	-																	18							25								
*33:184	-									11								18			23				25				29				
*33:187	-				5	6												18							25								
*34:01:01:01-34:01:07, 34:05, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 34:27	A34, -						8			11	12																				31		
*34:02:01:01-34:02:07, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24-34:26N, 34:28-34:30N	A34, Null, -						8			11	12																27				31		
*34:03	A34						8			11	12																27						
*34:04	A34						8			11	12		14														27				31		
*34:06, 66:40	A10, -						8			11	12																						
*34:08	A34						8			11	12	13																27			31		
*34:09	-						8			12											24							27	28		31		

Negative Control

[illegible]

[illegible]