

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*30:02:01:01-30:03, 30:07, 30:10, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85, 30:100-30:101Q, 30:107-30:108, 30:119, 30:121N, 30:124, 30:127, 30:133, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156-30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198, 30:200, 30:204N, 30:206	A30, Null, -														15										25								
*30:04:01:01-30:04:04, 30:06, 30:09:01:01-30:09:01:02, 30:29, 30:77, 30:90, 30:99, 30:105, 30:117, 30:139, 30:155, 30:160	A30, -														15										25	27							
*30:08:01-30:08:02	A30, -			4											15										25		28						
*30:12:01-30:12:02, 30:31, 30:152, 30:166, B*07:260, C*12:328 ⁴	A30, -														15										25								
*30:13, 30:44	A30, -		2												15										25						29		
*30:16	A30		2												15										25		28	29					
*30:17, 30:183	A30, -														15										25	27	28						
*30:22	A30		2												15										25								
*30:46	-		2												15										25	27	29						
*30:55	-												13	15																		28	
*30:89	-		3												15		17								25	27	28						
*30:103	-														15																	27	
*30:125, 30:140	-											11			15										25								
*30:132N	Null														15										25		28	30	31				
*30:150	-			4											15										25	27							
*30:163	-						7								15										25		28						
*30:177	-														15		17								25								
*30:186	-												13	15											25		28						
*31:01:02:01-31:02:02, 31:05-31:07, 31:10-31:20, 31:22-31:23, 31:26-31:27, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:88, 31:90-31:108, 31:110-31:114, 31:116-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:199, 31:201N-31:219	A31, Null, -														16										25								
*31:03	A31											12	14	16											25	27							
*31:04:01:01-31:04:02	A31, -											12	14	16										24	25	27							
*31:08, 33:53 ⁴	A31, A24, -				5										16		18								25								
*31:09, 31:28	A31, -														16										25								
*31:21	-														16	17									25								
*31:24-31:25	-												13	16											25								
*31:29	-				5	6									16										25								
*31:35	-													15	16	17									25								
*31:41	-														16						21				25								
*31:62	-														16							22			25								
*31:85	-				6										16										25								
*31:89, 31:115	-														16										25		28						
*31:109	-														16		18								25								
*31:123	-											12	14	16										24	25								
*31:176, 31:179	-														16										25	27							
*31:200	-														16										25	26							
*32:01:01:01-32:02, 32:07-32:14, 32:16, 32:18-32:23, 32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:48N, 32:50-32:59, 32:61-32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:115, 32:117N-32:127, 32:129-32:151, 32:153-32:155, 32:157-32:169, 32:171-32:173	A32, Null, -								9								17								25								
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																17								25								
*32:04	A32, A3		3					9									17										27						
*32:05, 32:79	A32, -				5										16	17	18								25								
*32:06	A32							9													21				25								
*32:15	A32						8	9									17	18							25								
*32:17	-							9									17								25		28						
*32:24, 32:49, 32:116	-							9									17								25								
*32:26:01-32:26:02, 32:152	-							9					13				17								25								
*32:42	-							9					14				17								25								
*32:64	-			4				9									17								25								
*32:72	-					6		9									17								25								
*32:93	-							9									17	18							25								
*32:128	-							9							16		17								25								
*32:156	-							9									17								25								
*32:170	-		1			5	6	9									17								25								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*33:01:01:01-33:01:18, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:162, 33:164-33:183, 33:186, 33:188-33:189, 33:191-33:234	A33, Null, -																									25							
*33:13, 33:48	-									10								18								25							

Negative Control

*69:07	-							8					13						21									28	Negative Control					
*69:08	-							8										21																
*74:01:01:01-74:01:02, 74:01:04-74:03:03, 74:05, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38, 74:40-74:41	A74, Null, -														19								25											
*74:01:03	-											12					19					24	25											
*74:04, 74:43	A74, -																18	19						25										
*74:06, 74:39	A74, -																19						25				29							
*74:07, 74:33	A74, -														17	19						25												
*74:13	-												13	16		19						25				28								
*74:19	-			4												19						25												
*74:21	-															18	19						25				29							
*74:23	-															19						25			27									
*74:30	-											11				19						25												
*74:35	-															19			21			25												
*74:37	-												13			19						25												
*74:42	-															19						25				28								
*74:44	-														16	19						25												
*80:01:01:01-80:01:02, 80:03, 80:05-80:09N	A80, Null, -						6															23												
*80:02	-						6					11																						
*80:04	-						6									17						23												
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -							9																										
B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569, C*07:990, C*07:994	-																					22												
B*15:642	-															17																		
B*18:27	-						6																											
B*35:01:60, B*58:01:24	-			w																														
C*03:365, C*12:131	-					4																												
C*06:187	-																					23												
C*06:247	-																										29							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27		28	29	30	31	32