

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																
*01:01:01:01-01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:01:156, 01:01:159-01:03:03, 01:09:01:01-01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303:01-01:306, 01:308N-01:313, 01:315-01:341, 01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:400N, 01:402, 01:404-01:417N, 01:419-01:426, 01:428-01:438, 01:440-01:458	A1, Null, -																																
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:01:157, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:396Q, 01:403, 01:418N, 01:439	A1, Null, -	1																									26						
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307, 01:342, 01:401	-				4																						26						
*01:01:56	-		1		4								12														26				30		
*01:01:158, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1			4																												
*01:04:01:01N-01:04:01:02N	Null	1			4																						26				30	31	
*01:07, 01:95, 01:289	A1, -	1			4													17									26						
*01:12, 01:19, 01:200, 01:244	A1, -	1																										27					
*01:13, 01:176	A1, -	1			4							11		13		15											26		28				
*01:14, 01:391	A1, -	1																															
*01:15N	Null	1			w																												
*01:17, 01:302	A1, -	1			4																						26		28				
*01:21, 01:126	A1, -	1			4																						26	27					
*01:26, 01:192	A1, -	1			4							11															26						
*01:28, 01:299	-	1			4							11				15											26						
*01:43	-	1			4							10															26						
*01:51	-	1			4				8			10															26						
*01:121	-	1			4															19							26						
*01:134	-	1			4																						26				29		
*01:136, 01:380:01-01-01:380:01:02	-	1			4							11																					
*01:194	-	1			4									13		15											26		28				
*01:229	-	1			4							11				15											26				29		
*01:298	-	1			4						9							17									26						
*01:301Q	-	1			4	5		7																				w					
*01:344	-	1			4				8																								
*01:360	-	1			4																										27		
*01:361N	Null	1			4								12														26						
*01:381	-	1			4				8																		26						
*01:427, 36:02 ⁴	-, A36	1		3											14									22	23								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:155, 02:157:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:1049, 02:1051-02:1094, 02:1096-02:1170, 02:1172	A2, Low A2, A203, A210, Null, -																																
*02:17:02:01-02:17:04	A2, -	2				w																											

Negative Control

*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907, 02:1171	A2, -	2											13				20	22					28											
*02:35:02	A2												13				20	22					28											
*02:46, 02:70, 02:1050	A2, -	2															20	22																
*02:48, 02:129, 02:571	A2, -																20	22																
*02:55	A2, A28	2						8										21	23															
*02:65, 02:407, 02:449, 02:932	A2, A19, -	2														19																		
*02:78	-	w	4														20	22					28											
*02:81, 02:124	A2	2						9																										
*02:135	-	2											13						24						30									
*02:149:01-02:149:02, 02:842, 02:947	-	2																21																
*02:156, 02:338, 02:952	-	2																					27											
*02:169	-	2	4																															
*02:237, 02:809, 02:927	-	2											14	16					23															
*02:243:01-02:243:03	-	2														18		21																
*02:309	-	2										12	14					21		24														
*02:454	-	2										12	14						24															
*02:527, 02:582	-	2					8																											
*02:576, 02:682	-	2																	22															
*02:644	-						8	10										21																
*02:651	-	2															20																	
*02:671	-	2																	23															
*02:741	-	2					8						13				20	21	22							31								
*02:804	-	2				7																												
*02:815	-	2					8											21																
*02:829	-		4					9																										
*02:929	-	2														18					25													
*02:1095	-	2															20	22					28											
Weil No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Weil No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*03:01:01:01-03:01:05, 03:01:07-03:01:13, 03:01:15-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:29, 03:01:31-03:01:34, 03:01:36-03:01:37, 03:01:39-03:01:46, 03:01:48, 03:01:51, 03:01:53-03:01:82, 03:01:84-03:01:89, 03:01:91-03:01:93, 03:01:95-03:01:105, 03:01:107-03:04:03, 03:06-03:07:02, 03:11:01N-03:11:02N, 03:13-03:14, 03:16-03:17:02, 03:20, 03:22-01-03:23:02, 03:25-03:29, 03:31, 03:33, 03:35, 03:37-03:39, 03:44:01-03:49, 03:51:01-03:53, 03:55-03:56:01:02, 03:58, 03:60-03:61, 03:65-03:68N, 03:70-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:97, 03:99-03:103:01, 03:104, 03:106, 03:109-03:110, 03:112-03:121, 03:123:01, 03:124-03:126, 03:128-03:134, 03:136-03:139, 03:141, 03:143-03:151, 03:154:01-03:164, 03:166, 03:168N-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197:02N, 03:201-03:202, 03:206-03:207, 03:209-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:232-03:233, 03:235-03:245, 03:247:01-03:247:02, 03:250-03:251, 03:253-03:259, 03:261-03:262N, 03:264-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:287, 03:289:01-03:321, 03:323N-03:325, 03:327-03:329N, 03:331, 03:333-03:342N, 03:344:01-03:346, 03:348-03:376, 03:378-03:381N, 03:383-03:394, 03:396-03:397, 03:399-03:412, 03:415, 03:417-03:427, 03:430-03:446, 03:448N-03:450, 03:453-03:462, 03:465-03:467, 03:469-03:474,	A3, Null, -																																	
*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:69N, 03:127, 03:140, 03:153, 03:165, 03:231:02, 03:234Q, 03:249, 03:288, 03:322,	A3, -	3												13														28						
*03:01:19, 03:103:02	-		3										12	13						24							27	28						
*03:01:23, 03:01:83, 03:01:94, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211, 03:252, 03:267, 03:463N	Null, -		3																								27	28						
*03:01:30, 03:01:35, 03:01:50, 03:01:90, 03:01:106, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208, 03:225, 03:326, 03:332, 03:452	A3, -													13														27	28					
*03:01:38, 03:123:02, 03:171, 03:428	-		3											13	14													27	28					
*03:01:49, 03:204, 03:215, 03:414, 03:468	-													13														28						
*03:08:01:01-03:08:01:02, 03:36N, 03:107, 03:176, 03:398, 03:413	A3, Null, -		3																									27						
*03:09	A3		3											13														27	28	29				
*03:12	A3		3	4																								27	28					
*03:15, 03:19, 03:30	A3		3		5									13														27	28					
*03:18, 03:135	-		4											13													26	28						
*03:21N	Null		3											13														27	28		30	31		
*03:24, 03:475	A3, -		3				8							13														27	28					
*03:32, 03:479	-		3													17												27						

Negative Control

Negative Control

*26:16	A26							7			10		13																30						
*26:19	-			4							10		13 14																30						
*26:22	A26								8		10		13 14					21											30						
*26:30	A26								8				13							23 24							29	30							
*26:54	-								8		10		12 13				20											30							
*26:65	-								8		10		13								24							30							
*26:72, 26:184	-			4							10		13															30							
*26:75	-								8		10																	30							
*26:78	-								8			11								23								29							
*26:92	-											11	13								24						29	30							
*26:120	-		1						8		10		13															30							
*26:122	-								8		10		13				20											30							
*26:152	-								8		10 11		13															30							
Well No.				1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.				1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																		
*26:177	-											11	13					18				21	23						29	30					
*26:193	-								8		10 11		13																29	30					
*26:198	-								8		10		13 14																	30					
*26:204:01:01-26:204:01:02	-			4					8		10																								
*26:207	-										10		13				18								25				30						
*26:217, 26:231	-								8		10		12 13																30						
*26:227	-								8				13											23				29	30						
*26:229	-								8			11 12	13						20				23					29	30						
*29:01:01:01-29:01:14, 29:02:01:01-29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40-29:47, 29:50-29:65, 29:67:02-29:73, 29:75-29:79, 29:81-29:97, 29:99-29:104, 29:107-29:118, 29:120-29:140, 29:142-29:154, 29:157-29:158, 29:160-29:163, 29:165-29:171, 29:173, 29:175Q, 29:177-29:184, 29:187-29:192	A29, Null, -													14												25			28						
*29:01:15, 29:119	-													14															28						
*29:07, 29:49	A29, -					5	6							14												25			28						
*29:13, 29:39	-													14			17									25			28						
*29:14, 29:172	-													14		16										25			28						
*29:19	-													14												25			29						
*29:20, 29:34, 29:66, 29:74, 29:106	-													14												25									
*29:28, 29:176, 29:185	-								8					14												25			28						
*29:30, 29:141	-													14												25									
*29:48	-													14			18									25			29						
*29:67:01, 29:156	-			4										14												25			28						
*29:80	-													14										21		25			28						
*29:98	-													14			17									25									
*29:105, 29:164	-										10			14			18									25			28						
*29:155	-													14								21				25			28						
*29:159	-													14												25			28	29					
*29:174	-													14									22			25			28						
*29:186	-													14												25		27	28						
*30:01:01:01-30:01:07, 30:01:09-30:01:28, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48-30:49, 30:52-30:54, 30:56, 30:58-30:59N, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:88, 30:91-30:98, 30:102, 30:104, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:126, 30:128-30:131, 30:134-30:138, 30:141-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:164-30:165, 30:167-30:168, 30:170-30:173, 30:176, 30:178N-30:181, 30:184Q-30:185, 30:187, 30:190-30:192, 30:194-30:197N, 30:199, 30:201-30:203, 30:205, 30:207, 30:209N-30:211, 30:215, 30:217-30:218, 30:220Q-30:224, 30:226, 30:228, 30:230	A30, Null, -													15												25			28						
*30:01:08, 30:18:01-30:18:02, 30:41, 30:60	A30, -														15														28						
*30:02:01:01-30:03, 30:07, 30:10, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85, 30:100-30:101Q, 30:107-30:108, 30:119, 30:121N, 30:124, 30:127, 30:133, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156-30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198, 30:200, 30:204N, 30:206, 30:208N, 30:213-30:214, 30:216, 30:219N, 30:222, 30:227	A30, Null, -														15											25									
*30:04:01:01-30:04:04, 30:06, 30:09:01:01-30:09:01:02, 30:29, 30:77, 30:90, 30:99, 30:105, 30:117, 30:139, 30:155, 30:160, 30:212	A30, -														15											25		27							
*30:08:01-30:08:02	A30, -			4											15											25			28						
*30:12:01-30:12:02, 30:31, 30:152, 30:166, B*07:260, C*12:328 ⁴	A30, -														15																				
*30:13, 30:44	A30, -			2											15											25			29						
*30:16	A30			2											15											25		28	29						
*30:17, 30:183	A30, -														15											25		27	28						

Negative Control

Negative Control

*30:22	A30	2													15												25						
*30:46	-	2													15												25	27	29				
*30:55	-												13		15														28				
*30:89	-		3												15	17										25	27	28					
*30:103	-														15													27					
*30:125, 30:140	-									11					15											25							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																
*30:132N	Null														15											25		28		30	31		
*30:150	-			4											15											25	27						
*30:163	-						7								15											25		28					
*30:177	-														15	17										25							
*30:186	-												13	15												25		28					
*31:01:02:01-31:02:02, 31:05-31:07, 31:10-31:20, 31:22-31:23, 31:26-31:27, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:88, 31:90-31:108, 31:110-31:114, 31:116-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:199, 31:201N-31:219, 31:221-31:230	A31, Null, -														16										25								
*31:03	A31											12	14	16												25	27						
*31:04:01:01-31:04:02	A31, -											12	14	16										24	25	27							
*31:08, 33:53 ⁴	A31, A24, -				5										16	18									25								
*31:09, 31:28	A31, -														16																		
*31:21	-														16	17									25								
*31:24-31:25	-												13	16											25								
*31:29	-				5	6									16										25								
*31:35	-														15	16	17								25								
*31:41	-														16						21				25								
*31:62	-														16							22			25								
*31:85	-					6									16										25								
*31:89, 31:115	-														16										25		28						
*31:109, 33:249 ⁴	-														16	18									25								
*31:123	-											12	14	16										24	25								
*31:176, 31:179, 31:220	-														16										25	27							
*31:200	-														16										25	26							
*32:01:01:01-32:02, 32:07-32:14, 32:16, 32:18-32:23, 32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:48N, 32:50-32:59, 32:61-32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:115, 32:117N-32:127, 32:129-32:151:02, 32:153-32:155, 32:157-32:169, 32:171-32:186	A32, Null, -							9									17								25								
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																17								25								
*32:04	A32, A3		3					9									17									27							
*32:05, 32:79	A32, -				5										16	17	18								25								
*32:06	A32							9									17				21				25								
*32:15	A32						8	9									17	18							25								
*32:17	-							9									17								25		28						
*32:24, 32:49, 32:116	-							9									17																
*32:26:01-32:26:02, 32:152	-							9				13					17								25								
*32:42	-							9					14				17								25								
*32:64	-			4				9									17								25								
*32:72	-					6		9									17								25								
*32:93	-							9									17	18							25								
*32:128	-							9							16	17									25								
*32:156	-	1						9									17								25								
*32:170	-				5	6		9									17								25								
*33:01:01:01-33:01:20, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:162, 33:164-33:183, 33:186, 33:188-33:189, 33:191-33:236, 33:238-33:242, 33:244-33:248N, 33:250-33:251, 33:253-33:270	A33, Null, -																18								25								
*33:13, 33:48	-							10									18								25								
*33:19	-				5	7											18								25								
*33:22, 33:252	-																18			21				25									
*33:24	-																18					23		25			29						
*33:49, 33:144	-																18							25		27							
*33:51	-							8									18							25									
*33:61, 33:190	-											13					18							25									
*33:119, 33:243	-					7											18							25									
*33:125, 33:131	-														16	18								25		28							
*33:163, 33:185, 33:237	-																18																
*33:184	-									11							18					23		25				29					
*33:187	-				5	6											18							25									
*34:01:01:01-34:01:07, 34:05, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 34:27	A34, -						8			11	12																				31		
*34:02:01:01-34:02:07, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24-34:26N, 34:28-34:30N, 34:32	A34, Null, -						8			11																							

Negative Control

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*34:03	A34								8			11	12																27					
*34:04	A34								8			11	12		14														27			31		
*34:06, 66:40 ⁴	A10, -								8			11	12																					
*34:08	A34								8			11	12	13															27			31		
*34:09	-								8				12												24			27	28			31		
*34:19	-				4							11	12																					
*34:22	-								8			11	12										22					27						
*34:31	-						5		8			11	12																			31		
*36:01:01:01-36:01:02, 36:03, 36:05-36:14	A36, -	1																					22											
*36:04	A36	1			4																		22				26							
*43:01-43:03	A43, Null, -										10		12	13													26				30			
*66:01:01:01-66:01:01:08, 66:01:03-66:01:07, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41, 66:44-66:47	A66, A26, Null, -								8			11		13																		30		
*66:01:02	-								8			11		13																				
*66:02:01:01-66:02:01:03, 66:16, 66:21, 66:25-66:26Q, 66:34, 66:43, 66:48	A66, -								8				12												24				28					
*66:03:01:01-66:03:01:03, 66:28N, 66:39N	A10, -, Null								8				12												24									
*66:06	A66								8			11		13									21									30		
*66:09	-								8			11		13	14								21									30		
*66:10	-								8			11			w																	30		
*66:12	-								8					13											24				28			30		
*66:19	-								8			11	12	13																		30		
*66:23	-				4							11		13																		30		
*66:42	-											11		13											24							30		
*68:01:01:01-68:01:17, 68:01:19-68:01:56, 68:01:58-68:02:19, 68:02:21-68:02:22, 68:02:24, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30, 68:32-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:73-68:83, 68:85-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179, 68:181N-68:205N, 68:207-68:211, 68:213N-68:217, 68:219-68:226, 68:228-68:229, 68:231-68:248, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:293Q, 68:295-68:299	A68, A28, Null, -								8												20								28					
*68:01:18, 68:01:57, 68:02:20, 68:02:23, 68:03:01:01-68:04:02, 68:31, 68:72, 68:90, 68:93, 68:97, 68:180, 68:206, 68:212	A68, -								8													20												
*68:05, 68:15, 68:20, 68:255	A68, -								8													20			23						29			
*68:08:02, 68:105:01:01-68:105:01:02, 68:157, 68:276Q	A68, -								8													20	21							28				
*68:11N, 68:41, 68:230	Null, -								8													20		22						28				
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8													20									28			
*68:19, 68:130:02	A68, -								8						14							20									28			
*68:25, 68:262	A68, -								8													20									28			
*68:29	A68								8												18	20	21								28			
*68:45, 68:117	-						7		8													20									28			
*68:71	-								8					13								20									28			
*68:84	-								8		10											20									28			
*68:103:01-68:103:02	-								8													20							27	28				
*68:109	-								8													20			23									
*68:130:01	-								8				12									20					26			28				
*68:136	-								8													20			23						28			
*68:178	-				4																	20									28			
*68:218, 68:258	-								8													20									28			
*68:227	-								8		10											20												
*68:249	-					5			8													20												
*68:250	-								8											18		20	21								28			
*68:271	-								8			11										20										28		
*68:294	-								8				12									20				24					28			
*69:01:01:01-69:01:05, 69:03-69:06, 69:09	A69, -								8														21								28			
*69:02	-								8			11											21											
*69:07	-								8					13									21								28			
*69:08	-								8														21											
*74:01:01:01-74:01:02, 74:01:04-74:03:03, 74:05, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38, 74:40-74:41, 74:45, 74:47N, 74:49-74:50	A74, Null, -																				19					25								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*74:01:03	-												12								19				24	25								
*74:04, 74:43	A74, -																																	

Negative Control

*74:19	-			4													19						25											
*74:21	-																18	19					25			29								
*74:23	-																19						25		27									
*74:30	-									11							19						25											
*74:35	-																19		21				25											
*74:37	-										13						19						25											
*74:42	-																19						25			28								
*74:44	-															16	19						25											
*74:46	-																19						25											
*80:01:01:01-80:01:02, 80:03, 80:05-80:09N	A80, Null, -				6																	23												
*80:02	-				6					11																								
*80:04	-				6												17					23												
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -								9																									
B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569, C*07:990, C*07:994	-																					22												
B*15:01:83, B*35:01:60, B*58:01:24	-			w																														
B*15:642	-																17																	
B*18:27	-							6																										
C*03:365, C*12:131, C*12:388	-			4																														
C*06:187	-																						23											
C*06:247	-																									29								
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control