

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A Allele ^{1,2,4}	ser ³																																	
*01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:01:156, 01:01:159-01:03:03, 01:09:01:01-01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303:01-01:306, 01:308N-01:313, 01:315-01:341, 01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:400N, 01:402, 01:404-01:417N, 01:419-01:426, 01:428-01:438, 01:440-01:463, 01:465, 01:467-01:478	A1, Null, -																																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:01:157, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:396Q, 01:403, 01:418N, 01:439	A1, Null, -	1																									26							
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307, 01:342, 01:401	-				4																						26							
*01:01:56	-		1		4								12														26					30		
*01:01:158, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1			4																													
*01:04:01:01N-01:04:01:02N	Null	1			4																						26					30	31	
*01:07, 01:95, 01:289	A1, -	1			4													17									26							
*01:12, 01:19, 01:200, 01:244	A1, -	1																											27					
*01:13, 01:176	A1, -	1			4							11		13		15											26			28				
*01:14, 01:391	A1, -	1																																
*01:15N	Null	1			w																													
*01:17, 01:302	A1, -	1			4																						26			28				
*01:21, 01:126	A1, -	1			4																						26	27						
*01:26, 01:192	A1, -	1			4							11															26							
*01:28, 01:299	-	1			4							11				15											26							
*01:43	-	1			4						10																26							
*01:51	-	1			4				8		10																26							
*01:121	-	1			4																19						26							
*01:134	-	1			4																						26					29		
*01:136, 01:380:01-01:380:01:02	-	1			4							11																						
*01:194	-	1			4									13		15											26			28				
*01:229	-	1			4							11				15											26				29			
*01:298	-	1			4						9							17									26							
*01:301Q	-	1			4	5		7																				w						
*01:344	-	1			4				8																									
*01:360	-	1			4																								27					
*01:361N	Null	1			4								12														26							
*01:381	-	1			4				8																		26							
*01:427, 36:02 ⁵	A36, -	1		3											14								22	23										
*01:464	-	1			4																		22											
*01:466	-	1		3																				23					27					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:155, 02:157:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:1049, 02:1051-02:1094, 02:1096-02:1170, 02:1172-02:1206	A2, A203, A210, Low A2, Null, -																																	
*02:17:02:01-02:17:04	A2, -		2					w																										
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907, 02:1171	A2, -		2											13							20		22						28					
*02:35:02	A2													13							20		22						28					
*02:46, 02:70, 02:1050	A2, -		2																		20		22											
*02:48, 02:129, 02:571	A2, -																				20		22											
*02:55	A2, A28		2					8														21		23										
		ative Control																																

Negative Control

Relative Control

*02:65, 02:407, 02:449, 02:932	A2, A19, -		2														19																Neg	
*02:78	-		w	4													20	22											28					
*02:81, 02:124	A2		2						9																									
*02:135	-		2										13												24						30			
*02:149:01-02:149:02, 02:842, 02:947:01-02:947:02	-		2																	21														
*02:156, 02:338, 02:952	-		2																										27					
*02:169	-		2	4																														
*02:237, 02:809, 02:927	-		2										14	16										23										
*02:243:01-02:243:03	-		2														18			21														
*02:309	-		2									12	14							21				24										
*02:454	-		2									12	14										24											
*02:527, 02:582	-		2				8																											
*02:576, 02:682	-		2																															
*02:644	-						8	10												21														
*02:651	-		2																20															
*02:671	-		2																					23										
*02:741	-		2				8						13							20	21	22									31			
*02:804	-		2				7																											
*02:815	-		2				8													21														
*02:829	-			4			9																											
*02:929	-		2														18									25								
*02:1095	-		2																	20	22									28				
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*03:01:01-03:01:05, 03:01:07-03:01:13, 03:01:15-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:29, 03:01:31-03:01:34, 03:01:36-03:01:37, 03:01:39-03:01:46, 03:01:48, 03:01:51, 03:01:53-03:01:82, 03:01:84-03:01:89, 03:01:91-03:01:93, 03:01:95-03:01:105, 03:01:107-03:01:130, 03:01:132-03:04:03, 03:06-03:07:02, 03:11:01N-03:11:02N, 03:13-03:14, 03:16-03:17:02, 03:20, 03:22-01-03:23:02, 03:25-03:29, 03:31, 03:33, 03:35, 03:37-03:39, 03:44:01-03:49, 03:51:01-03:53, 03:55-03:56:01:02, 03:58, 03:60-03:61, 03:65-03:68N, 03:70-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:97, 03:99-03:103:01, 03:104, 03:106, 03:109-03:110, 03:112-03:121, 03:123:01, 03:124-03:126, 03:128-03:134, 03:136-03:139, 03:141, 03:143-03:151, 03:154:01-03:164, 03:166, 03:168N-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197:02N, 03:201-03:202, 03:206-03:207, 03:209-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:232-03:233, 03:235-03:245, 03:247:01-03:247:02, 03:250-03:251, 03:253-03:259, 03:261-03:262N, 03:264-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:287, 03:289:01-03:321, 03:323N-03:325, 03:327-03:329N, 03:331, 03:333-03:342N, 03:344:01-03:346, 03:348-03:376, 03:378-03:381N, 03:383-03:394, 03:396-03:397, 03:399-03:412, 03:415, 03:417-03:427, 03:430-03:446, 03:448N-03:450, 03:453-03:462, 03:465-03:467, 03:469-03:474, 03:476-03:478, 03:480-03:484, 03:486-03:497, 03:499-03:503	A3, Null, -																																	
*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:69N, 03:127, 03:140, 03:153, 03:165, 03:231:02, 03:234Q, 03:249, 03:288, 03:322, 03:416	A3, -			3											13																28			
*03:01:19, 03:103:02	-			3									12	13											24				27	28				
*03:01:23, 03:01:83, 03:01:94, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211, 03:252, 03:267, 03:463N, 03:498	Null, -			3																										27	28			
*03:01:30, 03:01:35, 03:01:50, 03:01:90, 03:01:106, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208, 03:225, 03:326, 03:332, 03:452	A3, -														13															27	28			
*03:01:38, 03:123:02, 03:171, 03:428	-			3											13	14														27	28			
*03:01:49, 03:204, 03:215, 03:414, 03:468	-														13																28			
*03:01:131, 03:34, 03:62:01-03:62:01:02, 03:263, 03:447, 03:451, 03:464Q, 03:504N	Null, -			3											13															27				
*03:08:01-03:08:01:02, 03:36N, 03:107, 03:176, 03:398, 03:413	A3, Null, -			3																										27				
*03:09	A3			3											13															27	28	29		
*03:12	A3			3	4																										27	28		
*03:15, 03:19, 03:30	A3			3		5									13																27	28		
*03:18, 03:135	-			3	4										13															26	28			
*03:21N	Null			3											13																27	28		30
*03:24, 03:475	A3, -			3				8							13																27	28		
*03:32, 03:479	-			3																											27			
*03:41, 03:382	-			3													17														27			
*03:43, 03:82, 03:186	-			3											13	15	17														27	28		
*03:50	-			3				8							13																27	28		
*03:63	-			3									11		13										23						27			
*03:75, 03:177	-														13																23	28		
*03:88	-			3	4																										23	27		
*03:89:01	-			2	3																											27		
*03:89:02, 03:108, 03:172, 03:198	-			3																												27		29
*03:95, 03:343	-														13	14	16								23						27		28	
*03:152, 03:273	-			3		5																										27		
*03:187	-														13																	27		28
*03:219, 03:429	-			3													17															27		28

Negative Control

Negative Control

[illegible]

Regenerative Control

Negative Control