

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele <sup>1,2</sup>	ser <sup>3</sup>																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:04N, 01:06-01:07, 01:09-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:14-01:15N, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N-01:179N	A1, Null, -	1																							
*01:01:56	-	1			4									w											
*01:12, 01:19	A1, -	1		3																					
*01:13	A1	1		4							11		13		15										
*01:21, 01:126	A1, -	1		3	4																				
*01:28	-	1		4							11				15										
*01:43	-	1		4							10														
*01:51	-	1		4					8		10														
*01:95	-	1		4														17							
*01:121	-	1		4																19					
*01:136, 11:94, 11:112, 11:211 <sup>5</sup>	-	1		4							11														
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:570, 02:572-02:573	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																							
*02:17:01-02:17:03	A2, -	2						w														21			
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2	2												13							20	22			
*02:35:02	A2													13							20	22			
*02:46, 02:70	A2	2																			20	22			
*02:48, 02:129	A2, -																				20	22			
*02:55	A2/A28	2						8														21	23		
*02:65, 02:407, 02:449	A2/A19, -	2																		19					
*02:78	-	w		4																	20	22			
*02:108, 02:110, 02:268, 02:300, 02:303	A2, -	2																				21			
*02:135	-	2												13											
*02:169	-	2		4																					
*02:237	-	2													14		16								23
*02:243:01-02:243:02	-	2																		18		21			
*02:309, 02:454	-	2											12		14										
*02:338	-	2		3																					
*02:527	-	2						8																	
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44:01-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:171, 03:174-03:175, 03:179-03:185, 03:188-03:197N, 03:200Q-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:221	A3, A3/blank, Null, -			3										13											
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*03:01:19	-			3									12	13											

Negative Control

*03:01:23, 03:08, 03:32, 03:36N, 03:57, 03:59, 03:72, 03:89, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205, 03:211, 03:219	A3, Null, -			3																						
*03:12	A3			3	4																					
*03:15, 03:19, 03:30	A3			3		5							13													
*03:18, 03:135	-	1			4								13													
*03:24, 03:50	A3, -			3				8					13													
*03:41	-			3																				23		
*03:43, 03:82, 03:186	-			3									13	15		17										
*03:63	-			3							11		13												23	
*03:75, 03:177	-												13												23	
*03:88, 11:130 <sup>5</sup>	-			3	4						11														23	
*03:95	-												13	14		16										23
*03:152, 24:92 <sup>4</sup>	-			3		5																				
*03:187	-												13												22	
*03:215	-												13													
*11:01:01:01-11:02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:59, 11:61- 11:93, 11:95-11:111, 11:114-11:129, 11:131- 11:138, 11:140-11:154, 11:156-11:157, 11:159- 11:161, 11:163-11:165, 11:167-11:174, 11:176- 11:182Q, 11:184-11:190, 11:192-11:198, 11:200- 11:208N, 11:210N, 11:212-11:215N	A11, Null, -				4						11															
*11:03, 11:20, 11:25, 11:60, 11:158, 11:175, 11:209	A11, -			3	4						11															
*11:10, 11:191	A11, -			4				8			11															
*11:11	-			4							11	12														
*11:17, 11:40	-			4						10	11															
*11:113, 11:162	-			4							11			15												
*11:139	-			4	5	7					11															
*11:155	-			4							11														22	
*11:166	-			4		6					11															
*11:183	-			3	4			8			11															
*11:199	-				4						11	13														
*23:01:01-23:01:12, 23:01:14-23:01:18, 23:03:01- 23:13, 23:15-23:56, 23:58-23:65, 23:67-23:69	A23, Null, -					5	6																			
*23:01:13	-					5	6														20			22		
*23:02	A23					5	6																21			
*23:14:01-23:14:02, 24:24, 24:71 <sup>4</sup>	A23, A9, A24, -					5	6	7																		
*23:57, 23:66, 24:94, 24:138, 24:188, 24:228 <sup>4</sup>	-					5																	21			
*24:02:01:01-24:05, 24:07:01-24:11N, 24:13:02, 24:17, 24:20-24:21:02, 24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56, 24:58-24:63, 24:66- 24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95- 24:113, 24:115-24:137, 24:139-24:166, 24:168- 24:187, 24:189-24:203, 24:205-24:206, 24:208- 24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:290, 24:292-24:295, 24:297-24:303N, 24:305	A24, Low A24, A2403/A24, A9, A24/A9, Null, -					5		7																		
*24:06, 24:13:01, 24:22, 24:82, 24:167, 24:207	A24, -					5		7															21			
*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:114, 24:222N, 24:291, 24:296, C*04:01:03 <sup>5</sup>	A24, Cw4, Null, -					5																				
*24:18, 24:204, 24:213	A24/A3, -					5		7																	23	
*24:19, 24:44	A9					5		7			11															
*24:243	-					5		7													17					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*25:01:01-25:01:08, 25:02-25:04, 25:07-25:16, 25:18-25:29, 25:31	A25, Null, -							8	9				13													
*25:01:09	-							8	9																	
*25:05	A25							8	9			12	13									20				
*25:06	A25							8	9			12														
*25:17	-								9					13												
*25:30	-							8	9					13									21			

Negative Control



*36:02	A36	1	3																	22	23				
*36:04	A36	1		4																22					
*43:01	A43								10	12	13														
*66:01:01, 66:01:03, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:20, 66:22	A66, A26, -						8		11	13															
*66:01:02	-						8		11																
*66:02-66:03, 66:16, 66:21	A66, A10, -						8			12															
*66:06	A66						8		11	13										21					
*66:09	-						8		11	13	14									21					
*66:10	-						8		11	w															
*66:12, 66:15	-						8			13															
*68:01:01:01-68:04, 68:06-68:08:01, 68:09-68:10, 68:12, 68:13:02-68:14, 68:16-68:19, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:132	A68, A28, Null, -						8													20					
*68:05, 68:15, 68:20, 68:109	A68, -						8													20	23				
*68:08:02, 68:105	A68, -						8													20	21				
*68:11N	Null						8													20	22				
*68:13:01, 68:66	A68, -			4			8													20					
*68:25	A68						8													19	20				
*68:29	A68						8										18			20	21				
*68:45, 68:117	-					7	8													20					
*68:71	-						8				13									20					
*68:84	-						8		10											20					
*68:130	-						8			12										20					
*69:01, 69:03	A69, -						8														21				
*69:02	-						8		11												21				
*74:01-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24	A74, Null, -																			19					
*74:04, 74:21	A74, -																			18	19				
*74:07	A74																			17	19				
*74:13	-										13									16	19				
*74:19	-			4																	19				
*74:23	-			3																	19				
*80:01:01:01-80:01:01:02, 80:03	A80, -						6														23				
*80:02	-						6			11															
B*18:27	-						6																		
C*07:404	-									11															
C*12:131	-			4																					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control