

HLA-A low screening 2017 1F4 R04

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele <sup>1,2</sup>	ser <sup>3</sup>																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:04N, 01:06-01:07, 01:09-01:01:11N, 01:15N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:180-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:211, 01:213, 01:215-01:217	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:14, 01:30, 01:79, 01:99, 01:147Q, 01:149, 01:159, 01:167, 01:178N-01:179N, 01:205, 01:208Q, 01:214	A1, Null, -	1																							
*01:01:56	-	1			4									w											
*01:12, 01:19, 01:200	A1, -	1	3																						
*01:13, 01:176	A1, -	1			4						11			13		15									
*01:21, 01:126	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211 <sup>5</sup>	A1, -	1			4						11														
*01:28	-	1			4						11					15									
*01:43	-	1			4						10														
*01:51	-	1			4				8		10														
*01:95	-	1			4														17						
*01:121	-	1			4																19				
*01:145	-	1												w											
*01:194	-	1			4									13		15									
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:570, 02:572-02:579, 02:581, 02:583-02:616, 02:618Q-02:641, 02:643N, 02:645-02:654	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -		2																						
*02:17:01-02:17:03	A2, -		2						w														21		
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -		2											13							20		22		
*02:35:02	A2													13							20		22		
*02:46, 02:70	A2		2																		20		22		
*02:48, 02:129, 02:571	A2, -																				20		22		
*02:55	A2/A28		2						8													21		23	
*02:65, 02:407, 02:449	A2/A19, -		2																	19					
*02:78	-		w		4																20		22		
*02:108, 02:110, 02:268, 02:300, 02:303, 02:617	A2, -		2																				21		
*02:135	-		2						8					13											
*02:169	-		2		4																				
*02:237	-		2													14		16							23
*02:243:01-02:243:02	-		2																	18		21			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:309, 02:454	-		2									12		14											
*02:338	-		2	3																					
*02:527, 02:582	-		2						8																
*02:644	-							8		10													21		

Negative Control







HLA-A low screening 2017 1F4 R04

*68:05, 68:15, 68:20, 68:109, 68:136	A68, -						8											20		23	Negative Control				
*68:08:02, 68:105, 68:157	A68, -						8											20	21						
*68:13:01, 68:66, 68:134	A68, -			4			8											20							
*68:19	A68						8					14						20							
*68:25	A68						8										19	20							
*68:29	A68						8										18	20	21						
*68:45, 68:117	-						7	8										20							
*68:71	-						8						13					20							
*68:84	-						8		10									20							
*68:130	-						8				12							20							
*69:01:01-69:01:02, 69:03	A69, -						8												21						
*69:02	-						8			11									21						
*74:01:01-74:01:02, 74:01:04-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:28	A74, Null, -																	19							
*74:01:03	-										12							19							
*74:04, 74:21	A74, -																	18	19						
*74:07	A74																	17	19						
*74:13	-											13		16				19							
*74:19	-				4													19							
*74:23	-				3													19							
*80:01:01-80:01:01:02, 80:03	A80, -						6													23					
*80:02	-						6			11															
B*07:81, B*08:52, B*18:67, B*38:41, B*51:185, B*53:05, B*53:16, B*53:33	B53, -								9																
B*18:27	-						6																		
C*06:187	-																			23					
C*07:404	-									11															
C*07:481, C*07:504	-																			?					
C*12:131	-				4																				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24