

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54
HLA-A allele ^{1,2,3}																																																						
*01:01:01:01, 01:01:02-01:01:22, 01:01:24-01:01:37, 01:01:39-01:01:47, 01:01:49-01:01:72, 01:37, 01:67:01-01:67:02, 01:70, 01:74-01:76, 01:78-01:80, 01:82, 01:84-01:85, 01:88, 01:90-01:91, 01:93-01:94, 01:96-01:97, 01:99, 01:102, 01:105, 01:108, 01:110-01:112, 01:114, 01:116-01:122, 01:124-01:125, 01:128-01:129, 01:131, 01:133, 01:135, 01:138-01:140, 01:146, 01:148, 01:151-01:154, 01:156-01:158, 01:161, 01:163, 01:165-01:170, 01:172, 01:174-01:175, 01:180-01:182, 01:184-01:185, 01:187, 01:189 ⁵	1	2															18																																					
*01:01:01:02N	1	2						8								16	18																																					
*01:01:23, 01:73, 01:89, 01:149, 01:159		2															18																																					
*01:01:38L	1	2															18											28																										
*01:01:48	1	2													15		18																																					
*01:02	1		3														18																																					
*01:03		2		4													18																																					
*01:04N	1	2			5												18																																					
*01:06	1	2					7										18																																					
*01:07	1	2						8									18													30					35																			
*01:08		2							9								18																																					
*01:09	1	2				6											18																																					
*01:10	1	2								10							18		20																																			
*01:11N	1	2									11						18																																					
*01:12		2											13		15		18						24																															
*01:13	1	2									11			14			18																																					
*01:14		2													15		18																																					
*01:15N		2														16	18																																					
*01:16N	1	2															17	18																																				
*01:17	1	2												14			18																																					
*01:18N	1	w															18	19																																				
*01:19		2											13		15		18						24	25																														
*01:20			3														18			21																																		
*01:21	1	2											13				18		20																																			
*01:22N	1	2															18					22																																
*01:23, 01:83:01-01:83:02	1	2						8									18																																					

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]