

INTERPRETATION TABLE
HLA-A*02 high resolution SSP typing
Amplification patterns of the HLA-A*0201 to *9295 alleles

HLA-A allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8					
Well No.										1	1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4	4
HLA-A allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8					
*0120																																						+					
*0144																																											
*030103, 0323																																						+					
*030112																																						+					
*0309																																						+					
*0317																																						+					
*0344																																						+					
*1106, 1118																																						+					
*1116, 1135																																											
*230101, 230102, 2305-2307N, 2309, 2311N, 2313, 2315- 2322, 2424																																											
*2302																																						W					
*230301, 230302, 2907																																											
*2304																																											
*2308N																																											
*2310																																						W					
*2312																																						W					
*2314																																											
*24020101-240215, 240217-240226, 2404, 2405, 2407- 2409N, 2411N, 2417, 2419, 2420, 2425- 2427, 2429, 2431, 2432, 2434-2440N, 2443-2445N, 2447- 2450, 2452, 2453, 2457, 2458, 2460N- 2464, 2466-2471, 2474, 2476-2486N, 2488, 2490N, 2495- 2498, 3319																																											
*240216, 2406, 2454, 2487																																											
*240301, 240302, 2433, 2475																																											
*2410																																						+					
*241301																																											
*241302																																											
*2414, 2493																																						+					
*2415, 2441, 2451, 2492																																											
*2418																																											
*2423																																											
*2428, 2430, 2442, 2489																																											
*2446																																						W					
*2455																																											
*2456																																											
*2459																																											
*2472																																											
*2473																																											
*2491																																											
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4	

HLA-A allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8						
Well No.										1	1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4	4
HLA-A allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8						
*2494																																												
*2499																																												
*2603, 2606, 2621, 2630																																												
*260702, 330102, 3308, 3309, 7404																																												
*2610																																												
*2622, 6609																																												
*2636																																												
*2919, 3208, 3324																																												
*300101, 300102, 3008, 301101, 301102, 3014L, 3015, 3017-3020, 3023, 3024, 3026, 3030, 3031																																												
*3013																																												
*3016																																												
*3022																																												
*3315																																												
*3322																																												
*680101-680107, 680301-6804, 6807, 6810, 6811N, 6813, 6816, 6817, 6819, 6821, 6822, 6824, 6829, 6832, 6835- 6839, 6841, 6843, 6845-6847																																												
*68020101-680202, 6818N, 6827, 6828, 6831, 6834, 6840																																												
*6805, 6820																																												
*6806																																												
*680801, 680802																																												
*6809																																												
*6812																																												
*6814																																												
*6815																																												
*6823																																												
*6825																																												
*6826																																												
*6830																																												
*6833																																												
*6842																																												
*6844																																												
*6848																																												
*6901																																												
*7406																																												
*8001																																												
B*1567																																												
B*9573																																												
Well No.																																												
HLA-A allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8						

INTERPRETATION TABLE

HLA-A*02 high resolution SSP typing

Amplification patterns of the HLA-A*0201 to *9295 alleles

9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6		HLA-A allele ²
4	5	5	5	5	5	5	5	6	6	6	6	6	6	6	6	7	7	7	7	7	7	7	7	7	7	8	8	8	8	8	8	8	9	9	9	9	9	Well No.	
9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	HLA-A allele ²	
																																					*0120		
																																					*0144		
																																					*030103, 0323		
																																					*030112		
																																					*0309		
																																					*0317		
																																					*0344		
																																					*1106, 1118		
																																					*1116, 1135		
																																					*230101, 230102, 2305-2307N, 2309, 2311N, 2313, 2315- 2322, 2424		
																																					*2302		
																																					*230301, 230302, 2907		
																																					*2304		
																																					*2308N		
																																					*2310		
																																					*2312		
																																					*2314		
																																					*24020101-240215, 240217-240226, 2404, 2405, 2407- 2409N, 2411N, 2417, 2419, 2420, 2425- 2427, 2429, 2431, 2432, 2434-2440N, 2443-2445N, 2447- 2450, 2452, 2453, 2457, 2458, 2460N- 2464, 2466-2471, 2474, 2476-2486N, 2488, 2490N, 2495- 2498, 3319		
																																					*240216, 2406, 2454, 2487		
																																					*240301, 240302, 2433, 2475		
																																					*2410		
																																					*241301		
																																					*241302		
																																					*2414, 2493		
																																					*2415, 2441, 2451, 2492		
																																					*2418		
																																					*2423		
																																					*2428, 2430, 2442, 2489		
																																					*2446		
																																					*2455		
																																					*2456		
																																					*2459		
																																					*2472		
																																					*2473		
																																					*2491		
4	5	5	5	5	5	5	5	5	6	6	6	6	6	6	6	7	7	7	7	7	7	7	7	7	8	8	8	8	8	8	8	8	9	9	9	9	9	Well No.	

