

## HLA-A23 2020 7K4

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36			
<b>HLA-A allele</b> <sup>1,2,3</sup>																																							
*23:01:01:01-23:01:15, 23:01:17-23:01:28, 23:76-23:78, 23:80, 23:85-23:95, 23:97-23:102	1	2																18			21																		
*23:01:16	1	2													15			18			21																		
*23:02		2											13					w			?							?											
*23:03:01	1		3																		?							?											
*23:03:02	1		3																		21																		
*23:04	1													14				18			21																		
*23:05	1	2							9									18										28											
*23:06	1	2			5													18			21																		
*23:07N	1	2				6												18			21																		
*23:08N, 23:22 <sup>4</sup>	1	2					7											18			?							?											
*23:09, 23:26 <sup>4</sup>	1	2						8										18			21																		
*23:10, 23:23 <sup>4</sup>	1	2								10								18			?								?										
*23:11N	1	2									11							18			21																		
*23:12	1	2										12						18			?							?											
*23:13, 23:33 <sup>4</sup>	1	2															17	18			?							?											
*23:14:01	1	2													15	16		18			?							?											
*23:14:02	1	2														16		18			?							?											
*23:15	1	2																18	19		21																		
*23:16, 23:29 <sup>4</sup>	1	2																18		20	?							?											
*23:17:01:01-23:17:02, 23:84N, 23:96	1	2																18										28											
*23:18	1	2																18			21															34			
*23:19N	1	2																18			21	22																	
*23:20	1	2																18			?		23					?											
*23:21	1	2																18			?			24				?											
*23:24	1	2											13								?							?											
*23:25	1	2	3															18			?							?											
*23:27	1	2																18			?							?									35		
*23:28	1	2		4														18			?							?											
*23:30	1	2											13					18			21																		
*23:31	1	2																18			?	22						?											
*23:32	1	2									11							18		20	?							?											
*23:34, 23:57	1	2																			?							27	?										
*23:35	1	2																18			?							27	?										
*23:36	1	2																18			?						26	?											
*23:37:01-23:37:02	1	2																18			?							?	29										
*23:38N, 23:56 <sup>4</sup>	1	2																18			21														30				
*23:39	1	2					7	8										18			?							?											
*23:40	1	2																18			?			24			?	29											

Negative Control

HLA-A23 2020 7K4

*23:41-23:42 <sup>4</sup>	1	2														18		?			25		?														
*23:43	1									10						18		?					?														
*23:44	1	2														18		?					?			31											
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>	
*23:45	1	2																18		?	22	23						?									
*23:46	1	2																18	19	20	?							?									
*23:47	1	2													15			18		?								?	29								
*23:48	1	2						8										18		?								?	29								
*23:49	1	2																18		?			24					?							35		
*23:50-23:52, 23:54-23:55, 23:60-23:64, 23:67, 23:71-23:75, 23:81-23:82	1	2																18		?								?									
*23:53	1	2											14					18		?	22						27	?									
*23:58	1	2																18		?		23						?						34			
*23:59	1	2																18		?								?			32						
*23:65	1	2																18		?	21												33				
*23:66		2											13							?								?									
*23:68	1	2												14				18		?	21																
*23:70	1		3											14						?	22						27	?									
*23:79	1																	18		?	21																
*23:83	1		3											14						?									?								
*01:02, 01:20, 01:188, 01:190, 01:307								8																													
*01:04:01:01N-01:04:01:02N, 03:21N, 11:21N						6																															
*01:88:02-01:88:03, 01:236, 02:36-02:37, 02:54, 02:255, 02:417, 02:723, 02:848, 02:883, 11:209, 68:26, 68:65, 68:115, 68:131		2																																			
*01:203, 02:671, 11:166, 31:85, 32:72, 80:01:01:01-80:06																																			32		
*01:301Q			3							10				14	16							22						27									
*02:01:22, 02:06:04, 02:40:01-02:40:02, 02:51, 02:99:02, 02:130, 02:171:01, 29:101:01, 68:08:01:01-68:08:01:02, 68:168, B*40:323:02, B*51:136, C*07:204:01, C*07:482																		18																			
*02:17:02:01, 02:17:03-02:17:04	1													14	w													27									
*02:17:02:02, 02:108, 02:110, 02:268, 02:300, 02:303, 02:617, 02:657	1													14															27								
*02:19, 24:14:01:01-24:14:01:04, 24:53, 24:57, 24:412, 24:458		2																												28							
*02:41, 02:80, 02:117, 02:289:01, 02:304, 02:454, 02:872, 26:10																																					
*02:82N, 02:773N								7																													

Negative Control

HLA-A23 2020 7K4

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36				
*02:221, 29:15, 31:78																									25															
*02:680													13	14																										
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>				
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>				
*02:804, 24:02:01:01-24:02:09, 24:02:11-24:02:41, 24:02:43-24:02:69, 24:02:71-24:02:96, 24:02:98-24:02:122, 24:02:124-24:02:132, 24:04-24:05:02, 24:08-24:09N, 24:20:01:01-24:20:01:02, 24:28-24:29, 24:31, 24:35-24:37, 24:44, 24:48N, 24:50, 24:54, 24:56, 24:58, 24:68, 24:72, 24:79, 24:81, 24:86N, 24:90:01N, 24:95, 24:116, 24:122, 24:132N, 24:135:01, 24:136, 24:142:01, 24:143, 24:148, 24:152, 24:163N, 24:172:02, 24:183N, 24:194, 24:215, 24:225:02, 24:226:02, 24:231, 24:233, 24:235, 24:238, 24:248-24:252N, 24:254, 24:256, 24:259-24:266, 24:268-24:272, 24:278N, 24:280, 24:286, 24:293, 24:313:01, 24:314, 24:321, 24:331, 24:341, 24:352-24:353, 24:356, 24:358-24:359N, 24:361-24:364, 24:367-24:373, 24:389N, 24:394, 24:396N-24:402, 24:404-24:405, 24:407-24:411, 24:414, 24:417-24:420, 24:422-24:431, 24:433N-24:440, 24:442-24:443, 24:447Q-24:451, 24:454-24:456N, 24:462, 24:464, 24:466-24:470, 24:472-24:476		2													16																									
*03:72, 11:88, 29:37, 29:56, 30:01:01:01-30:01:15, 30:11:01-30:11:02, 30:14L-30:20, 30:23-30:26, 30:30-30:31, 30:35-30:43, 30:48-30:49, 30:52-30:54, 30:58-30:60, 30:62-30:63, 30:65, 30:72-30:74, 30:78N, 30:81-30:83, 30:86-30:89, 30:91-30:98, 30:102, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:125-30:126, 30:128-30:131, 30:134-30:138, 30:140-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161																	17																							
*03:347	1		3											14	16							22						27												
*11:139			3						w					14	16							22						27												
*24:02:10, 24:02:123, 24:460, 24:465		2													16																									
*24:02:42, 24:02:70, 24:23, 24:357N															16														28											
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>				

Negative Control

HLA-A23 2020 7K4

<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>			
*24:02:97, 24:26-24:27, 24:34, 24:38-24:40N, 24:43, 24:45N, 24:47, 24:49, 24:55, 24:59-24:60N, 24:63, 24:67, 24:69-24:71, 24:73-24:74:02, 24:76, 24:78, 24:80, 24:82-24:85, 24:88-24:89, 24:90:02N-24:91, 24:93, 24:96-24:105, 24:107-24:109, 24:111-24:113, 24:115, 24:117-24:121, 24:123-24:124, 24:126-24:127, 24:130, 24:133-24:134, 24:135:02, 24:137, 24:139-24:141, 24:142:02, 24:144-24:147, 24:149-24:151, 24:153-24:157, 24:159-24:162, 24:164-24:166, 24:168-24:172:01, 24:173-24:182, 24:184-24:187, 24:189-24:193, 24:195-24:203, 24:205-24:206, 24:209, 24:212-24:214, 24:216-24:218, 24:220-24:221, 24:223-24:225:01, 24:226:01, 24:227, 24:229-24:230, 24:232N, 24:236-24:237, 24:239-24:247, 24:253, 24:257-24:258, 24:274-24:277, 24:279, 24:281-24:282, 24:284, 24:287, 24:292, 24:295, 24:301, 24:303N, 24:305-24:307, 24:310:01-24:312N, 24:317-24:320, 24:322-24:323N, 24:325-24:329, 24:332-24:338, 24:342-24:346, 24:348, 24:350-24:351, 24:354, 24:360, 24:365-24:366, 24:375-24:380, 24:382-24:385, 24:388N, 24:391, 24:393, 24:395, 24:413, 24:415-24:416, 24:421, 24:432, 24:445N-24:446, 24:452, 33:19																16																							
		2																																					
*24:03:01:01-24:03:04, 24:22, 24:441, 24:459, 24:461														14		16																							
*24:06, 24:87, 24:285, 24:381		2												13		16																							
*24:07:01:01-24:07:03, 24:294Q, 24:406, 24:453, 24:457, 24:477		2														16	17																						
*24:10:01:01-24:10:01:02											w			14		16																							
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>			
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>			
*24:10:02, 24:210, 24:300, 24:340										w				14		16																							
*24:11N		2				6										16																							
*24:13:01	1	2														16													27	?									
*24:13:02	1	2														15	16		18																				

Negative Control



HLA-A23 2020 7K4

*24:302	2													16						24						28	29												
*24:308	2							9							16		19	20									?												
*24:313:02	2	3													16												?												
*24:315	1													14	15	16	18										?												
*24:339													13	14	15	16	17									?	29												
*24:347:02	2														15	16	17									?	29												
*24:355	1	2														16										28													
*24:387														14	16	17																							
*24:392	1	2														16	18									?													
*24:403	2						8									16										28	29												
*24:444	2															16										28													35
*24:463	2															16				21							24												
*25:11, 32:08												12																											
*26:16, 33:119																16																							
*29:07, 29:49	1	3												14								22				27													
*30:06, C*04:361																																						35	
*30:22							7	8																															
*30:56																	17																				32		
*30:75																	17	19	20																				
*30:104							7	8									17																						
*30:132N						6											17																						
*31:29, 33:187	1	3												14												22													
*31:36						5																																	
*31:67-31:68, 32:28, 32:66, 33:32:01																																							
*32:05, 32:79																																							
*68:45, 68:117																16	17																						
*68:51																																							
B*18:27	1	3							10					14	15																								
B*51:293										11																													
C*04:344																																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36			

Negative Control