

HLA-A23 2023 8R3.xlsx

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	
HLA-A allele^{1,2,3}																																							
*23:01:01:01-23:01:15, 23:01:17-23:01:37, 23:10:02, 23:76-23:78, 23:80, 23:85-23:95, 23:97-23:98, 23:100-23:103N, 23:106N, 23:111, 23:114-23:119, 23:121-23:124, 23:126-23:127, 23:129	1	2																18			21																	37	
*23:01:16	1	2													15			18			21																	37	
*23:02		2											13					w			21																	37	
*23:03:01	1		3																		?							?										37	
*23:03:02:01-23:03:02:02	1		3																		21																	37	
*23:04	1													14				18			21																		37
*23:05	1	2						9										18										28											37
*23:06	1	2			5													18			21																		37
*23:07N	1	2				6												18			21																		37
*23:08N, 23:22 ⁴	1	2					7											18			?							?											37
*23:09, 23:26 ⁴	1	2						8										18			21																		37
*23:10:01, 23:23 ⁴	1	2								10								18			?								?										37
*23:11N	1	2									11							18			21																		37
*23:12	1	2										12						18			21																		37
*23:13	1	2															17	18			21																		37
*23:14:01	1	2													15	16		18			?							?											37
*23:14:02	1	2														16		18			?							?											37
*23:15	1	2																18	19		21																		37
*23:16, 23:29 ⁴	1	2																18		20	?							?											37
*23:17:01:01-23:17:03, 23:84N, 23:96, 23:107Q, 23:112, 23:125	1	2																18										28											37
*23:18	1	2																18			21													34				37	
*23:19N	1	2																18			21	22																37	
*23:20	1	2																18			?		23					?											37
*23:21	1	2																18			?			24				?											37
*23:24	1	2											13								?							?											37
*23:25	1	2	3															18			?							?											37
*23:27	1	2																18			?							?							35				37
*23:28	1	2		4														18			?							?											37
*23:30	1	2											13					18			21																		37
*23:31	1	2																18			?	22						?											37
*23:32	1	2									11							18		20	?							?											37
*23:33	1	2															17	18			?							?											37
*23:34, 23:57	1	2																			?							27	?										37
*23:35	1	2																18			?							27	?										37
*23:36	1	2																18			?						26	?											37
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	
*23:37:01-23:37:02	1	2																18			?							?	29										37
*23:38N, 23:56 ⁴	1	2																18			21											30							37

Negative Control

HLA-A23 2023 8R3.xlsx

*23:39	1	2					7	8							18		?					?												37						
*23:40	1	2													18		?		24			?	29											37						
*23:41-23:42 ⁴	1	2													18		?			25		?												37						
*23:43	1														18		?					?												37						
*23:44	1	2													18		?					?		31										37						
*23:45	1	2													18		?	22	23			?											37							
*23:46	1	2													18	19	20	?				?											37							
*23:47	1	2													18		?					?	29										37							
*23:48	1	2						8							18		?					?	29										37							
*23:49	1	2													18		?		24			?									35		37							
*23:50-23:52, 23:54-23:55, 23:60-23:64, 23:67, 23:71-23:75, 23:81-23:82	1	2													18		?					?											37							
*23:53	1	2													18		?	22				27	?										37							
*23:58	1	2													18		?	23				?									34		37							
*23:59	1	2													18		?					?			32								37							
*23:65	1	2													18		21									33							37							
*23:66		2													13		?					?																		
*23:68	1	2													14		18	21															37							
*23:70	1		3												14		?	22				27	?										37							
*23:79, 23:108N, 23:120	1														18		21																37							
*23:83	1		3												14		?					?											37							
*23:99	1	2													18		21																							
*23:104	1	2													15	16	18					28										36								
*23:105		2														16	21																37							
*23:109	1	2													18		?																37							
*23:110	1								w						14		18	21															37							
*23:128																	21																							
*01:02:01:01-01:02:02, 01:20, 01:188, 01:190, 01:307, 01:412, 01:420N								8																																
*01:04:01:01N-01:04:01:02N, 03:21N, 11:21N									6																															
*01:88:02-01:88:03, 01:236, 02:36-02:37, 02:54, 02:255, 02:417, 02:723, 02:848, 02:883, 02:949, 11:209, 68:26, 68:65, 68:115, 68:131, 69:09		2																																						
*01:203, 02:671, 11:166, 31:85, 32:72, 80:01:01:01-80:09N																										32														
*01:301Q			3						10					14	16								22																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
*02:01:22, 02:06:04, 02:40:01-02:40:02, 02:51, 02:99:02, 02:130, 02:171:01, 29:101:01, 68:08:01:01-68:08:01:02, 68:168, B*40:323:02, B*51:136, B*53:60, C*07:204:01, C*07:482																		18																						
*02:17:02:01-02:17:04	1													14	w													27												
*02:19, 24:14:01:01-24:14:01:04, 24:53, 24:57, 24:412, 24:458, 24:481, 24:557		2																																						

Negative Control

Control

HLA-A23 2023 8R3.xlsx

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38									
*02:41, 02:80, 02:117, 02:289:01, 02:304, 02:454, 02:872, 26:10, 68:262 *02:82N, 02:773N *02:108, 02:110, 02:268, 02:300, 02:303, 02:617, 02:657, 02:1054 *02:221, 29:15, 30:162, 31:78 *02:680, 02:963							7							14												23																					Negative
*02:804, 24:02:01:01-24:02:09, 24:02:11-24:02:41, 24:02:43-24:02:69, 24:02:71-24:02:96, 24:02:98-24:02:122, 24:02:124-24:02:161, 24:04-24:05:02, 24:08-24:09N, 24:20:01:01-24:20:02, 24:28-24:29, 24:31, 24:35-24:37, 24:44, 24:48N, 24:49:02-24:50, 24:54, 24:56:01-24:56:02, 24:58, 24:68, 24:72, 24:79, 24:81, 24:86N, 24:90:01N, 24:93, 24:95, 24:97, 24:116, 24:120-24:122, 24:132N, 24:135:01, 24:136, 24:142:01, 24:143, 24:148, 24:152, 24:163N, 24:172:02-24:172:03, 24:183N, 24:194, 24:215, 24:225:01-24:225:02, 24:226:02, 24:231, 24:233, 24:235, 24:238, 24:244, 24:248-24:252N, 24:254, 24:256-24:257, 24:259-24:266, 24:268-24:272, 24:278N, 24:280, 24:286, 24:293, 24:310:02, 24:313:01, 24:314, 24:321, 24:331, 24:341, 24:352-24:353, 24:356, 24:358-24:359N, 24:361-24:364, 24:367-24:373, 24:389N, 24:394, 24:396N-24:402, 24:404-24:405, 24:407-24:411, 24:414, 24:417-24:420, 24:422-24:431:02, 24:433N-24:440, 24:442-24:443, 24:445N, 24:447Q-24:451, 24:454-24:456N, 24:462, 24:464, 24:466-24:470, 24:472-24:476, 24:478-24:480, 24:482, 24:484-24:487, 24:490-24:493, 24:496-24:498, 24:500-24:502, 24:504-24:505, 24:507-24:509, 24:511-24:513Q, 24:516-24:523, 24:525-24:526, 24:529N-24:530, 24:532-24:534, 24:536Q-24:539, 24:543, 24:545-24:550, 24:552-24:554, 24:556N, 24:558-24:561:02, 24:564-24:565, 24:567N-24:568N, 24:571-24:573, 24:575-24:585, 24:587-24:589														16												28																					Negative Control

HLA-A23 2023 8R3.xlsx

*24:03:01:01-24:03:04, 24:22, 24:441, 24:459:01, 24:461, 24:483, 24:524, 24:540, 24:563, 24:570														14	16																						28				
*24:06, 24:87, 24:285, 24:381	2													13	16																						?				
*24:07:01:01-24:07:04, 24:294Q, 24:406, 24:453, 24:457, 24:477, 24:510, 24:541, 24:544, 24:566	2														16	17																					28				
*24:10:01:01-24:10:01:02, 24:494-24:495										w					14	16																					28				
*24:10:02, 24:210, 24:300, 24:340										w					14	16																					?				
*24:11N	2				6										16																						28				
*24:13:01	1	2													16																						27	?			
*24:13:02	1	2													15	16		18																				28			
*24:15, 24:51-24:52, 24:64, 24:92, 24:114, 24:222N, 24:291, 24:296, 24:304, 24:316, 24:324	2																																					?			
*24:17:01:01-24:17:01:02, 24:41, 24:503, 24:535	2														14	16							22														27	28			
*24:18	1														14	16																						28			
*24:19, 24:131, 24:288, 24:290, 24:347:01, 24:386	2															16	17																					?			
*24:21:01, 24:21:03			3													16																							28		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38			
*24:21:02, 24:75, 24:110, 24:158N, 24:219, 24:283, 24:349, 24:390																16																									?
*24:24	1	2														16	17	18																							37
*24:25	2								9							16																								28	
*24:30, 24:309	2											12				16																								28	
*24:32	2															16											26													28	
*24:33															14	16																									
*24:42	2											12				16																							?		
*24:46	2									10						16																								?	
*24:61	2															16																							28	30	
*24:62	2															16																								?	
*24:66	2															16				19	20																		?		
*24:71	2															16																								28	
*24:77	2			4												16																								?	
*24:94	1														14																								27	?	
*24:106	2															16																								28	
*24:125, 24:204, 24:299, 24:374															14	16																								?	
*24:128	2															16						20																		28	
*24:129	2								8							16																								28	
*24:138															13	14																								?	
*24:167, 24:289															13	14	16																							?	
*24:188, 24:228	1	2																																						27	?
*24:207:01	1														14	16																								27	28
*24:207:02	1														14	15	16		18																					28	
*24:208:01				3											14	16																								?	
*24:208:02:01-24:208:02:02, 24:499				3											14	16																								27	28
*24:234	2														15	16																								28	29
*24:255	2															16																									28

Negative Control

HLA-A23 2023 8R3.xlsx

*24:267	2														16														28				34						
*24:273	2														16														28	29									
*24:297	2														16		20												?										
*24:298	2														16														?				33						
*24:302	2														16							24						28	29										
*24:308	2							9							16		19	20										?											
*24:313:02	2	3													16													?											
*24:315	1													14	15	16	18											?						37					
*24:330	2														16							22						?											
*24:339													13	14	15	16	17											?	29										
*24:347:02	2														15	16	17											?	29										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	
*24:355	1	2													16														28										
*24:387														14	16	17																							
*24:392	1	2													16		18												?									37	
*24:403		2						8							16													28	29										
*24:444		2													16								24					28							35				
*24:459:02														14	15	16													28	29									
*24:463		2													16							21																	
*24:488														14	16							22						27	28										
*24:489			3												16	17													28										
*24:506									w					14	16													28	29										
*24:515	1													14	16						21						27												
*24:527														14	16														28									37	
*24:528		2							10						16	17													28										
*24:531		2												13	16														28										
*24:551																													28										
*24:562			3						10						16														28										
*24:569	1	2												15	16	17	18											28									37		
*24:574		2															17												28										
*24:586N		2										11			16														28										
*25:11, 32:08												12																											
*25:77, 32:05, 32:79																												26											
*26:16, 33:119															16																								
*29:07, 29:49	1		3											14								22						27									37		
*30:06, C*04:361																								24												35			
*30:22								7	8																														
*30:56																		17																	32				
*30:75																		17		19	20																		
*30:104								7	8									17																					
*30:132N																																							
*30:163, 68:45, 68:117						6												17																					
*30:163, 68:45, 68:117															16	17																							
*31:29, 32:170, 33:187	1		3											14									22															37	
*31:67-31:68, 32:28, 32:66, 32:158, 33:32:01																																							
*68:51																			w																				

Negative Control

HLA-A23 2023 8R3.xlsx

<i>B*18:27</i>	1		3						10			14	15																							37		
<i>B*51:293</i>										11						20																						
<i>C*04:344</i>																						24																
<i>C*04:446</i>																																					30	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38