

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-A allele^{1,2,3}																
*25:01:01, 25:01:03-25:01:09, 25:14, 25:20-25:23, 25:25-25:26, 25:28-25:29, 25:31-25:32	1	2	3											14		
*25:01:02, 25:33	1	2	3													
*25:02	1	2		4	5									14		
*25:03	1	2	3			6								14		
*25:04	1	2	3				7							14		
*25:05	1	2	3					8						14		
*25:06		2	3				7							14		
*25:07	1	2	3							10				14		
*25:08	1	2	3						9					14		
*25:09	1	2	3								11			14		
*25:10	1	2	3									12		14		
*25:11	1	2	3										13	14		
*25:12N	1	2	3				7	8						14		
*25:13	1	2													15	
*25:15, 25:17	1													14		
*25:16	1	2	3					8	9					14		
*25:18, 32:62⁴		2	3											14		
*25:19:01, 25:30	1	2	3			6	7							14		
*25:19:02	1		3			6	7							14		
*25:24	1	2	3	4										14		
*25:27:01-25:27:02	1	2	3					8		10				14		
*01:01:01:01-01:01:27, 01:01:29-01:01:55, 01:01:58-01:01:61, 01:01:63-01:04N, 01:06, 01:08-01:12, 01:14-01:16N, 01:18N-01:33, 01:35-01:50, 01:52:01N-01:70, 01:72-01:99, 01:101-01:104, 01:106-01:143, 01:146-01:175, 01:177-01:189, 01:191-01:193, 01:195-01:199, 02:346, 02:427, 03:41, 11:17, 11:19, 11:40, 11:98, 11:121, 26:01:21, 26:04, 26:34, 26:78, 26:103, 33:13, 36:01-36:05, 74:10, 80:01:01:01-80:03			3													
*01:01:56, 01:145, 26:29, 26:49	w		3													
*01:13, 01:17, 01:176, 03:63, 03:88, 11:01:01:01-11:01:67, 11:01:69-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:95, 11:97, 11:99N-11:105, 11:107-11:115N, 11:117-11:120, 11:122-11:139, 11:141-11:158, 11:160-11:177, 11:179-11:235Q, 29:66, C*07:335			4													
*01:51			3			6										
*02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:78, 02:103, 02:580, 03:242, 23:13, 24:07:01-24:07:02, 24:19, 24:24, 24:112, 24:131, 24:288, 24:290, 24:294Q, 68:01:01:01-68:01:31, 68:02:01:01-68:02:09, 68:02:11-68:02:12, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:43:01, 68:44-68:56, 68:58-68:70, 68:72-68:89, 68:91-68:108, 68:110-68:132, 68:135-68:142N, 69:01, 69:03					5											
*02:55, 03:24						6										
*02:135, 26:30, 26:65	1														15	
*02:309						6	7					12			15	
*02:454						6	7	8				12			15	
*03:01:19, 31:04						6	7				11	12			15	
*11:116, 11:140, 26:33, 66:23	1			4												
*23:12, 24:30, 24:42, 24:309													13			
*24:44, 24:109, 24:260			w													
*26:01:01:01-26:01:20, 26:01:23-26:01:27, 26:01:29-26:03:01, 26:05-26:07:02, 26:10-26:12, 26:15-26:17, 26:21-26:27, 26:31-26:32, 26:35-26:42, 26:45-26:46, 26:50-26:53, 26:55-26:63, 26:66, 26:68-26:72, 26:74-26:77, 26:79-26:80, 26:82, 26:84-26:85, 26:87-26:90, 26:92-26:102, 26:104-26:111, 26:113-26:120, 43:01, 66:05, 66:15	1		3													
*26:01:22, 26:01:28, 26:64, 26:83, 31:24, 33:61	1															
*26:08, 26:48, 26:67, 26:81, 26:86	1		3				7									
*26:09, 26:91			3				7									
*26:13, 26:19, 66:01:01, 66:01:03, 66:04, 66:07-66:09, 66:11, 66:13, 66:17, 66:19-66:20, 66:24	1			4	5											
*26:14, 26:18, 26:28	1		3								11					
*26:20	1		3			6										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*26:43:01-26:43:02	1		3			6	7									
*26:47	1		3						9							
*26:54	1		3					8								
*26:73			3				7				11					
*26:112	1		3			6	7				11					
*31:03			3			6	7				11	12				
*32:01:01-32:01:13, 32:01:15-32:02, 32:04, 32:06-32:07, 32:09-32:14, 32:16-32:37, 32:39-32:59, 32:61, 32:63-32:70, 32:72-32:77, 32:81-32:84		2														
*32:08		2											13			
*32:15		2				6										

Negative Control

*34:01:01-34:01:02, 34:05, 34:12				4	5		7					12					
*34:02:01-34:02:04, 34:04, 34:08, 34:13				4	5	6	7					11	12				
*34:03				4	5	6	7					11	12				
*34:06				4	5	6	7					11					
*34:07					5	6	7					12					
*34:09					5	6	7					12				15	
*34:10N				4	5							12					
*34:11, 66:01:02, 69:02				4	5												
*34:14				4			7										
*66:02, 66:16, 66:21					5											15	
*66:03																15	
*66:06	1			4	5	6	7										
*66:10	w			4	5												
*66:12	1				5											15	
*66:14	1			4	5			8		10							
*66:18	1			4	5				9								
*66:22	1			4	5						11						
*68:71					5	6											
*74:01:03						6	7				11					15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	

Negative Control