

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
<b>HLA-A allele<sup>1,2,3</sup></b>																
*25:01:01:01-25:01:01:02, 25:01:03-25:01:11, 25:14, 25:18, 25:20-25:23, 25:25-25:26, 25:28-25:29, 25:31-25:32, 25:34-25:35, 25:37-25:41, 25:45	1	2	3											14		
*25:01:02, 25:33	1	2	3													
*25:02	1	2		4	5									14		
*25:03	1	2	3			6								14		
*25:04, 25:44	1	2	3				7							14		
*25:05	1	2	3					8						14		
*25:06		2	3				7							14		
*25:07	1	2	3							10				14		
*25:08	1	2	3						9					14		
*25:09	1	2	3								11			14		
*25:10	1	2	3									12		14		
*25:11	1	2	3										13	14		
*25:12N	1	2	3				7	8						14		
*25:13	1	2													15	
*25:15, 25:17	1													14		
*25:16	1	2	3					8	9					14		
*25:19:01, 25:30	1	2	3			6	7							14		
*25:19:02	1		3			6	7							14		
*25:24	1	2	3	4										14		
*25:27:01-25:27:02	1	2	3					8		10				14		
*25:36, 25:43, 32:62 <sup>†</sup>		2	3											14		
*25:42N	1	2	3				7		9					14		
*01:01:01:01-01:01:27, 01:01:29-01:01:56, 01:01:58-01:01:61, 01:01:63-01:04N, 01:06, 01:08-01:12, 01:14-01:16N, 01:18N-01:33, 01:35-01:50, 01:52:01N-01:70, 01:72-01:99, 01:101-01:104, 01:106-01:143, 01:145-01:175, 01:177-01:189, 01:191-01:193, 01:195-01:203, 01:205-01:241, 02:346, 02:427, 03:41, 11:17, 11:19, 11:40, 11:98, 11:121, 26:04, 26:29, 26:49, 26:75, 26:78, 33:13, 36:01-36:05, 74:10, 80:01:01:01-80:03			3													
*01:13, 01:17, 01:176, 03:63, 03:88, 11:01:01:01-11:01:67, 11:01:69-11:01:72, 11:01:74-11:01:75, 11:01:77-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:95, 11:97, 11:99N-11:105, 11:107-11:115N, 11:117-11:120, 11:122-11:139, 11:141-11:158, 11:160-11:177, 11:179-11:249, 11:251N-11:264, 29:66, C*07:335				4												
*01:51, 02:644			3			6										
*02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:78, 02:103, 02:580, 03:242, 23:13, 24:07:01-24:07:02, 24:19, 24:24, 24:112, 24:131, 24:288, 24:290, 24:294Q, 24:339, 24:347:01-24:347:02, 24:387, 68:01:01:01-68:02:12, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:56, 68:58:01-68:70, 68:72-68:89, 68:91-68:108, 68:110-68:169, 69:01:01:01-69:01:02, 69:03					5											
*02:55, 03:24						6										
*02:135, 26:30, 26:65	1														15	
*02:309						6	7					12			15	

Negative Control

*02:454						6	7	8				12			15	
*03:01:19, 31:04						6	7					11	12		15	
*11:116, 11:140, 26:33	1			4												
*23:12, 24:30, 24:42, 24:309, 24:379															13	
*24:44, 24:109, 24:260			w													
*26:01:01:01-26:01:21, 26:01:23-26:01:27, 26:01:29-26:03:01, 26:05-26:07:02, 26:10-26:12, 26:15-26:17, 26:21-26:27, 26:31-26:32, 26:34-26:42, 26:45-26:46, 26:50-26:53, 26:55-26:63, 26:66, 26:68-26:72, 26:74, 26:76-26:77, 26:79-26:80, 26:82, 26:84-26:85, 26:87-26:90, 26:92-26:111, 26:113-26:123, 26:125-26:129, 26:131, 26:133-26:145N, 43:01, 66:05, 66:15	1		3													
*26:01:22, 26:01:28, 26:64, 26:83, 26:132, 31:24-31:25, 33:61	1															
*26:08, 26:48, 26:67, 26:81, 26:86, 26:124	1		3					7								
*26:09, 26:91			3					7								
*26:13, 26:19, 66:01:01:01-66:01:03, 66:04, 66:07-66:09, 66:11, 66:13, 66:17, 66:19-66:20, 66:23-66:24, 66:27N	1			4	5											
*26:14, 26:18, 26:28	1		3									11				
*26:20	1		3					6								
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>
*26:43:01-26:43:02	1		3			6	7									
*26:47	1		3						9							
*26:54	1		3					8								
*26:73, 26:146	1		3				7				11					
*26:112	1		3			6	7				11					
*26:130	1		3					8		10						
*31:03			3			6	7				11	12				
*31:123, 74:01:03						6	7				11				15	
*32:01:01:01-32:01:13, 32:01:15-32:02, 32:04, 32:06-32:07, 32:09-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:59, 32:61, 32:63-32:70, 32:72-32:77, 32:81-32:92N, 32:94-32:103			2													
*32:08		2													13	
*32:15		2				6										
*32:26:01-32:26:02	1	2														
*34:01:01-34:01:02, 34:05, 34:12, 34:16				4	5		7									
*34:02:01-34:02:04, 34:04, 34:08, 34:13, 34:15				4	5	6	7					12				
*34:03				4	5	6	7				11	12				
*34:06				4	5	6	7				11					
*34:07					5	6	7					12				
*34:09					5	6	7					12			15	
*34:10N				4	5							12				
*34:11, 66:10, 69:02				4	5											
*34:14				4			7									
*34:17				4	5		7				11					
*66:02, 66:16, 66:21, 66:25-66:26Q					5										15	
*66:03, 66:28N															15	
*66:06	1			4	5	6	7									
*66:12	1				5										15	

Negative Control

*66:14	1			4	5			8		10							
*66:18	1			4	5				9								
*66:22	1			4	5						11						
*68:71					5	6											
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	