

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
<b>HLA-A allele<sup>1,2,3</sup></b>																
*25:01:01:01-25:01:01:12, <b>25:01:03</b> -25:01:21, 25:14, 25:18, 25:20-25:23, 25:25-25:26, 25:28-25:29, 25:31-25:32, 25:34-25:35, 25:37-25:41, 25:45, 25:47-25:48, 25:51, 25:53-25:54, 25:56-25:58, 25:60-25:63, 25:66-25:70, 25:72-25:74, 25:76, 25:79-25:80, 25:84	1	2	3											14		
*25:01:02, 25:33, 25:55	1	2	3													
<b>*25:02</b>	1	2		4	5									14		
<b>*25:03</b>	1	2	3			6								14		
*25:04, 25:44	1	2	3				7							14		
*25:05	1	2	3					8						14		
*25:06		2	3				7							14		
*25:07	1	2	3							10				14		
<b>*25:08</b>	1	2	3						9					14		
<b>*25:09</b>	1	2	3								11			14		
*25:10	1	2	3									12		14		
<b>*25:11</b>	1	2	3										13	14		
<b>*25:12N</b>	1	2	3				7	8						14		
*25:13	1	2													15	
*25:15, 25:17, 25:71, 25:77	1													14		
<b>*25:16</b>	1	2	3					8	9					14		
<b>*25:19:01</b> , 25:30	1	2	3			6	7							14		
*25:19:02	1		3			6	7							14		
<b>*25:24</b>	1	2	3	4										14		
<b>*25:27:01-25:27:02</b>	1	2	3					8		10				14		
*25:36, 25:43, 25:50, 25:52, 25:65, 32:62 <sup>4</sup>		2	3											14		
*25:42N	1	2	3				7		9					14		
*25:46, 25:78, 25:83	1		3											14		
*25:49N	1	2	3						9	10				14		
*25:59, 01:315 <sup>4</sup>			3		5											
*25:64, 32:26:01-32:26:02, 32:152 <sup>4</sup>	1	2														
*25:75	1	2		4												
*25:81, 26:01:22, 26:01:28, 26:64, 26:83, 26:132, 26:148, 26:151, 26:194-26:195, 31:24-31:25, 33:61, 33:190, 74:37 <sup>4</sup>	1															
*25:82, 02:135, 26:30, 26:65 <sup>4</sup>	1														15	
*01:01:01:01-01:01:01:27, 01:01:29-01:01:56, 01:01:58-01:01:61, 01:01:63-01:01:106, 01:01:108-01:01:123, 01:01:125-01:04:01:02N, 01:06, 01:08-01:12, 01:14-01:16N, 01:18N-01:33, 01:35-01:50, 01:52:01N-01:70, 01:72-01:99, 01:101-01:104, 01:106-01:143, 01:145-01:175, 01:177-01:189, 01:191-01:193, 01:195-01:203, 01:205-01:249, 01:251-01:288, 01:290N-01:297, 01:299-01:301Q, 01:303:01-01:314, 01:316-01:325, 01:327-01:333, 01:335-01:424, 02:346, 02:427, 02:749, 02:952, 03:41, 11:17, 11:19, 11:40, 11:98, 11:121, 11:308, 11:312, 26:04, 26:29, 26:49, 26:75, 26:78, 26:204:01:01-26:205, 31:164, 33:13, 33:184, 36:01:01:01-36:12, 74:10, 74:30, 80:01:01:01-80:09N			3													
*01:13, 01:17, 01:176, 01:302, 03:63, 03:88, 03:382, 11:01:01:01-11:01:67, 11:01:69-11:01:72, 11:01:74-11:01:75, 11:01:77-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:52Q, 11:54:01-11:95, 11:97, 11:99N-11:105, 11:107-11:115N, 11:117-11:120, 11:122-11:139, 11:141-11:158, 11:160-11:177, 11:179-11:249, 11:251N-11:290, 11:292-11:300, 11:302:01-11:307, 11:309-11:311, 11:313Q-11:324, 11:326-11:398, 11:400N-11:433N, 29:66, 30:125, 30:140, C*07:335				4												
*01:51, 02:644			3			6										
*01:298, 02:829, 32:01:01:01-32:01:13, 32:01:15-32:02, 32:04, 32:06-32:07, 32:09-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:59, 32:61, 32:63-32:70, 32:72-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:151, 32:153-32:171		2														
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>
*02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:78, 02:103, 02:580, 02:609, 02:907, 02:1022, 03:242, 03:312, 23:09, 23:13, 24:07:01:01-24:07:04, 24:19, 24:24, 24:112, 24:129, 24:131, 24:288, 24:290, 24:294Q, 24:339, 24:347:01-24:347:02, 24:387, 24:406, 24:453, 24:457, 24:477, 24:489, 24:510, 24:528, 24:541, 24:544, 24:566, 24:569, 24:574, 29:155, 68:01:01:01-68:02:23, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:56:02, 68:58:01-68:70, 68:72-68:89, 68:91-68:108, 68:110-68:179, 68:181N-68:205N, 68:207-68:226, 68:228-68:248, 68:250-68:254, 68:256-68:276Q, 68:278-68:293Q, 68:295-68:296, 69:01:01:01-69:01:04, 69:03-69:06, 69:08-69:09					5											
*02:55, 02:815, 03:24						6										
*02:309						6	7					12			15	
*02:454						6	7	8				12			15	
*02:741				4	5	6										
*03:01:19, 03:103:02, 31:04:01:01-31:04:02						6	7				11	12			15	
*11:116, 11:140, 26:33	1			4												
*23:12, 24:30, 24:42, 24:309, 24:379													13			
*23:122, 24:44, 24:109, 24:260			w													

Negative Control

*26:01:01:01-26:01:21, 26:01:23-26:01:27, 26:01:29-26:03:01:02, 26:05-26:07:02, 26:10-26:12:01:02, 26:15-26:17, 26:21-26:27, 26:31-26:32, 26:34-26:42, 26:45-26:46, 26:50-26:53, 26:55-26:61, 26:63, 26:66, 26:68-26:71N, 26:74, 26:76-26:77, 26:79-26:80, 26:82, 26:84-26:85, 26:87-26:90, 26:92-26:111, 26:113-26:123, 26:125-26:129, 26:131, 26:133-26:145N, 26:147, 26:149, 26:152-26:175, 26:177-26:188, 26:190-26:193, 26:196-26:203, 26:206:01N-26:218, 26:220-26:224, 43:01-43:03, 66:05, 66:15	1		3																
*26:08:01:01-26:08:03, 26:48, 26:67, 26:81, 26:86, 26:124, 26:150, 26:189, 26:219	1		3			7													
*26:09, 26:91			3			7													
*26:13, 26:19, 66:01:01:01-66:01:07, 66:04, 66:07-66:09, 66:11, 66:13, 66:17, 66:19-66:20, 66:23-66:24, 66:27N, 66:29-66:32, 66:35-66:38, 66:41-66:42, 66:44-66:46	1			4	5														
*26:14, 26:18, 26:28	1		3							11									
*26:20:01-26:20:02	1		3			6													
*26:43:01-26:43:02	1		3			6	7												
*26:47	1		3							9									
*26:54	1		3						8										
*26:62, 26:72	1		3		5														
*26:73, 26:146	1		3				7				11								
*26:112:01-26:112:02	1		3			6	7				11								
*26:130	1		3					8		10									
*26:176	1		3									12							
*31:03			3			6	7				11	12							
*31:123, 74:01:03						6	7				11								15
*32:08		2																13	
*32:15		2				6													
*34:01:01:01-34:01:07, 34:05, 34:12, 34:16, 34:18-34:19, 34:23, 34:27				4	5	7													
*34:02:01:01-34:02:07, 34:04, 34:08, 34:13, 34:15, 34:20-34:22, 34:24-34:26N, 34:29N-34:30N				4	5	6	7					12							
*34:03				4	5	6	7				11	12							
*34:06				4	5	6	7				11								
*34:07				5	6	7						12							
*34:09				5	6	7						12							15
*34:10N				4	5							12							
*34:11, 66:10, 66:40, 68:227, 69:02				4	5														
*34:14				4		7													
*34:17				4	5	7					11								
*34:28				4	5	6	7												
*66:02:01:01-66:02:01:03, 66:16, 66:21, 66:25-66:26Q, 66:34				5															15
*66:03:01:01-66:03:01:03, 66:28N, 66:39N																			15
*66:06, 66:33	1			4	5	6	7												15
*66:12	1				5														15
*66:14	1			4	5			8		10									
*66:18	1			4	5				9										
*66:22	1			4	5						11								
*66:43				5	6	7					11								15
*68:71, 69:07				5	6														
*68:294				5		7	8												15
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>Negative Control</b>		