

HLA-A29 2019 3H6

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
<b>HLA-A allele</b> <sup>1,2,3</sup>																														
*29:01:01:01, 29:01:01:03-29:01:06, 29:01:08-29:01:10, 29:28, 29:57-29:58, 29:60-29:62, 29:67, 29:76, 29:79, 29:83, 29:99, 29:101, 29:107, 29:110, 29:113, 29:115, 29:117-29:118	1													14	15															
*29:01:01:02N	1	2												14	15															
*29:01:07, 29:109														14	15															
*29:01:11	1											12		14	15															
*29:02:01:01-29:02:03, 29:02:05-29:02:26, 29:30-29:31, 29:38, 29:41-29:42, 29:45, 29:47, 29:50, 29:52, 29:59, 29:63, 29:65-29:66, 29:68, 29:70, 29:72, 29:74-29:75, 29:80, 29:84-29:86, 29:88-29:89, 29:91, 29:93-29:97, 29:100, 29:102, 29:106, 29:108, 29:111, 29:114, 29:116, 29:119-29:121			3											14	15															
*29:02:04												12		14	15															
*29:03			3	4										14	15															
*29:04			3		5									14	15															
*29:05, 29:87, 29:104			3			6								14	15															
*29:06			3				7							14	15															
*29:07, 29:46 <sup>4</sup>			3					8						14	15															
*29:08N			3						9					14	15															
*29:09			3							10				14	15															
*29:10:01-29:10:02			3								11			14	15															
*29:11, 29:92 <sup>4</sup>			3										13	14	15															
*29:12	1												13	14	15															
*29:13			3											14								22								
*29:14			3								11			15																
*29:15	1													14	15	16														
*29:16	1								9					14	15															
*29:17	1									10				14	15															
*29:18												12		15																
*29:19			3											14															29	
*29:20, 29:82 <sup>4</sup>	1													14	15														29	
*29:21			3											14	15	16														
*29:22			3											14	15						21									
*29:23			3		5						11			14	15															

ive Control



HLA-A29 2019 3H6

*01:20, 01:66, 01:130, 02:19, 02:44, 02:149, 02:309, 02:408, 02:436, 02:619, 03:95, 11:288, 24:14:01:01-24:14:01:03, 24:93, 24:324, 26:22, 30:47, 66:09, <i>B*08:204, C*02:74</i>											21											
*01:143								15														
*01:148, 11:128, 24:82, 26:85, 68:58:01-68:58:02												22										
*01:157, 03:27, 11:233						9																
*02:24:02, 02:507			4					10	13													23
*02:65, 31:123, 32:01:08, 32:01:18, 32:01:30, 32:56N, 32:70, 32:72, 32:108, 32:113, 74:01:06, 74:14N												20										23
*02:152, 23:83, 31:01:02:01-31:02, 31:09, 31:11-31:15, 31:17-31:23, 31:25-31:50, 31:52-31:61, 31:63-31:66, 31:70-31:76, 31:80-31:98, 31:100-31:107, 31:109-31:119, 31:121-31:122, 31:124-31:135, 31:137-31:146, 33:01:01:01-33:01:04, 33:01:06-33:01:12, 33:03:01:01-33:03:18, 33:03:20-33:03:23, 33:03:25-33:03:26, 33:03:28-33:09, 33:11-33:12, 33:14-33:17, 33:20-33:21, 33:23-33:31, 33:33, 33:35-33:37, 33:39-33:47, 33:49-33:57, 33:60-33:75, 33:77-33:101, 33:103-33:109, 33:111-33:118, 33:120-33:138, 33:140N-33:149, 33:151, 33:153-33:158, 33:160-33:163																						23
*02:221, 23:41													16									
*03:01:18								10														
*03:01:39, 32:01:07, 32:01:12, 32:28, 32:30:02, 32:33:02, 32:66, 32:94, 32:101Q, 68:01:28, 68:01:36									12													
*11:01:28, 11:01:77				6				10													20	
*11:01:42, 30:01:11, 68:130:02			3																			
*11:139, 23:53, 23:70, 24:17, 24:41, 24:208:02						8																
*23:03:01, 31:05, 33:10			4																			23
*24:21:03			4	6				10														23
*24:208:01			4	6	8			10														
*31:07-31:08, 31:10, 33:139																						22 23
*31:16, 33:58										13												23
*31:24				6				10														23
*31:51, <i>B*15:17:03</i>				w																		23
*31:77																						23 24
*31:78												16										23
*31:79, 33:13, 33:48										15												23

Negative Control

HLA-A29 2019 3H6

*31:99, 33:22																				21	23												
*31:108, 33:110																						23	25										
*31:136, 33:59, 33:102, 33:150						6																23											
*32:01:01:01-32:01:06, 32:01:09-32:01:11, 32:01:13-32:01:17, 32:01:19-32:01:27, 32:01:29, 32:01:32, 32:03, 32:05-32:06, 32:08-32:11Q, 32:14-32:21, 32:23-32:27N, 32:29, 32:31, 32:34-32:41, 32:43-32:47, 32:49-32:55:02, 32:57-32:65, 32:67-32:69, 32:71, 32:73-32:93, 32:95-32:100, 32:102-32:107, 32:110-32:112N, 32:114-32:118, 74:01:01-74:01:05, 74:01:07-74:13, 74:15-74:25, 74:27-74:32N											12										20	23											
*32:01:28, 32:30:01, 32:32, 32:48N											12											20											
*32:01:31											12												23										
*32:02, 32:22						6					12											20	23										
*32:07											12					18						20	23										
*32:12, 74:26							w				12											20	23										
*32:13				4							12											20	23										
*32:33:01										10	12											20	23										
*32:42											12											20	22	23									
*32:109				4						10	12	13				17						20	23										
*33:34										10													23										
*33:76																							23								28		
*33:119																18							23										
<i>B*08:01:07, B*15:02:07, B*27:07:05, B*41:01:05, B*55:02:10, C*02:02:15, C*03:03:20, C*04:175</i>							w																										
<i>C*07:04:13, C*08:01:15</i>							w				11																						
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>			