

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
HLA-A allele^{1,2,3}																																
*29:01:01:01, 29:01:01:03-29:01:06, 29:01:08-29:01:10, 29:01:12, 29:01:14-29:01:15, 29:28, 29:57-29:58, 29:60-29:62, 29:67, 29:76, 29:79, 29:83, 29:99, 29:101:01-29:101:02, 29:107, 29:110, 29:113, 29:115, 29:117-29:118, 29:124-29:125, 29:147, 29:155, 29:157, 29:162, 29:166	1													14	15																	
*29:01:01:02N	1	2													14	15																
*29:01:07, 29:01:13, 29:109, 29:136															14	15																
*29:01:11	1											12			14	15																
*29:02:01:01-29:02:03, 29:02:05-29:02:38, 29:30-29:31, 29:38, 29:41-29:42, 29:45, 29:47, 29:50, 29:52, 29:59, 29:63, 29:65-29:66, 29:68, 29:70, 29:72, 29:74-29:75, 29:80, 29:84-29:86, 29:89, 29:91, 29:93-29:97, 29:100, 29:102, 29:106, 29:108, 29:111, 29:114, 29:116, 29:119-29:122, 29:126Q-29:132, 29:134-29:135, 29:137-29:138, 29:140-29:143, 29:145-29:146, 29:148-29:149, 29:152-29:154, 29:156, 29:158, 29:160, 29:163, 29:165, 29:167			3											14	15																	
*29:02:04												12			14	15																
*29:03			3	4											14	15																
*29:04			3		5										14	15																
*29:05, 29:87, 29:104			3			6									14	15																
*29:06			3				7								14	15																
*29:07, 29:46 ⁴			3					8							14	15																
*29:08N, 29:139 ⁴			3						9						14	15																
*29:09			3							10					14	15																
*29:10:01-29:10:02, 29:161			3								11				14	15																
*29:11, 29:92 ⁴			3										13	14	15																	
*29:12	1												13	14	15																	
*29:13			3												14						22											
*29:14			3								11				15																	
*29:15	1													14	15	16																
*29:16	1								9					14	15																	
*29:17, 29:144 ⁴	1									10				14	15																	
*29:18												12			15																	
*29:19			3											14																29		
*29:20, 29:82 ⁴	1													14	15															29		
*29:21			3											14	15	16																
*29:22			3											14	15						21											
*29:23			3		5						11			14	15																	
*29:24	1													14	15		17															
*29:25														14	15					19												
*29:26			3											14	15					19												
*29:27			3											14	15		17															
*29:29			3											14	15							22										
*29:32			3											14	15								23									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
*29:33	1			4		6				10				14	15																	
*29:34			3											14	15															29		
*29:35			3	4							11			14	15																	
*29:36			3											14	15				18													
*29:37			3															18														
*29:39, 29:159			3											14																		
*29:40			3			6								14	15		17															
*29:43			3							10				14	15	16																
*29:44			3											14	15						20											
*29:48	1											12																				
*29:49	1							8						14	15																	
*29:51, 29:73			3	4						10			13	14	15		17															
*29:53			3										14	15	16		17															
*29:54			3							10				14	15		17															
*29:55	1												13	14	15				19													
*29:56	1																	18														
*29:64														14	15					20												
*29:69			3	4										14	15		17															
*29:71	1													14	15										24							
*29:77	1					6								14	15																	
*29:78N			3									12		14	15																	
*29:81	1													14	15								23					27				
*29:88			3											14	15																30	
*29:90			3											14	15																	
*29:98	1													14												25						
*29:103			3											14	15																	
*29:105			3											14	15																	
*29:112N			3											14	15															28		
*29:123	1						w							14	15																	
*29:133:01:01-29:133:01:02			3											14	15						23						27					
*29:150	1													14	15											25						
*29:151			3				w							14	15																	
*29:164	1													15																		
*01:01:114				4						10											20											
*01:01:126, 01:69:03, 31:04:02			3																											29		

Negative Control

Negative Control

