

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele^{1,2,3}																								
*29:01:01:01, 29:01:02-29:01:06, 29:01:08-29:01:09, 29:28, 29:57-29:58, 29:60-29:62, 29:67, 29:71, 29:76, 29:79, 29:81-29:83	1													14	15									
*29:01:01:02N	1	2												14	15									
*29:01:07														14	15									
*29:02:01:01-29:02:03, 29:02:05-29:02:17, 29:30-29:31, 29:38, 29:41-29:42, 29:45, 29:47, 29:50, 29:52, 29:59, 29:63, 29:65-29:66, 29:68, 29:70, 29:72, 29:74-29:75, 29:80			3											14	15									
*29:02:04												12		14	15									
*29:03			3	4										14	15									
*29:04			3		5									14	15									
*29:05			3		6									14	15									
*29:06, 29:34 ⁴			3				7							14	15									
*29:07, 29:46 ⁴			3					8						14	15									
*29:08N			3						9					14	15									
*29:09			3							10				14	15									
*29:10			3								11			14	15									
*29:11			3										13	14	15									
*29:12	1												13	14	15									
*29:13			3											14								22		
*29:14			3								11			15										
*29:15	1													14	15	16								
*29:16	1								9					14	15									
*29:17	1									10				14	15									
*29:18												12		15										
*29:19			3			6	7							14										
*29:20	1						7							14	15									
*29:21			3											14	15	16								
*29:22			3											14	15						21			
*29:23			3		5						11			14	15									
*29:24	1													14	15		17							
*29:25														14	15									
*29:26			3											14	15				19					
*29:27			3											14	15		17							
*29:29			3											14	15							22		
*29:32			3											14	15								23	
*29:33	1			4		6				10				14	15									
*29:35			3	4							11			14	15									
*29:36			3											14	15				18					
*29:37			3																18					
*29:39			3											14										
*29:40			3			6								14	15		17							
*29:43			3							10				14	15	16								
*29:44			3											14	15					20				
*29:48	1											12												
*29:49	1						8							14	15									
*29:51, 29:73			3	4						10			13	14	15		17							
*29:53			3											14	15	16	17							

Negative Control

*29:54			3						10				14	15		17									
*29:55	1												13	14	15			19							
*29:56	1																18								
*29:64													14	15					20						
*29:69			3	4									14	15		17									
*29:77	1					6							14	15											
*29:78N			3								12		14	15											
*01:20, 01:66, 01:130, 02:19, 02:44, 02:149, 02:309, 02:408, 02:436, 03:95, 24:14, 24:93, 24:324, 26:22, 30:47, 66:09, C*02:74																						21			
*01:143														15											
*01:148, 11:128, 24:82, 26:85, 68:58																									22
*01:157, 03:27, 11:233									9																
*02:24:02, 02:507				4						10			13					17							23
*02:65, 32:01:08, 32:01:18, 32:56N, 32:70, 32:72, 74:14N																						20			23
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*02:152, 31:01:02:01-31:02, 31:09, 31:11-31:15, 31:17- 31:23, 31:25-31:50, 31:52-31:61, 31:63-31:66, 31:70-31:77, 31:80-31:98, 31:100-31:105, 33:01:01-33:01:04, 33:01:06- 33:01:08, 33:03:01-33:03:18, 33:03:20-33:03:23, 33:03:25- 33:03:26, 33:03:28-33:09, 33:11-33:12, 33:14-33:17, 33:20- 33:21, 33:23-33:31, 33:33, 33:35-33:37, 33:39-33:47, 33:49- 33:57, 33:60-33:101, 33:103-33:106																									23
*02:221, 23:41																16									
*03:01:18									10																
*03:01:39, 32:01:07, 32:01:12, 32:28, 32:33:02, 32:66, 68:01:28												12													
*11:01:28						6			10												20				
*11:01:42, 30:01:11			3																						
*11:139, 23:53, 24:17, 24:41								8																	
*23:03:01, 31:05, 33:10				4																					23
*24:21:03				4		6			10																23
*24:208				4		6	8		10																
*31:07-31:08, 31:10																							22	23	
*31:16, 33:58													13											23	
*31:24						6			10																23
*31:51							7																		23
*31:78																		16							23
*31:79, 33:13, 33:48															15										23
*31:99, 33:22																						21			23
*32:01:01-32:01:06, 32:01:09-32:01:11, 32:01:13-32:01:17, 32:01:19-32:01:23, 32:03, 32:05-32:06, 32:08-32:11Q, 32:14- 32:21, 32:23-32:27N, 32:29, 32:31, 32:34-32:41, 32:43- 32:47, 32:49-32:55:02, 32:57-32:65, 32:67-32:69, 32:71, 32:73-32:84, 74:01:01-74:13, 74:15-74:25, 74:27												12									20				23
*32:02, 32:22						6				12												20			23
*32:07										12								18				20			23
*32:12, 74:26							7			12												20			23
*32:13				4						12												20			23

Negative Control

*32:30, 32:32, 32:48N											12								20					
*32:33:01									10		12								20			23		
*32:42											12								20		22	23		
*33:34									10													23		
*33:59, 33:102						6																23		
B*15:17:03							7												20			23		
B*55:02:10							7												20					
C*08:01:15											11													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24