



Length of spec.	235	225	205	215	185	275	225	240	230	65	225	190	145	105	225	275	215	135	230	60	130	250	95	470	155	190	95	495	100	130	200	150			
PCR product(s)										205				235				210		175	195	155	210	185			240	145			210	155	230	270	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
HLA-A allele <sup>4</sup>																																			
*03:01:01:01, 03:01:01:03, 03:01:03- 03:01:05, 03:01:07, 03:01:09-03:01:11, 03:01:13, 03:01:15- 03:01:22, 03:01:24- 03:01:29, 03:01:31- 03:01:38, 03:26, 03:77, 03:80, 03:83-03:85, 03:92, 03:94, 03:100-03:101, 03:110, 03:114-03:117, 03:119-03:121, 03:124- 03:126, 03:131, 03:136- 03:139, 03:141, 03:144- 03:148, 03:151, 03:154- 03:159 <sup>19</sup>	1	2	3					8									17																29		
*03:01:01:02N	1	2	3					8					13				17																29		
*03:01:02, 03:01:06, 03:01:14, 03:140	1	2						8									17																29		
*03:01:08, 03:109	1	2	3														17																29		
*03:01:12, 03:01:23	1	w	3					8									17																29		
*03:01:30, 03:98, 03:105		2	3					8									17																29		
*03:02:01-03:02:03, 03:82, 03:90, 03:106, 03:160	1	2		4				8		10																								29	
*03:03N	1	2	3		5			8									17																29		
*03:04:01-03:04:03	1	2	3			6		8									17																29		
*03:05:01		2	3					8	9								17																29		
*03:05:02, 30:55 <sup>5</sup>		2						8	9								17																29		
*03:06	1	2	3				7	8									17																29		
*03:07	1	2						8		10							17																	29	
*03:08	1		3					8			11						17																		
*03:09	1	2	3					8				12					17																	29	
*03:10		2		4				8	9	10																								29	
*03:11N	1	2	3					8						14			17																	29	
*03:12	1		3												15		17																	29	
*03:13	1	2	3					8									17	18																29	
*03:14	1	2	3					8								16	17																	29	
*03:15	1	2	3					8									17		19															29	
*03:16, 03:61 <sup>7</sup>	1	2	3					8									17			20														29	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
Length of spec.	235	225	205	215	185	275	225	240	230	65	225	190	145	105	225	275	215	135	230	60	130	250	95	470	155	190	95	495	100	130	200	150			
PCR product(s)										205				235				210		175	195	155	210	185			240	145			210	155	230	270	









Length of spec.	235	225	205	215	185	275	225	240	230	65	225	190	145	105	225	275	215	135	230	60	130	250	95	470	155	190	95	495	100	130	200	150				
PCR product(s)										205				235				210		175	195	155	240	210	185		240	145		210	155	230	220	150		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
*02:01:09, 02:50, 02:76:02								8				12														25										
*02:05:05, 02:06:07, 02:122												12														25										
*02:11:02, 33:08								8																		25										
*02:19, 02:36-02:37, 02:255, 24:44, 31:05, 33:10, 33:19								8																												
*02:34-02:35:02, 02:56:01-02:56:02, 02:62, 02:103		w						8																									29			
*02:35:03		w						8																		25							29			
*02:48, 02:90, 74:01-74:03, 74:08-74:09, 74:11-74:12N, 74:14N-74:17								8			11																									
*02:54, 23:01:01-23:03:02, 23:05-23:06, 23:08N-23:12, 23:14:01-23:15, 23:17-23:30, 23:32-23:41, 23:44, 23:46-23:54, 24:02:01:01-24:02:04, 24:02:06-24:02:28, 24:02:30-24:02:63, 24:04-24:06, 24:09N, 24:13:01-24:14, 24:17, 24:20-24:21:02, 24:25-24:27, 24:29-24:32, 24:34-24:40N, 24:43, 24:45N-24:50, 24:52-24:64, 24:66-24:74, 24:76-24:88, 24:90N-24:91, 24:93, 24:95-24:103, 24:105-24:107, 24:109-24:111, 24:113-24:117, 24:119-24:124, 24:126-24:130, 24:133-24:137, 24:139-24:166, 24:168-24:203, 24:205-24:206, 24:208-24:209, 24:212, 24:214, 24:216-24:218, 24:220-24:222N																																				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				









*33:49								8	9	10					17																						
*34:02:01, 34:02:03- 34:04, 34:07, 34:10N			3						9					15	17																						
*34:02:02									9					15	17																						
*34:08		2	3					8	9						17						26																
*34:09			3						9					15	17																				29		
*36:01, 36:05									9		11																										
*36:02	1										11					18					23						28								31		
*66:17														15																							
*68:26, 68:65														15																					29		
*68:71		2						8													26						29										
*74:04								8			11										25																
*80:01:01:01- 80:01:01:02											w				17						22																
*80:02								8			11				17						22																
<i>C*07:35, C*08:48</i>																																					
<i>C*07:86</i>																																					
<b>HLA-A allele<sup>4</sup></b>																																					
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>					













285	210	180	160	125	275	235	210	160	120	225	120	95	250	195	165	235	200	240	275	185	220	190	260	150	240	195	165	270	150	190	110	240	145	110	70	545	195	130	285	215	85	260	225	240	175	215	275	180	215	240	390	170	190	200	175	225	Length of spec.	PCR product(s)
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	Well No.																										
																										*02:01:01:01-02:01:08, 02:01:10-02:01:15, 02:01:17- 02:01:19, 02:01:21-02:01:48, 02:01:50-02:01:81, 02:01:83- 02:04, 02:07:01-02:07:02, 02:09, 02:11:01, 02:11:03- 02:13, 02:15N-02:18, 02:20:01-02:20:02, 02:22:01- 02:22:02, 02:24:01-02:27, 02:29-02:33, 02:38-02:40:02, 02:42-02:43N, 02:45-02:47, 02:49, 02:52-02:53N, 02:55, 02:58-02:60:02, 02:63-02:71, 02:73-02:76:01, 02:77, 02:80, 02:82N-02:83N, 02:85-02:86, 02:88N-02:89, 02:92- 02:97:02, 02:101:01-02:102, 02:104-02:105, 02:107, 02:109-02:111, 02:113N- 02:121, 02:123, 02:125N, 02:128, 02:130-02:135, 02:138-02:141, 02:145, 02:147-02:153, 02:155, 02:157-02:163, 02:165- 02:168, 02:171:01-02:171:02, 02:173-02:177, 02:181- 02:183, 02:185-02:228, 02:230-02:231,  02:233-02:236, 02:238- 02:243, 02:245-02:246, 02:249-02:254, 02:256- 02:258, 02:260-02:270, 02:272-02:277, 02:279- 02:285, 02:287-02:289, 02:291-02:294, 02:296- 02:299, 02:301N-02:309, 02:311-02:323, 02:325- 02:327, 02:329, 02:332, 02:334-02:336, 02:339- 02:343, 02:345-02:354, 02:356N-02:357, 02:360- 02:372, 02:374-02:375, 02:377-02:379, 02:381, 02:383-02:386, 02:388- 02:392, 31:01:02-31:01:05, 31:01:07-31:02, 31:06, 31:09, 31:11-31:18, 31:20, 31:22- 31:32, 31:34-31:68, 31:70, 33:01:01-33:01:07, 33:03:01- 33:03:05, 33:03:07-33:07, 33:09, 33:11-33:12, 33:14- 33:16, 33:18:01-33:18:02, 33:20-33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52- 33:64, 74:05																																
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	Well No.																										







																																																															64	*24:213 *25:03, 32:15 *26:03:01-26:03:02, 26:06, 26:21, 26:30, 26:78, 68:05, 68:15, 68:20 *26:07:01-26:07:02 *29:01:01:01-29:02:13, 29:04-29:18, 29:21- 29:29, 29:31-29:33, 29:35-29:38, 29:40- 29:45, 32:17, 68:36, 68:46, 68:57, 68:84 *29:19, 33:24, 74:06 *29:39, 68:29 *30:01:01-30:01:07, 30:11:01-30:11:02, 30:14L-30:16, 30:18- 30:20, 30:23-30:24, 30:26, 30:35-30:43, 30:48-30:49, 30:53- 30:54, 30:56, 30:58- 30:60, 30:62, 30:65 *30:02:01-30:03, 30:07, 30:10, 30:12, 30:22, 30:27N, 30:32-30:33, 30:50-30:51, 30:57, 30:64, 30:66-30:70N *30:04:01-30:04:02, 30:06, 30:29 *30:08 *30:09																																					
																																																															51																																						
																																																																63																																					
																																																															51	*30:13, 30:31, 30:44 *30:17 *30:25 *30:30																																					
																																																															46	*30:34 *30:45 *30:46 *30:52, 30:63, 68:45																																					
																																																															51																																						
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	<b>Well No.</b>																																																																					
180	125	235	120	95	165	200	240	185	190	150	165	150	110	240	70	195	130	215	85	225	240	175	215	180	215	240	390	170	190	175	225	<b>Length of spec.</b>																																																																					
285	160	275	210	225	250	235	240	275	220	260	240	270	190	240	145	545	285	260	275	260	240	175	215	275	215	240	390	170	190	200	225	<b>PCR product(s)</b>																																																																					
																																	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	<b>Well No.</b>																																				
																																	35																																																																			63	*30:61 *31:03-31:04 *31:21, 74:07 *32:03, 32:08 *32:04 *32:05 *32:37 *32:52

