

HLA-A30 2018 1H7 R01

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
<b>HLA-A allele</b> <sup>1,2,3</sup>																																		
*30:01:01:01-30:01:11, 30:38-30:39, 30:43, 30:48-30:49, 30:53-30:54, 30:58, 30:62-30:63, 30:65, 30:72, 30:75, 30:82-30:83, 30:86-30:87, 30:91-30:92, 30:94, 30:96-30:98, 30:102, 30:104, 30:109-30:116, 30:118, 30:120, 30:126, 30:128-30:129	1	2							9	11		13					17																	
*30:02:01:01-30:02:04, 30:02:06-30:02:11, 30:02:13, 30:02:15-30:02:21, 30:50, 30:64, 30:66-30:69, 30:84, 30:107-30:108, 30:119, 30:124, 30:127	1		3						9	11		13																						
*30:02:05			3						9	11		13																						
*30:02:12, 30:100	1		3						9	11		13									21													
*30:02:14	1		3				7		9	11		13																						
*30:03	1		3	4					9	11		13																						
*30:04:01-30:04:02, 30:99, 30:105, 30:117					5				9	11		13																						
*30:06					5	6			9	11		13																						
*30:07	1		3			6			9			13																						
*30:08	1	2					7					13																						
*30:09	1								9	11	12	13																						
*30:10	1		3					8	9	11		13																						
*30:11:01-30:11:02	1	2		4							11						17																	
*30:12:01-30:12:02	1		3						9	10	11																							
*30:13	1		3						9	11		13	14																					
*30:14L, 30:122	1	2							9	11		13		15			17																	
*30:15	1	2							9	11		13			16		17																	
*30:16	1	2							9	11		13	14				17																	
*30:17					5				9	11		13					17																	
*30:18	1	2							9	10	11						17																	
*30:19	1	2			5				9	11		13					17																	
*30:20	1	2							9	11		13					17	18																
*30:22	1		3																					24										
*30:23	1	2							9	11		13					17			20														
*30:24	1	2							9	11		13					17				21													
*30:25	1		3						9	11		13					17							24										
*30:26		2							9	11		13					17						23											
*30:27N	1		3						9	11		13										22												
*30:28	1					7			9	11		13																						
*30:29					5				9	11		13		15																				
*30:30		2							9	11		13					17				21													
*30:31	1	2							9			13					17	19																
*30:32	1		3						9	11		13					18																	
*30:33	1		3						9	11		13			16																			
*30:34	1		3						9	11		13									20													

Control



