

HLA-A30 2019 2K3

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2,3}																																	
*30:01:01:01-30:01:11, 30:38-30:39, 30:43, 30:48-30:49, 30:53-30:54, 30:58, 30:62-30:63, 30:65, 30:72, 30:75, 30:82-30:83, 30:86-30:87, 30:91-30:92, 30:94, 30:96-30:98, 30:102, 30:104, 30:109-30:116, 30:118, 30:120, 30:126, 30:128-30:129, 30:134-30:138, 30:140-30:143, 30:145N, 30:147-30:148	1	2							9		11		13				17																Negative Control
*30:01:12, 30:93		2							9		11		13				17																
*30:02:01:01-30:02:04, 30:02:06-30:02:11, 30:02:13-30:02:24, 30:50, 30:64, 30:66-30:69, 30:84, 30:107-30:108, 30:119, 30:124, 30:127, 30:144, 30:146, 30:149, 30:151, 30:153	1		3						9		11		13																				
*30:02:05			3						9		11		13																				
*30:02:12, 30:100	1		3						9		11		13								21												
*30:03	1		3	4							11																						
*30:04:01-30:04:02, 30:99, 30:105, 30:117, 30:139					5				9		11		13																				
*30:06					5	6			9		11		13																				
*30:07	1		3			6			9				13																				
*30:08:01-30:08:02	1	2					7						13																				
*30:09	1								9		11	12	13																				
*30:10	1		3					8	9		11		13																				
*30:11:01-30:11:02	1	2		4							11						17																
*30:12:01-30:12:02	1		3						9	10	11																						
*30:13	1		3						9		11		13	14																			
*30:14L, 30:122	1	2							9		11		13		15		17																
*30:15	1	2							9		11		13			16	17																
*30:16	1	2							9		11		13	14			17																
*30:17					5				9		11		13				17																
*30:18	1	2							9	10	11						17																
*30:19	1	2			5				9		11		13				17																
*30:20	1	2							9		11		13				17	18															
*30:22	1		3																					24									
*30:23	1	2							9		11		13				17			20													
*30:24	1	2							9		11		13				17				21												
*30:25	1		3						9		11		13				17							24									

Negative Control

HLA-A30 2019 2K3

*30:26	2								9 11				13				17				23												
*30:27N	1	2	3						9	10	11	12	13					17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*30:28	1						7		9		11		13																				
*30:29					5				9		11		13		15																		
*30:30		2							9		11		13				17				21												
*30:31	1	2							9				13				17		19														
*30:32	1		3						9		11		13				18																
*30:33	1		3						9		11		13			16																	
*30:34	1		3						9		11		13							20													
*30:35	1	2							9		11	12	13				17																
*30:36	1	2							9		11		13				17					22											
*30:37	1	2							9		11		13				17													30			
*30:40	1	2							9		11		13				17								25								
*30:41	1	2							9		11						17																
*30:42, 30:81 ⁴	1	2							9		11		13				17									26							
*30:44	1	2							9		11		13	14																			
*30:45	1		3						9		11		13														27						
*30:46					5				9		11		13	14																			
*30:47	1								9		11		13																				
*30:51	1		3			6			9		11		13	14																			
*30:52, 30:89	1								9		11		13				17																
*30:55	1	2								10																							
*30:56, 30:132N ⁴	1	2							9		11		13				17										27						
*30:57	1		3						9		11		13										24										
*30:59N, 30:74	1	2							9		11		13				17													29			
*30:60	1	2							9				13				17																
*30:61	1		3						9		11		13																	29			
*30:70N	1		3						9		11		13					19															
*30:71	1	2		4							11							18															
*30:73N	1	2							9		11		13				17					22								29			
*30:76N	1		3						9		11		13										23		25								
*30:77					5				9		11		13					19															
*30:78N	1	2							9		11		13				17						24										
*30:79	1	2											13																				
*30:80	1								9		11		13																		31		
*30:85	1		3								11																						
*30:88	1		3					8	9		11		13				17						24										
*30:90					5				9		11		13										24										
*30:95	1	2							9		11		13		16		17								25								
*30:101Q	1		3						9		11		13																		31		

Negative Control

HLA-A30 2019 2K3

*30:103					5					9	11																					
*30:106	1	2			6					9	11	13	14			17																
*30:121N	1		3							9	11	13									26				29							
*30:123N		2								9	11	13					17	23								30						
*30:125	1		3							9	11	13					17															
*30:130N	1	2								9	11	13					17									28						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*30:131	1				7					9	11		13					17														
*30:133	1		3										13																			
*30:150					5		7						13																			
*30:152	1		3							10																						
*01:02, 01:20				4							w											24										
*01:04:01:01N-01:04:01:02N, 03:21N, 11:21N, 11:166, 23:07N, 24:11N																										27						
*01:07, 02:185, 02:601, 29:14, 31:01:02:01-31:01:06, 31:01:08-31:01:33, 31:01:35-31:02:02, 31:05-31:07, 31:09-31:43, 31:45-31:50, 31:52-31:86, 31:88, 31:90-31:141N, 31:143-31:153										10																						
*01:57N, 01:88:02-01:88:03, 01:236, 02:156, 02:338, 03:17:01, 03:171, 11:119:01-11:119:02, 11:209, 24:92, 68:103:01-68:103:02																										29						
*01:190				4																		24										
*01:244, 01:257, 03:42, 03:131, 03:133, 11:165, 33:49, 33:144, 74:23												12																				
*01:304, 03:05:02, 03:231:02, 11:24:01, 11:25:02, 34:02:02, 80:01:01:01-80:05				3																												
*02:52	1				5																											
*02:121, 02:425, 02:517, 03:154:01, 23:47, 24:234, 29:06, 33:168, 68:14, 68:158					6								14																			
*03:04:02-03:04:03, 11:153:01																		20														
*03:43, 03:186, 23:46	1																															
*03:45																		19														
*03:72, 11:88, 24:24, 24:290, 29:37, 29:56, 68:45, 68:117										11							17															
*03:82, 24:66, 24:308, 24:333, 68:06					5																											

Negative Control

HLA-A30 2019 2K3

[illegible]