

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
HLA-A allele ^{1,2,3}																																				
*30:01:01:01-30:01:11, 30:01:13-30:01:17, 30:38-30:39:02, 30:43, 30:48-30:49, 30:53-30:54, 30:58, 30:62-30:63, 30:65, 30:72, 30:75, 30:82-30:83, 30:86-30:87, 30:91-30:92, 30:94, 30:96-30:98, 30:102, 30:104, 30:109-30:116, 30:118, 30:120, 30:126, 30:128-30:129, 30:134-30:138, 30:140-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:165, 30:167-30:168	1	2							9	11	13						17																			
*30:01:12, 30:93, 30:164		2							9	11	13						17																			
*30:02:01:01-30:02:04, 30:02:06- 30:02:24, 30:50, 30:64, 30:66-30:69, 30:84, 30:107-30:108, 30:119, 30:124, 30:127, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156, 30:158N, 30:162, 30:169	1		3						9	11	13																									
*30:02:05			3						9	11	13																									
*30:03	1		3	4						11																										
*30:04:01:01-30:04:03, 30:99, 30:105, 30:117, 30:139, 30:155					5				9	11	13																									
*30:06					5	6			9	11	13																									
*30:07	1		3			6			9																											
*30:08:01-30:08:02	1	2					7																													
*30:09:01:01-30:09:01:02	1								9	11	12	13																								
*30:10	1		3					8	9	11	13																									
*30:11:01-30:11:02, 30:163	1	2	4						9	11																										
*30:12:01-30:12:02	1		3						9	10	11																									
*30:13	1		3						9	11			13	14																						
*30:14L, 30:122	1	2							9	11			13		15		17																			
*30:15	1	2							9	11			13			16	17																			
*30:16	1	2							9	11			13	14			17																			
*30:17					5				9	11			13				17																			
*30:18	1	2							9	10	11						17																			
*30:19	1	2			5				9	11			13				17																			
*30:20	1	2							9	11			13				17	18																		
*30:22	1		3																						24											
*30:23	1	2							9	11			13				17			20																
*30:24	1	2							9	11			13				17				21															
*30:25	1		3						9	11			13				17									24										
*30:26		2							9	11			13				17										23									
*30:27N	1		3						9	11			13														22									
*30:28	1						7		9	11			13																							
*30:29					5				9	11			13		15																					
*30:30		2							9	11			13				17				21															
*30:31	1	2							9				13				17		19																	
*30:32	1		3						9	11			13				17		18																	
*30:33	1		3						9	11			13			16																				
*30:34	1		3						9	11			13								20															
*30:35	1	2							9	11	12	13					17																			
*30:36	1	2							9	11			13				17					22														
*30:37	1	2							9	11			13				17																			
*30:40	1	2							9	11			13				17									25										
*30:41	1	2							9	11			13				17																			
*30:42, 30:81⁴	1	2							9	11			13				17										26									
*30:44	1	2							9	11			13	14																						
*30:45	1		3						9	11			13															27								
*30:46					5				9	11			13	14																						
*30:47, 30:160	1								9	11			13																							
*30:51	1		3			6			9	11			13	14																						
*30:52, 30:89	1								9	11			13				17																			
*30:55	1	2								10																										
*30:56, 30:132N⁴	1	2							9	11			13				17											27								
*30:57	1		3						9	11			13											24												
*30:59N, 30:74	1	2							9	11			13				17																			
*30:60	1	2							9				13				17																			
*30:61	1		3						9	11			13																							
Well No.	1	2	3	4																																

