

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-A allele ^{1,2,3}																																		
*30:01:01:01-30:01:11, 30:01:13-30:01:20, 30:39:01-30:39:02, 30:43, 30:48-30:49, 30:53, 30:62-30:63, 30:65, 30:72, 30:75, 30:82-30:83, 30:86-30:87, 30:91-30:92, 30:94, 30:96-30:98, 30:102, 30:104, 30:109-30:116, 30:118, 30:120, 30:126, 30:128-30:129, 30:134-30:138, 30:140-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:165, 30:167-30:168, 30:170-30:171, 30:173, 30:176, 30:178N-30:181, 30:184Q-30:185, 30:187, 30:190-30:192, 30:194-30:197N, 30:199	1	2							9	11		13					17																	
*30:01:12, 30:93, 30:164		2							9	11		13					17																	
*30:02:01:01-30:02:04, 30:02:06- 30:02:29, 30:50, 30:64, 30:66-30:69, 30:84, 30:107-30:108, 30:119, 30:124, 30:127, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156, 30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198, 30:200	1		3						9	11		13																						
*30:02:05			3						9	11		13																						
*30:03	1		3	4						11																								
*30:04:01:01-30:04:03, 30:99, 30:105, 30:117, 30:139, 30:155					5				9	11		13																						
*30:06					5	6			9	11		13																						
*30:07	1		3			6			9			13																						
*30:08:01-30:08:02	1	2					7					13																						
*30:09:01:01-30:09:01:02	1								9	11	12	13																						
*30:10	1		3					8	9	11		13																						
*30:11:01-30:11:02, 30:163	1	2	4							11							17																	
*30:12:01-30:12:02	1		3						9	10	11																							
*30:13	1		3						9	11		13	14																					
*30:14L, 30:122	1	2							9	11		13		15			17																	
*30:15	1	2							9	11		13				16	17																	
*30:16	1	2							9	11		13	14				17																	
*30:17				5					9	11		13					17																	
*30:18	1	2							9	10	11						17																	
*30:19	1	2		5					9	11		13					17																	
*30:20	1	2							9	11		13					17	18																
*30:22	1		3																					24										
*30:23	1	2							9	11		13					17		20															
*30:24	1	2							9	11		13					17			21														
*30:25	1		3						9	11		13					17							24										
*30:26		2							9	11		13					17						23											
*30:27N	1		3						9	11		13										22												
*30:28	1						7		9	11		13																						
*30:29				5					9	11		13		15																				
*30:30		2							9	11		13					17				21													
*30:31	1	2							9	11		13					17		19															
*30:32	1		3						9	11		13						18																
*30:33	1		3						9	11		13				16																		
*30:34	1		3						9	11		13									20													
*30:35	1	2							9	11	12	13					17																	
*30:36	1	2							9	11		13					17					22												
*30:37	1	2							9	11		13					17																	
*30:38	1	2							9	11		13					17	18	19													30		
*30:40	1	2							9	11		13					17									25								
*30:41	1	2							9	11		13					17																	
*30:42, 30:81⁴	1	2							9	11		13					17										26							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*30:44	1	2							9	11		13	14																					
*30:45	1		3						9	11		13																27						
*30:46					5				9	11		13	14																					
*30:47, 30:160	1								9	11		13																						
*30:51	1		3			6			9	11		13	14																					
*30:52, 30:172	1								9	11		13					17																	
*30:54, 30:130N ⁴	1	2							9	11		13					17													28				
*30:55	1	2								10																								
*30:56, 30:132N⁴	1	2							9	11		13					17																	
*30:57	1		3						9	11		13												24										
*30:58	1	2							9	11		13		15																				

