

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
HLA-A allele^{1,2,3}																														
*32:01:01-32:01:17, 32:01:19-32:01:23, 32:12, 32:29-32:38, 32:40-32:41, 32:43, 32:46, 32:49-32:51, 32:55:01-32:55:02, 32:57-32:59, 32:62-32:63, 32:65, 32:67, 32:70-32:73, 32:75-32:77, 32:80-32:83, 32:85-32:86	1	2							9																					
*32:01:18	1	2																												
*32:02	1	2	3	4					9																					
*32:03	1				5				9																					
*32:04	1	2				6																								
*32:05	1	2					7		9																					
*32:06	1	2		4					9																					
*32:07		2						8	9																					
*32:08	1	2							9	10																				
*32:09	1	2							9		11																			
*32:10, 32:16 ⁴	1	2							9			12																		
*32:11Q	1	2							9				13																	
*32:13	1	2							9						15															
*32:14	1	2							9							16														
*32:15		2							9				13																	
*32:17	1	2					7		9	10																				
*32:18	1	2							9					14	15															
*32:19N	1	2						8	9																					
*32:20	1	2				6			9		11																			
*32:21, 32:44 ⁴	1	2							9								17													
*32:22	1	2	3						9																					
*32:23, 32:54 ⁴	1	2							9											20										
*32:24	1	2							9								18													
*32:25		2							9											19										
*32:26	1	2							9												21									
*32:27N	1	2							9													22								
*32:28, 32:53, 32:66 ⁴	1	2							9														23							
*32:39, 32:42, 32:64		2							9																					
*32:45N	1	2							9										19											
*32:47	1	2							9																		27			
*32:48N	1	2							9															24						
*32:52	1	2				6			9																					
*32:56N	1	2							9													22		24						
*32:60	1	2							9										18	19										
*32:61	1	2							9																25					
*32:68	1	2							9																	26				
*32:69	1	2							9																		28			
*32:74	1	2							9																			29		
*32:78, 32:84	1								9																					
*32:79		2					7		9																					
*01:01:01-01:01:75, 01:03-01:04N, 01:06-01:19, 01:21-01:33, 01:35-01:42, 01:44-01:46, 01:48-01:50, 01:52-01N-01:59, 01:61-01:70, 01:72-01:82, 01:84-01:104, 01:106-01:142, 01:144-01:187, 01:189, 01:191-01:208Q, 02:45, 02:56:01-02:56:02, 02:103, 02:195, 03:01:01-01:03:01:03, 03:01:05-03:11N, 03:13-03:22:02, 03:25-03:29, 03:31-03:35, 03:37-03:58, 03:60-03:69N, 03:71, 03:73-03:87, 03:90-03:112, 03:114-03:151, 03:153-03:157:01, 03:158-03:193, 03:195-03:196, 03:199-03:204, 03:206-03:218, 03:220-03:249, 11:199:01-11:199:02, 11:222, 30:55, 31:21, 36:01-36:05, 74:01:01-74:03, 74:05-74:18, 74:22-74:25, 74:27	1																													
*01:02, 01:20, 23:09								w																						
*01:51, 02:55, 03:24, 25:03, 26:20, 34:08, 68:71													13																	
*02:24:02, 29:51, 29:80, 31:41				4																										
*02:41, 02:80, 02:117, 02:289:01, 02:304, 02:454, 02:480, 23:45, 24:62, 26:10, 31:67-31:68, 33:32:01, 33:39																							23							
*02:81, 02:124, 23:36, 25:19:01-25:19:02		2																												
*02:294, 33:46, 66:08																					20									
*02:314N																														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
*02:507, 33:94				4								12																		
*03:72, 11:88, 23:51, 24:24, 24:67, 24:145, 24:156, 24:191, 24:290, 26:16, 29:37, 29:56, 30:01:01-30:04:02, 30:06, 30:09-30:20, 30:23-30:30, 30:32-30:54, 30:56-30:59N, 30:61-30:78N, 30:80-30:99, 30:102, 68:45, 68:117								8																						
*03:152, 24:18, 24:204, 24:213							6																							
*03:197N	1																							24						
*03:219	1					6																								
*11:01:28, 29:05, 29:77, 29:87, 31:24, 33:59, 33:102			3																											
*23:03:01, 29:03, 31:05, 33:10														15																

Negative Control

Control

