

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
HLA-A allele ^{1,2,3}																																			
*32:01:01:01-32:01:06, 32:01:08-32:01:11, 32:01:13-32:01:17, 32:01:19-32:01:37, 32:26:02, 32:29-32:30:01, 32:31-32:33:01, 32:34-32:38, 32:40-32:41, 32:43, 32:46, 32:49-32:51, 32:55:01-32:55:02, 32:57-32:59, 32:62-32:63, 32:65, 32:67, 32:70-32:73, 32:75-32:77, 32:80-32:83, 32:85-32:91, 32:95-32:100, 32:102-32:108, 32:110-32:111, 32:113-32:116, 32:119-32:131	1	2							9																										
*32:01:07, 32:01:12, 32:30:02, 32:33:02, 32:78, 32:84, 32:94	1								9																										
*32:01:18	1	2																																	
*32:02	1	2	3	4					9																										
*32:03	1				5				9																										
*32:04	1	2				6																													
*32:05	1	2					7		9																										
*32:06, 32:109	1	2		4					9																										
*32:07		2						8	9																										
*32:08	1	2							9	10																									
*32:09	1	2							9		11																								
*32:10, 32:16, 32:118 ⁴	1	2							9			12																							
*32:11Q	1	2							9				13																						
*32:12	1	2							9					14																					
*32:13	1	2							9						15																				
*32:14	1	2							9							16																			
*32:15		2							9				13																						
*32:17	1	2					7		9	10																									
*32:18	1	2							9					14	15																				
*32:19N	1	2						8	9																										
*32:20	1	2				6			9	11																									
*32:21, 32:44 ⁴	1	2							9								17																		
*32:22	1	2	3						9																										
*32:23, 32:54 ⁴	1	2							9												20														
*32:24	1	2							9									18																	
*32:25		2							9										19																
*32:26:01	1	2							9													21													
*32:27N	1	2							9														22												
*32:28, 32:66	1								9															23											
*32:39, 32:42, 32:64, 32:93		2							9																										
*32:45N	1	2							9										19																
*32:47	1	2							9																						27				
*32:48N	1	2							9															24											
*32:52	1	2				6			9																										
*32:53	1	2							9																23										
*32:56N	1	2							9														22	24											
*32:60	1	2							9									18	19																
*32:61	1	2							9																	25									
*32:68	1	2							9																		26								
*32:69, 32:112N ⁴	1	2							9																						28				
*32:74	1	2							9																							29			
*32:79		2					7		9																										
*32:92N	1	2							9																									31	
*32:101Q	1								9																									30	
*32:117N	1	2							9																									30	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*01:01:01:01-01:01:106, 01:03:01:01-01:04:01:02N, 01:06-01:19, 01:21-01:33, 01:35-01:42, 01:44-01:46, 01:48-01:50, 01:52-01N-01:59, 01:61-01:70, 01:72-01:82, 01:84-01:104, 01:106-01:142, 01:144-01:187, 01:189, 01:191-01:219, 01:221-01:249, 01:251-01:301, 01:303-01:306, 01:308N-01:319, 02:45, 02:56:01-02:56:02, 02:103, 02:195, 03:01:01:01-03:01:03, 03:01:05-03:11N, 03:13-03:22:02, 03:25-03:29, 03:31-03:35, 03:37-03:58, 03:60-03:69N, 03:71, 03:73-03:87, 03:90, 03:92-03:112, 03:114-03:151, 03:153-03:157:01, 03:158-03:193, 03:195-03:196, 03:199, 03:201-03:204, 03:206-03:218, 03:220-03:259, 03:261-03:272, 03:274-03:277, 03:279N-03:302, 03:304-03:369, 11:199-01-11:199:02, 11:222, 30:55, 31:21, 36:01-36:09, 74:01:01:01-74:03, 74:05-74:18, 74:22-74:25, 74:27-74:28, 74:30-74:34, 74:36	1																																		
*01:02, 01:20, 23:09								w																											
*01:51, 02:55, 02:644, 02:741, 02:815, 03:24, 25:03, 25:30, 26:20, 34:08, 68:71													13																						
*02:19, 02:294, 33:46, 66:08, 68:206																																			
*02:24:02, 29:51, 29:80, 31:41				4																															

Negative Control

