

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2,3}																																	
*32:01:01:01-32:01:06, 32:01:08 -32:01:11, 32:01:13- 32:01:17 , 32:01:19-32:01:42, 32:29-32:30:01, 32:31- 32:33:01, 32:34-32:38, 32:40-32:41, 32:43:01- 32:43:02, 32:46:01-32:46:02, 32:49-32:51, 32:55:01- 32:55:03, 32:57-32:59, 32:62-32:63, 32:65, 32:67, 32:70-32:73, 32:75-32:77, 32:80-32:83, 32:85-32:91, 32:95-32:100, 32:102-32:108, 32:110-32:111, 32:113- 32:116, 32:119-32:141, 32:143-32:146	1	2							9																								
*32:01:07, 32:01:12 , 32:30:02, 32:33:02, 32:78, 32:84, 32:94	1								9																								
*32:01:18	1	2																															
*32:02	1	2	3	4					9																								
*32:03:01:01-32:03:01:02	1				5				9																								
*32:04	1	2				6																											
*32:05	1	2					7		9																								
*32:06, 32:109	1	2		4					9																								
*32:07		2						8	9																								
*32:08	1	2							9	10																							
*32:09	1	2							9		11																						
*32:10, 32:16 , 32:118 ⁴	1	2							9			12																					
*32:11Q	1	2							9				13																				
*32:12	1	2							9					14																			
*32:13	1	2							9						15																		
*32:14	1	2							9							16																	
*32:15		2							9				13																				
*32:17	1	2					7		9	10																							
*32:18	1	2							9					14	15																		
*32:19N	1	2						8	9																								
*32:20	1	2				6			9		11																						
*32:21, 32:44 ⁴	1	2							9								17																
*32:22	1	2	3						9																								
*32:23, 32:54 , 32:142 ⁴	1	2							9											20													
*32:24	1	2							9												18												
*32:25		2							9													19											
*32:26:01-32:26:02	1	2							9														21										
*32:27N	1	2							9														22										
*32:28, 32:66	1								9															23									
*32:39, 32:42, 32:64, 32:93		2							9																								
*32:45N	1	2							9													19											
*32:47	1	2							9																					27			
*32:48N	1	2							9																24								
*32:52	1	2				6			9																								
*32:53	1	2							9															23									
*32:56N	1	2							9														22	24									
*32:60	1	2							9													18	19										
*32:61	1	2							9																	25							
*32:68	1	2							9																		26						
*32:69, 32:112N ⁴	1	2							9																					28			
*32:74	1	2							9																						29		
*32:79		2					7		9																								
*32:92N	1	2							9																								31
*32:101Q	1								9																								30
*32:117N	1	2							9																								30
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*01:01:01:01-01:01:113, 01:01:115-01:01:120, 01:03:01:01-01:04:01:02N, 01:06-01:19, 01:21- 01:33, 01:35-01:42, 01:44-01:46, 01:48-01:50, 01:52:01N-01:59, 01:61-01:70, 01:72-01:82, 01:84- 01:104, 01:106-01:142, 01:144-01:187, 01:189, 01:191-01:219, 01:221-01:249, 01:251-01:301Q, 01:303:01-01:306, 01:308N-01:341, 01:343-01:353, 02:45, 02:56:01-02:56:02, 02:103, 02:195, 03:01:01:01-03:01:03, 03:01:05-03:11N, 03:13- 03:22:02, 03:25-03:29, 03:31-03:35, 03:37-03:58, 03:60-03:69N, 03:71, 03:73-03:87, 03:90, 03:92- 03:112, 03:114-03:151, 03:153-03:157:01, 03:158- 03:193, 03:195-03:196, 03:197:02N, 03:199, 03:201- 03:204, 03:206-03:218, 03:220-03:259, 03:261- 03:272, 03:274-03:277, 03:279N-03:302, 03:304- 03:398, 11:199:01-11:199:03, 11:222, 30:55, 30:166, 30:177, 31:21, 36:01:01:01-36:11, 74:01:01:01- 74:03:02, 74:05-74:18, 74:22-74:25, 74:27-74:28, 74:30-74:34, 74:36, 74:38-74:39	1																																

Negative Control

