

HLA-A33 2018 7G0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2,3}																																	
*33:01:01:01 -33:01:11, 33:49-33:50, 33:67, 33:89, 33:91, 33:121-33:122, 33:127, 33:136	1	2	3																														
*33:03:01:01 -33:03:02, 33:03:04- 33:03:34 , 33:35, 33:37, 33:40-33:41, 33:43, 33:45- 33:48, 33:55-33:56, 33:59-33:63, 33:66, 33:71-33:72, 33:75-33:76, 33:78-33:79, 33:81, 33:87-33:88, 33:93-33:95, 33:97- 33:105, 33:107-33:108, 33:110, 33:112- 33:118, 33:120, 33:124, 33:126, 33:128, 33:130, 33:132 -33:135, 33:137-33:139, 33:141- 33:142 , 33:144-33:146	1			4																													
*33:03:03Q , 33:86 ⁵	1			4																												30	
*33:04	1	2			5																												
*33:05	1	2	3			6																											
*33:06	1			4			7																										
*33:07	1	2	3					8																									
*33:08 , 31:99 ⁴				4					9	10																							
*33:09 , 02:309, 26:22, 66:09 ⁴				4					9																								
*33:10	1			4							11																						
*33:11 , 33:80N, 33:131 ⁵	1			4								12																					
*33:12	1			4									13																				
*33:13	1			4						10																							
*33:14	1			4					9																								
*33:15	1			4										14																			
*33:16 , 33:64 ⁵	1	2	3												15																		
*33:17 , 33:140N ⁵	1			4												16																	
*33:18:01 -33:18:02, 29:105 ⁴	1																17																
*33:19	1													14																			
*33:20	1			4		6																											
*33:21				4			7									16																	
*33:22	1			4					9	10																							

Control

HLA-A33 2018 7G0

<p>*01:145, 02:41, 02:65, 02:80, 02:117, 02:135, 02:152, 02:289:01, 02:304, 02:454, 03:103:01-03:103:02, 03:282, 11:116, 11:140, 23:45, 24:62, 25:01:01:01-25:42N, 25:44-25:46, 26:01:01:01-26:21, 26:23, 26:25N-26:39, 26:41-26:43:02, 26:45-26:59, 26:61-26:67, 26:69-26:98, 26:100-26:106, 26:108-26:149, 29:32, 31:01:02:01-31:01:28, 31:09-31:23, 31:25-31:28, 31:30-31:47, 31:49-31:53, 31:55-31:65, 31:67-31:68, 31:70-31:87, 31:90, 31:92-31:93, 31:95-31:98, 31:100-31:108, 31:111-31:114, 31:116-31:131N, 31:133-31:135, 32:01:01:01-32:01:27, 32:01:29-32:03, 32:05-32:09, 32:11Q-32:12, 32:14-32:29, 32:31, 32:33:02-32:40, 32:42-32:47, 32:49-32:50, 32:52, 32:54-32:62, 32:64-32:97, 32:99-32:100, 32:102-32:110, 34:01:01:01-34:01:02, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:17, 43:01, 66:01:01:01-66:03:01:02, 66:05-66:08, 66:10-66:14, 66:16-66:31, 74:01:01-74:27</p>	<p>4</p>							
<p>*01:166</p>						<p>22</p>	<p>28</p>	
<p>*01:244, 03:42, 03:133, 23:53, 23:70, 29:01:01:01-29:02:22, 29:04, 29:06, 29:08N, 29:10:01-29:11, 29:13-29:18, 29:20-29:21, 29:23-29:31, 29:34, 29:36-29:47, 29:50, 29:52-29:58, 29:60, 29:62-29:65, 29:67-29:70, 29:72-29:76, 29:78N, 29:81, 29:84-29:86, 29:88-29:91, 29:93-29:103, 29:106-29:112N, 31:04, 31:06, 32:30:01-32:30:02, 32:32, 68:168</p>				<p>17</p>				<p>Negative Control</p>

HLA-A33 2018 7G0

*02:10, 02:17:01-02:17:04, 02:39, 02:108, 02:110, 02:140, 02:148, 02:242, 02:244, 02:268, 02:300, 02:303, 02:398, 02:453, 02:604, 02:617, 02:628, 02:630, 02:657, 02:680, 03:15, 03:19, 11:139, 24:04, 24:19, 24:28, 24:44, 24:89, 24:109, 24:129, 24:290													14																					
*02:24:02								9	10												22													
*02:241, 02:684																																		
*02:243:01-02:243:03							8	9	10																									
*02:292																																	29	
*02:332, 24:220			3																							23								
*02:342																					21													
*02:444							8																											
*02:480, 68:176																					20													
*03:01:18																		19							24									
*03:22:02, 29:83, B*15:02:07, B*15:17:03, C*02:02:15, C*03:03:20																	19																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*03:104, 31:88, 66:04		2	3																															
*03:145:02, 11:54																																28		
*03:205, 11:137:01N												12																						
*11:01:28, 11:01:77																									24									
*11:43									10		12																							
*11:98, 11:250, 68:04:01							7																											
*23:03:01												11																						
*24:21:03, 24:208, 29:33												11													24									
*26:24				4																		22												
*26:60N, 31:66, 31:89, 31:115				4								12																						
*26:68				4	5																													
*26:99, 31:29, 31:48				4										14																				
*26:107N, 32:98				4																													31	
*29:03											11								17															
*29:07, 29:49														14				17																
*29:09, 29:59																		17						24										

Control

