

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*33:111																										26								
*33:123N	1			4																		22												
*33:125	1	2	3									12																						
*33:129N	1	2	3																			21			25									
*33:143N	1	2	3																			20												
*33:184, 26:177 ⁵	1			4				8		10																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*01:01:38L, 02:01:14Q, 03:265, 11:01:98, 24:02:03Q, 30:02:25, B*18:173, B*44:287, C*07:756																																30		
*01:01:114, 24:21:03, 24:208:01, 29:33											11														24									
*01:52:01N, 24:114																																31		
*01:57N, 03:381N, 30:59N																16																		
*01:84, 02:214, 03:371																																28	31	
*01:140:02, 02:784, 03:145:02, 11:54:01, B*40:381, B*40:390, C*04:35:02																																	28	
*01:143, 24:82, C*07:449										10																								
*01:145, 02:41, 02:65, 02:80, 02:117, 02:135, 02:152, 02:289:01, 02:304, 02:309, 02:454, 02:829, 02:872, 03:103:01-03:103:02, 03:282, 11:116, 11:140, 11:372, 23:45, 24:62, 25:01:01:01-25:42N, 25:44-25:58, 25:60-25:70, 26:01:01:01-26:23, 26:25N-26:39, 26:41-26:43:02, 26:45-26:59, 26:61-26:67, 26:69-26:98, 26:100-26:106, 26:108-26:176, 26:178-26:189, 26:191N-26:203, 26:205-26:206N, 29:32, 31:01:02:01-31:01:45, 31:09-31:23, 31:25-31:28, 31:30-31:47, 31:49-31:53, 31:55-31:65, 31:67-31:68, 31:70-31:87, 31:90, 31:92-31:93, 31:95-31:98, 31:100-31:108, 31:111-31:114, 31:116-31:131N, 31:133-31:163, 31:165-31:178, 31:180-31:184N, 32:01:01:01-32:01:27, 32:01:29-32:03:01:02, 32:05-32:09, 32:11Q-32:12, 32:14-32:29, 32:31, 32:33:02-32:40, 32:42-32:47, 32:49-32:50, 32:52, 32:54-32:62, 32:64-32:97, 32:99-32:100, 32:102-32:118, 32:120-32:144, 32:146, 34:01:01:01-34:01:05, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:19, 34:23, 43:01-43:02N, 66:01:01:01-66:03:01:03, 66:05-66:14, 66:16-66:38, 66:40-66:43, 74:01:01:01-74:27, 74:29-74:31, 74:33-74:39				4																														
*01:244, 03:42, 03:133, 23:53, 23:70, 29:01:01:01-29:02:22, 29:02:24-29:02:30, 29:04, 29:06, 29:08N, 29:10:01-29:11, 29:13-29:18, 29:20-29:21, 29:23-29:31, 29:34, 29:36-29:47, 29:50, 29:52-29:58, 29:60, 29:62-29:65, 29:67-29:70, 29:72-29:76, 29:78N, 29:81, 29:84-29:86, 29:88-29:91, 29:93-29:103, 29:106-29:115, 29:117-29:129, 29:131, 29:133:01:01-29:137, 29:139-29:143, 29:146, 31:04:01:01-31:04:02, 31:06, 31:179, 32:30:01-32:30:02, 32:32, 68:168																																		
*01:268, 11:137:01N														14													26							
*01:301Q, 02:10, 02:17:02:01-02:17:04, 02:39, 02:108, 02:110, 02:140, 02:148, 02:242, 02:244, 02:268, 02:300, 02:303, 02:398, 02:453, 02:604, 02:617, 02:628, 02:630, 02:657, 02:680, 02:789N, 02:804, 02:821, 03:15, 03:19, 03:347, 11:139, 24:04, 24:19, 24:28, 24:44, 24:89, 24:109, 24:129, 24:290, 24:406, 24:424														14																				
*02:24:02								9	10																									
*02:241, 02:684, 02:751																							22											
*02:243:01-02:243:03							8	9	10																									
*02:292																																	29	
*02:332, 24:220			3																						23									
*02:342																						21												
*02:444							8																											
*02:480, 02:924-02:925, 68:176																						20												

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*03:01:18																			19				24										
*03:22:02, 29:83, 29:132, B*15:02:07, B*15:17:03, C*02:02:15, C*03:03:20																			19														
*03:104, 31:88, 66:04		2	3																														
*03:205											12																						
*11:01:28, 11:01:77																							24										
*11:43										10	12																						
*11:98, 11:250, 68:04:01							7																										
*23:03:01											11																						
*26:24				4																		22											
*26:60N, 74:32N				4										14												26							
*26:68, B*27:186				4	5																												
*26:99, 31:29, 31:48				4										14																			
*26:107N, 32:98				4																												31	
*29:02:23									9								17																
*29:03										11							17																
*29:07, 29:49, 29:130														14			17																
*29:09, 29:59, 29:144																	17						24										
*29:12, 29:92					6				9								17																
*29:19							8										17																
*29:22									9	10							17																
*29:48							8	9	10								17																
*29:61																	17																31
*29:66, 31:03										10							17																
*29:116																	17					22											
*29:145																	17									26							
*31:02:01-31:02:02, 31:07-31:08, 31:91, 31:109, 31:164				4						10																							
*31:05, 32:13				4							11																						
*31:24, 32:33:01				4																			24										
*31:54				4			8																										
*31:66, 31:89, 31:115				4								12																					
*31:94				4																													
*31:110, 32:10, 32:63, 32:145, 74:28				4		6														19													
*31:132				4																						26							
*32:51				4																													30
*32:53				4																													
*66:39N				4												16						20											
*68:29, 68:250		1										12																					
C*01:171N																						22									28		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control