

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele^{1,2,3}																																	
*33:01:01:01-33:01:01:15, 33:01:03-33:01:17, 33:49-33:50, 33:67, 33:89, 33:91, 33:121-33:122, 33:127, 33:136, 33:157N, 33:165-33:168, 33:170-33:171, 33:180, 33:182, 33:191, 33:193, 33:197, 33:207-33:208, 33:210, 33:212-33:213N, 33:221, 33:224	1	2	3																														
*33:01:02	1	2	3					9																									
*33:03:01:01-33:03:02, 33:03:04-33:03:62, 33:35, 33:37, 33:40:01-33:41, 33:43, 33:45-33:48, 33:55-33:56, 33:59-33:63, 33:66, 33:71-33:72, 33:75-33:76, 33:78-33:79, 33:81, 33:87-33:88, 33:93-33:95, 33:97-33:105, 33:107-33:108, 33:110, 33:112-33:118, 33:120, 33:124, 33:126, 33:128, 33:130, 33:132-33:135, 33:137-33:139, 33:141-33:142, 33:144-33:146, 33:148-33:156N, 33:158-33:163, 33:169, 33:172-33:179, 33:181, 33:185-33:186, 33:188-33:190, 33:192, 33:194N-33:195, 33:198N-33:206, 33:209, 33:211, 33:214-33:219, 33:222-33:223, 33:225, 33:227-33:230N, 33:233	1			4																													
*33:03:03Q, 33:86 ⁴	1			4																												30	
*33:04	1	2			5																												
*33:05, 33:196	1	2	3			6																											
*33:06	1			4			7																										
*33:07	1	2	3					8																									
*33:08				4					9	10		12																					
*33:09				4					9																								
*33:10	1			4							11																						
*33:11, 33:131, 33:147	1			4								12																					
*33:12	1			4									13																				
*33:13, 26:207 ⁵	1			4						10																							
*33:14	1			4					9																								
*33:15, 33:187 ⁴	1			4										14																			
*33:16, 33:64 ⁴	1	2	3												15																		
*33:17, 33:140N ⁴	1			4												16																	
*33:18:01-33:18:02, 29:105, 29:164 ⁵	1																17																
*33:19	1												14																				
*33:20	1			4		6																											
*33:21	1			4			7										16																
*33:22	1			4					9	10																							
*33:23	1			4											15																		
*33:24	1			4				8																									
*33:25	1			4							11		13																				
*33:26	1			4													17																
*33:27	1	2	3															18															
*33:28	1			4															19														
*33:29, 33:39 ⁴	1			4																20													
*33:30, 33:73N ⁴	1			4																	21												
*33:31, 33:44 ⁴	1			4																		22											
*33:32:01-33:32:02	1	2	3																				23										
*33:33	1			4	5																												
*33:34	1	2	3																					24									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*33:36	1			4												16											26						
*33:42	1			4																										29			
*33:51, 33:119, 31:210, 66:15 ⁵				4			7																										
*33:52	1			4																			23										
*33:53				4			7			10	12																						
*33:54, 33:164 ⁴	1			4																				24									
*33:57	1			4														18															
*33:58	1			4		6			9																								
*33:65	1			4											15		17																
*33:68	1	2	3																														31
*33:69				2	3																					25							
*33:70	1			4																													
*33:74N	1			4																							26	27					
*33:77, 33:220	1			4																							26						
*33:80N	1			4									14														26						
*33:82	1			4																													
*33:83	1			4														18			21												
*33:84	1			4									14												25								
*33:85	1			4																													
*33:90	1			4																													
*33:92		2	3																														

