

Well No.	1	2	3	4	5	6	7	8	9	10	11	12
<b>HLA-A allele<sup>1,2,3</sup></b>												
*34:01:01:01-34:01:07, 34:12, 34:14, 34:16, 34:18, 34:23, 34:27, 34:31	1	2										11
*34:02:01:01-34:02:07, 34:13, 34:15, 34:20-34:21, 34:24-34:26N, 34:28-34:30N	1		3									11
*34:03	1			4								11
*34:04	1		3		5							11
*34:05	1	2				6						11
*34:06	1			4			7					11
*34:07			3				7					11
*34:08	1		3					8				11
*34:09			3					8				11
*34:10N	1		3							10		
*34:11	1	2							9			
*34:17	1	2		4								11
*34:19, 34:22	1											11
*01:13, 01:17, 01:176, 01:302, 03:63, 03:88, 03:382, 11:01:01:01-11:01:67, 11:01:69-11:01:72, 11:01:74-11:01:75, 11:01:77-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:52Q, 11:54:01-11:95, 11:97, 11:99N-11:105, 11:107-11:120, 11:122-11:158, 11:160-11:177, 11:179-11:190, 11:192-11:249, 11:251N-11:256Q, 11:258-11:290, 11:292-11:300, 11:302:01-11:307, 11:309-11:311, 11:313Q-11:324, 11:326-11:398, 11:400N-11:403N, 11:405-11:444, 25:02, 25:75, 26:13, 26:19, 26:33, 26:226, 29:66, 30:125, 30:140, 66:01:01:01-66:01:07, 66:04, 66:06-66:11, 66:13-66:14, 66:17-66:20, 66:23-66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:40-66:42, 66:44-66:47, 68:227, 69:02, C*07:335	1											
*01:51, 02:55, 02:135, 02:644, 02:815, 03:24, 25:03, 25:13, 25:30, 25:82, 26:20:01-26:20:02, 26:30, 26:65, 32:15, 66:02:01:01-66:03:01:03, 66:12, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 68:71, 69:07								8				
*01:226, 26:198, 31:01:07, 32:128, 74:44, B*15:82, B*15:260, B*40:186:01, C*03:186:01, C*04:344, C*12:57:02					5							
*02:91, 02:322, 03:94, 68:239, C*05:149, C*07:81, C*07:243, C*07:878, C*15:161, C*16:76						6						
*02:309, 02:454, 68:294								8				11
*02:741	1	2						8				
*03:01:19, 03:103:02				4				8				11
*03:248, 25:43			3									
*11:96							7					
*11:191, 11:404	1		3									
*11:257	1				5							
*25:06, 26:09, 26:91												11
*25:09, 26:14, 26:28, 26:73, 26:146				4								
*26:18, 26:112:02				4			7					
*26:48, 26:69		2										
*26:112:01				4			w					
*31:03				4			7					11
*31:04:01:01-31:04:02, 31:123, 74:01:03				4			7	8				11
*66:22	1			4								
*66:43				4			7	8				
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>

Negative Control