

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-A allele <sup>1,2,3</sup>																
*66:01:01	1	2	3			6										
*66:02			3	4										14		
*66:03				4			7							14		
*66:04	1	2	3		5	6										
*66:05, 26:92 <sup>4</sup>	1					6		8								
*66:06	1	2	3			6			9							
*66:07	1	2	3		5	6		8								
*66:08, 66:17 <sup>5</sup>	1	2	3			6						12				
*66:09	1	2	3			6				10						
*66:10	1	2	3			6					11					
*66:11	1	2	3											13		
*66:12	1		3												14	
*66:13, 66:19 <sup>5</sup>	1	2	3			6								13		
*66:14	1	2	3			6			9		11					
*66:15	1		3					8								
*66:16			3	4										14	15	
*66:18	1	2	3			6									15	
*01:01:01-01:01:01:27, 01:01:29-01:01:55, 01:01:58-01:04N, 01:06, 01:08-01:12, 01:14-01:16N, 01:18N-01:27N, 01:29-01:33, 01:35-01:70, 01:72-01:99, 01:101-01:104, 01:106-01:142, 03:41, 11:17, 11:40, 11:98, 25:06, 25:18, 26:01:21, 26:04, 26:09, 26:34, 26:73, 26:91, 33:13, 36:01-36:05, 74:10, 80:01:01-01:80:01:01:02, 80:03								8								
*01:01:56	1							8			11			13		
*01:13, 03:63, 03:88, 11:01:01-11:01:50, 11:01:52-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:44, 11:46-11:52Q, 11:54-11:95, 11:97, 11:99N-11:105, 11:107-11:110, 11:112-11:120, 11:122-11:158		2				6										
*01:17, 11:01:51, 11:45, 11:111, C*07:335		2														
*01:28, 02:346, 11:19, 11:121, 80:02						6		8								
*02:03:01-02:03:06, 02:25, 02:38, 02:117, 02:148, 02:171:01-02:171:02, 02:230, 02:253, 02:258, 02:264, 02:267, 02:280-02:281, 02:315, 02:345, 02:355, 02:370, 02:412, 02:431, 02:447										10						
*02:11:01-02:11:05, 02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:69, 02:78, 02:103, 02:128, 02:297-02:298, 02:308, 68:01:01-01:68:02:05, 68:04, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:35, 68:37-68:56, 68:58-68:89, 68:91-68:106, 69:01			3													
*02:16, 02:131							7									
*02:55, 33:24, 68:05, 68:15, 68:20					5											
*02:135, 25:13, 26:65	1														14	
*02:294, 02:453, 32:54											12					
*02:309, 02:454, 03:01:19, 31:04									9						14	
*02:427						6		8		10						
*11:96, 11:106						6										
*24:19			3			6										
*24:44						6		w								
*24:109								w								
*25:01:01-25:01:07, 25:03-25:05, 25:07, 25:09-25:12N, 25:14, 25:16, 25:20-25:23, 26:01:01-26:01:20, 26:01:23-26:01:27, 26:01:29-26:02:02, 26:05, 26:07:01-26:08, 26:10-26:12, 26:14-26:18, 26:20, 26:23-26:28, 26:31-26:32, 26:35-26:42, 26:45-26:46, 26:48, 26:50-26:63, 26:66-26:72, 26:74-26:77, 26:79-26:82, 26:84-26:90, 26:93-26:94, 43:01	1							8								
*25:02, 26:33	1	2														
*25:08, 26:47	1							8							15	
*25:15, 25:17, 26:01:22, 26:01:28, 26:64, 26:83	1															
*25:19, 26:43:01-26:43:02	1							8	9							
*26:03:01-26:03:02, 26:06, 26:21	1				5	6		8								
*26:13, 26:19	1	2	3													
*26:22	1							8		10						
*26:29, 26:49	1							8			11					
*26:30	1				5										14	
*26:78					5	6		8								
*31:03								8	9							
*34:01:01-34:01:02, 34:05		2	3			6							?			
*34:02:01-34:04, 34:06, 34:08		2	3			6			9							
*34:07			3			6			9							
*34:09			3						9						14	
*34:10N-34:11, 69:02		2	3			6										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control