

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-A allele ^{1,2,3}																
*66:01:01-66:01:03, 66:20	1	2	3			6										
*66:02, 66:21			3	4										14		
*66:03				4			7							14		
*66:04	1	2	3			6				11					15	
*66:05	1					6	8									
*66:06	1	2	3			6			9							
*66:07	1	2	3			6	8			11					15	
*66:08, 66:17 ⁴	1	2	3			6						12				
*66:09	1	2	3			6			10							
*66:10	1	2	3			6				11						
*66:11	1	2	3										13			
*66:12	1		3											14		
*66:13, 66:19 ⁴	1	2	3			6							13			
*66:14	1	2	3			6			9	11						
*66:15	1		3				8									
*66:16			3	4										14	15	
*66:18	1	2	3			6									15	
*66:22	1	2	3			6							13		15	
*01:01:01:01:01:27, 01:01:29:01:01:55, 01:01:58:01:01:61, 01:01:63:01:04N, 01:06, 01:08-01:12, 01:14-01:16N, 01:18N-01:27N, 01:29-01:33, 01:35-01:70, 01:72-01:99, 01:101-01:104, 01:106-01:133, 01:135-01:143, 01:145-01:175, 01:178N-01:179N, 03:41, 11:17, 11:40, 11:98, 25:01:09, 25:06, 25:18, 26:04, 26:09, 26:34, 26:91, 26:103, 32:62, 33:13, 36:01-36:05, 74:10, 80:01:01:01-80:01:01:02, 80:03							8									
*01:01:56	1						8			11			13			
*01:13, 03:63, 03:88, 11:01:01:01-11:01:50, 11:01:52-11:05, 11:07-11:11, 11:13-11:16, 11:20-11:27, 11:29-11:39, 11:41-11:44, 11:46-11:52Q, 11:54-11:95, 11:97, 11:99N-11:105, 11:107-11:110, 11:112-11:120, 11:122-11:158, 11:160-11:177, 11:179-11:189, 11:191-11:215N		2				6										
*01:17, 11:01:51, 11:45, 11:111, 11:190, 29:66, C*07:335		2														
*01:28, 02:346, 11:19, 11:121, 80:02						6	8									
*01:134					5		8									
*02:01:09, 02:05:05, 02:06:07, 02:50, 02:76:02, 02:122, 02:243:02, 03:09, 03:108, 03:172, 03:198, 11:18, 24:28, 24:89, 29:19, 29:48, 33:24, 68:05, 68:15, 68:20, 74:06, 74:21					5											
*02:03:01-02:03:08, 02:25, 02:38, 02:117, 02:148, 02:171:01-02:171:02, 02:230, 02:253, 02:258, 02:264, 02:267, 02:280-02:281, 02:315, 02:345, 02:355, 02:370, 02:412, 02:431, 02:447, 02:463, 02:466, 02:480, 02:505, 02:529, 02:544, 02:568									10							
*02:11:01-02:11:07, 02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:69, 02:78, 02:103, 02:128, 02:297-02:298, 02:308, 02:457, 02:490N, 02:494, 02:503, 02:514N, 24:290, 68:01:01:01-68:02:09, 68:04, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:35, 68:37-68:56, 68:58-68:89, 68:91-68:108, 68:110-68:132, 69:01, 69:03			3													
*02:16, 02:131, 02:487, 02:560						7										
*02:135, 25:13, 26:65	1													14		
*02:294, 02:453, 32:54											12					
*02:309, 02:454								9						14		
*02:427						6	8			10						
*02:557										10	12					
*03:01:19, 31:04								9					13	14	15	
*11:06		2			5	6										
*11:96, 11:106						6										
*24:19			3			6										
*24:44						6	w									
*24:109, 24:260							w									
*25:01:01-25:01:08, 25:03-25:05, 25:07, 25:10-25:12N, 25:14, 25:16, 25:20-25:26, 25:27:02-25:29, 25:31, 26:01:01-26:01:21, 26:01:23-26:01:27, 26:01:29-26:02:02, 26:05, 26:07:01-26:08, 26:10-26:12, 26:15-26:17, 26:20, 26:23-26:27, 26:31-26:32, 26:35-26:42, 26:45-26:46, 26:48, 26:50-26:63, 26:66-26:72, 26:74-26:77, 26:79-26:82, 26:84-26:90, 26:93-26:102, 26:104-26:110, 43:01	1						8									
*25:02, 26:33	1	2														
*25:08, 26:47	1						8								15	
*25:09, 26:14, 26:18, 26:28, 26:73	1						8						13		15	
*25:15, 25:17, 26:01:22, 26:01:28, 26:64, 26:83	1															
*25:19:01-25:19:02, 25:30, 26:43:01-26:43:02	1						8	9								
*25:27:01	1						8	9		11						
*26:03:01, 26:06, 26:21, 26:92, 26:111	1				5	6	8									
*26:13, 26:19	1	2	3													
*26:22	1						8			10						
*26:29, 26:49	1						8			11						
*26:30	1				5									14		
*26:78					5	6	8									
*31:03							8	9					13		15	
*34:01:01-34:01:02, 34:05		2	3			6						?				
*34:02:01-34:02:04, 34:04, 34:08		2	3			6		9								
*34:03, 34:06		2	3			6		9					13		15	
*34:07			3			6		9								
*34:09			3					9						14		
*34:10N-34:12, 69:02		2	3			6										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control