

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-A allele^{1,2,3}																
*66:01:01:01-66:01:03, 66:20, 66:23-66:24	1	2	3			6										
*66:02, 66:21, 66:25			3	4										14		
*66:03				4			7							14		
*66:04	1	2	3			6				11					15	
*66:05	1					6		8								
*66:06	1	2	3			6			9							
*66:07	1	2	3			6		8			11				15	
*66:08, 66:17 ⁴	1	2	3			6						12				
*66:09	1	2	3			6				10						
*66:10	1	2	3			6					11					
*66:11	1	2	3											13		
*66:12	1		3												14	
*66:13, 66:19 ⁴	1	2	3			6								13		
*66:14	1	2	3			6			9		11					
*66:15	1		3					8								
*66:16			3	4										14	15	
*66:18	1	2	3			6									15	
*66:22	1	2	3			6								13	15	
*66:26Q			3											13	14	
*66:27N	1	2	3		5	6										
*66:28N				4			7			10					14	
*01:01:01:01:01:01:27, 01:01:29:01:01:55, 01:01:58:01:01:61, 01:01:63:01:04N, 01:06, 01:08:01:12, 01:14:01:16N, 01:18N:01:27N, 01:29:01:33, 01:35:01:70, 01:72:01:99, 01:101:01:104, 01:106:01:133, 01:135:01:143, 01:145:01:175, 01:177:01:189, 01:191:01:193, 01:195:01:203, 01:205:01:228Q, 01:230:01:241, 02:644, 03:41, 11:17, 11:40, 11:98, 25:01:09, 25:06, 25:43, 26:04, 26:09, 26:75, 26:91, 32:62, 33:13, 36:01:36:05, 74:10, 80:01:01:01:80:01:01:02, 80:03								8								
*01:01:56	1							8			11			13		
*01:13, 01:176, 03:63, 03:88, 11:01:01:01:11:01:50, 11:01:52:11:01:67, 11:01:69:11:01:72, 11:01:74:11:01:75, 11:01:77:11:05, 11:07:11:11, 11:13:11:16, 11:20:11:27, 11:29:11:39, 11:41:11:44, 11:46:11:52Q, 11:54:11:95, 11:97, 11:99N:11:105, 11:107:11:110, 11:112:11:120, 11:122:11:158, 11:160:11:177, 11:179:11:189, 11:191:11:214, 11:216:11:222, 11:224:11:249, 11:251N:11:252, 11:254:11:264, 34:14		2				6										
*01:17, 11:01:51, 11:45, 11:111, 11:190, 11:223, 11:253, 29:66, C*07:335		2														
*01:28, 02:346, 11:19, 11:121, 80:02						6		8								
*01:134					5			8								
*01:229, 26:78					5	6		8								
*02:01:09, 02:05:05, 02:06:07, 02:50, 02:76:02, 02:122, 02:243:02, 02:528:02, 02:591:01, 03:09, 03:89:02, 03:91N, 03:108, 03:172, 03:198, 11:18, 24:28, 24:89, 24:379, 29:19, 29:48, 33:24, 68:05, 68:15, 68:20, 74:06, 74:21					5											
*02:03:01:02:03:08, 02:25, 02:38, 02:117, 02:148, 02:171:01:02:171:02, 02:230, 02:253, 02:258, 02:264, 02:267, 02:280:02:281, 02:315, 02:345, 02:355, 02:370, 02:412, 02:431, 02:447, 02:463, 02:466, 02:480, 02:505, 02:529, 02:544, 02:568, 02:595, 02:612, 02:633:02:634, 02:641, 02:666, 02:684										10						
*02:11:01:02:11:09, 02:34:02:35:03, 02:56:01:02:56:02, 02:62, 02:69, 02:78, 02:103, 02:128, 02:297:02:298, 02:308, 02:457, 02:490N, 02:494, 02:503, 02:514N, 02:579:02:580, 02:584, 02:589, 03:242, 23:13, 24:07:01:24:07:02, 24:24, 24:112, 24:131, 24:288, 24:290, 24:294Q, 24:339, 24:347:01:24:347:02, 24:387, 68:01:01:01:68:02:12, 68:04:01:68:04:02, 68:06:68:14, 68:16:68:19, 68:21:01:68:30, 68:32:68:56, 68:58:01:68:89, 68:91:68:108, 68:110:68:169, 69:01:01:01:69:01:02, 69:03			3													
*02:16, 02:131, 02:487, 02:560, 02:693							7									
*02:135, 25:13, 26:65	1														14	
*02:294, 02:453, 02:690, 32:54												12				
*02:309, 02:454									9						14	
*02:427						6		8			10					
*02:557										10		12				
*03:01:19, 31:04, 31:123, 74:01:03									9					13	14	15
*11:01:68, 11:01:76, 11:96, 11:106						6										
*11:06, 11:215N						5	6									
*24:19			3			6										
*24:44						6		w								
*24:109, 24:260								w								
*24:315, 24:374								w								
*25:01:01:01:25:01:08, 25:01:10:25:01:11, 25:03:25:05, 25:07, 25:10:25:12N, 25:14, 25:16, 25:18, 25:20:25:26, 25:27:02:25:29, 25:31:25:35, 25:37:25:42N, 25:44:25:45, 26:01:01:01:26:01:21, 26:01:23:26:01:27, 26:01:29:26:02:02, 26:05, 26:07:01:26:08, 26:10:26:12, 26:15:26:17, 26:20, 26:23:26:27, 26:31:26:32, 26:34:26:42, 26:45:26:46, 26:48, 26:50:26:63, 26:66:26:72, 26:74, 26:76:26:77, 26:79:26:82, 26:84:26:90, 26:93:26:110, 26:113:26:129, 26:131, 26:133:26:145N, 43:01	1							8								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*25:02, 26:33	1	2														
*25:08, 26:47	1							8							15	
*25:09, 26:14, 26:18, 26:28, 26:73, 26:146	1							8						13	15	
*25:15, 25:17, 26:01:22, 26:01:28, 26:64, 26:83, 26:132	1															
*25:19:01:25:19:02, 25:30, 26:43:01:26:43:02	1							8	9							
*25:27:01, 26:130	1							8	9		11					
*25:36								8			11					
*26:03:01, 26:06, 26:21, 26:92, 26:111	1				5	6		8								

Negative Control

51

*26:13, 26:19	1	2	3					8		10								Negative Contr
*26:22	1							8		10								
*26:29, 26:49	1							8		11								
*26:30	1				5											14		
*26:112	1							8	9						13	15		
*31:03								8	9						13	15		
*34:01:01-34:01:02, 34:05		2	3					6					?					
*34:02:01-34:02:04, 34:04, 34:08, 34:13, 34:15		2	3					6		9								
*34:03, 34:06		2	3					6		9					13	15		
*34:07			3					6		9								
*34:09			3							9						14		
*34:10N-34:12, 34:16, 69:02		2	3					6										
*34:17		2	3					6							13	15		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16		