

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48				
*68:32	1	2					7															22	23																								47					
*68:33	1	2																				22	23			26																						47				
*68:34	1		3														17										27																									
*68:35		2																				22	23					28								36												40				
*68:36		2																				22	23						29																			40				
*68:37	1	2																				22	23						29	30																		47				
*68:38	1	2										12										22	23					28																					47			
*68:39	1	2	3																			22	23																										40			
*68:40	1		3													16	17																																			
*68:41	1	2																				22	23																										45	47		
*68:42	1	2																				22	23							29																			40			
*68:43:01-68:43:02	1	2																				22	23					27																					40			
*68:44	1		3														17											28																								
*68:45	1	2																				22	23																										40	41		
*68:46	w	2						8														22	23																										40			
*68:47	1	2						9														22	23																											47		
*68:48	1		3														17																																			
*68:49N	1		3														17																																			
*68:50	1	2																								26																										
*68:51	1		3										13									22	23																												40	
*68:52, 68:142N ⁴	1	2																				22	23																										46	47		
*68:53	1		3														17																																			
*68:54, 68:61	1		3														17																																			
*68:55:01	1	2																				22	23																													
*68:55:02	1	2																				22	23																												47	
*68:57, 68:84, 68:90, 68:109, 68:136		2																				22	23																													
*68:59N	1	2																				22	23																													
*68:60	1		3														17																																			
*68:63	1	2						9															23						29																							
*68:65	1	2																				22																													47	
*68:66:01	1																					22	23																												47	
*68:66:02	1																					22	23																													
*68:67	1		3														17																																			
*68:68, 68:106 ⁴	1	2																				22	23																													
*68:72	1	2																				22	23																													
*68:75:01, 68:130:02	1																					22	23																													
*68:75:02	1																					22	23																													
*68:80	1		3													15	17										26																									
*68:81	1		3													15	17																																			
*68:86	1		3														17																																			
*68:88	1																					22	23																													
*68:91	1	2																				22	23																													
*68:92			3														17																																			47
*68:93	1	2																				22	23																													
*68:94N	1	2																				22	23																													
*68:96	1	2																				22	23																													
*68:97	1		3														17																																			
*68:99, 68:101 ⁴	1	2																				22	23																													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48				

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48								
*11:219	1									10																	27																													
*24:22																																																								
*24:24, 24:67, 24:392, 32:68, 33:119																																																								
*24:141																15												26																												
*24:290	1																																																							
*24:339																15		17																																						
*24:394																																																								
*25:01:01:01, 25:01:02-25:01:03, 25:01:05-25:01:08, 25:02, 25:06-25:07, 25:10-25:11, 25:13-25:14, 25:16, 25:18, 25:21, 25:23-25:25, 25:27:01-25:27:02, 25:29, 26:01:01:01, 26:01:02-26:01:04, 26:01:07-26:01:14, 26:01:16, 26:01:18-26:01:25, 26:01:27-26:01:29, 26:01:31-26:01:37, 26:02:01, 26:03:01-26:06, 26:09, 26:12-26:13, 26:15, 26:17, 26:21, 26:23-26:26, 26:30-26:34, 26:36-26:39, 26:41-26:42, 26:45-26:46, 26:50-26:53, 26:55-26:56, 26:58-26:63, 26:65-26:66, 26:68-26:69, 26:74-26:80, 26:82, 26:84-26:85, 26:87-26:88, 26:90, 26:94-26:109, 26:111, 26:113, 66:03:01:01, 66:05, 66:11																	17																																							
*25:01:01:02, 25:01:09-25:01:12, 25:09, 25:28, 25:31-25:35, 25:37, 25:39-25:42N, 25:45, 26:01:01:02-26:01:01:09, 26:01:15, 26:01:38-26:01:44, 26:14, 26:28, 26:110, 26:114-26:121, 26:123, 26:126-26:131, 26:133-26:145N, 26:147-26:148, 34:14, 66:03:01:02, 66:28N																	17																																							
*25:01:04, 25:03, 25:15, 25:17, 26:01:30, 26:07:01-26:07:02, 26:19-26:20, 26:35, 26:64, 26:72, 26:83, 26:89, 26:92																	17																																							
*25:04, 25:12N, 26:01:05-26:01:06, 26:08, 26:27, 26:48, 26:67, 26:81, 26:86, 26:91, 26:93																																																								
*25:05, 25:22, 25:26, 26:54																	17																																							
*25:08, 26:01:26, 26:02:02, 26:11N, 26:47																	17																																							
*25:19:01, 26:43:01-26:43:02										9																																														
*25:19:02, 26:112									9																																															
*25:20, 26:10																	17																																							
*25:30									9																																															
*25:36, 26:125																	17																																							
*25:38																	17																																							
*25:43																																																								
*25:44, 26:40, 26:73, 26:124, 26:146, 26:149			2																																																					
*25:46																	17																																							
*26:01:17																	17																																							
*26:16																	17																																							
*26:18, 26:122																	17																																							
*26:22																	17																																							
*26:29, 26:49																	17																																							
*26:57																	17																																							
*26:70																	17																																							
*26:71N																	17																																							
*26:132																	17																																							
*29:05, 29:77, 29:87, 29:104, C*16:67																	10																																							
*29:28																																																								
*29:86																																																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48								
*29:105, 33:18:01-33:18:02	2																																																							

Negative Control

