

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
HLA-A allele <sup>1,2,3</sup>																																																		
*68:01:01:01-68:01:01:14, 68:01:03-68:01:05, 68:01:07-68:01:12, 68:01:14, 68:01:18-68:01:20, 68:01:28, 68:01:32, 68:01:35, 68:01:38, 68:01:45, 68:01:58, 68:01:60-68:01:61, 68:56, 68:58:01, 68:76:02, 68:79, 68:87, 68:114:02, 68:119:01, 68:121:02-68:122, 68:126-68:127, 68:133, 68:135, 68:139, 68:143, 68:154, 68:165-68:166, 68:172, 68:175-68:177, 68:185, 68:188, 68:190-68:191, 68:196, 68:209, 68:211-68:213N, 68:223, 68:226, 68:231, 68:239, 68:244:02, 68:246-68:247, 68:252-68:253, 68:267N-68:268, 68:277, 68:281N-68:282, 68:284	1	2																			22	23												36				40												
*68:01:02:01-68:01:02:35, 68:01:06, 68:01:13, 68:01:15-68:01:16, 68:01:21-68:01:27, 68:01:30-68:01:31, 68:01:33-68:01:34, 68:01:36-68:01:37, 68:01:39-68:01:40, 68:01:42-68:01:44, 68:01:46-68:01:57, 68:01:59, 68:58:02, 68:69-68:70, 68:76:01, 68:85, 68:95, 68:98, 68:100, 68:101:02-68:102, 68:107, 68:111, 68:114:01, 68:116, 68:118, 68:119:02, 68:121:01, 68:132, 68:141, 68:146, 68:149-68:150, 68:152, 68:156, 68:162, 68:164, 68:167, 68:173, 68:179, 68:181N, 68:189, 68:194-68:195, 68:197, 68:200, 68:202, 68:205N, 68:207-68:208, 68:210N, 68:214, 68:217-68:218, 68:220-68:222, 68:224, 68:228-68:229, 68:233-68:235, 68:241, 68:243-68:244:01, 68:248, 68:251N, 68:254, 68:256, 68:258, 68:260-68:261, 68:263Q, 68:266N, 68:270, 68:273, 68:279-68:280, 68:283	1	2																			22	23																	40					47						
*68:01:17, 68:73, 68:83, 68:103:02, 68:145, 68:203N, 68:240	1	2																			22	23																								47				
*68:01:29, 68:71, 68:89, 68:155, 68:232	1	2																			22	23																	40											
*68:01:41	1	2		4																	22	23																							47					
*68:02:01:01-68:02:10, 68:02:12-68:02:20, 68:62, 68:64, 68:74, 68:77-68:78, 68:82, 68:125, 68:128, 68:138, 68:147, 68:153, 68:160, 68:163, 68:169-68:170, 68:174, 68:184, 68:186-68:187, 68:193, 68:198, 68:201, 68:216N, 68:219, 68:225, 68:236N-68:237, 68:242, 68:259, 68:272, 68:274-68:275, 68:278	1		3														17																																	
*68:02:11	1		3																																															
*68:02:21, 68:92			3														17																																	
*68:03:01:01-68:03:02, 68:180, 68:206		2		4																		22	23																							40				
*68:03:03, 68:227, 68:269N		2																				22	23																											
*68:04:01		2			5																	22	23																											
*68:04:02		2			5																	22	23																											
*68:05, 68:255		2		4		6																22	23																											
*68:06	1						7																																											
*68:07	1							8														22	23																											
*68:08:01:01-68:08:01:02	1	2							9														23																											
*68:08:02, 68:168	1	2							9														23																											
*68:09, 68:204	1	2								10													23																											
*68:10	1										11											22	23																											
*68:11N	1	2										12											22	23																										
*68:12:01	1	2											13										22	23																										
*68:12:02	1	2											13										22	23																										
*68:13:01, 68:199N	1													14									22	23																										
*68:13:02	1													14									22	23																										

Negative Control

*68:14	1									11			15							22	23											40																			
*68:15		3			6										16	17																																			
*68:16	1	2																			22	23					33														47										
*68:17:01-68:17:02	1	2								11											22	23																			47										
*68:18N	1	3															17							27			34																								
*68:19, 68:158 <sup>4</sup>	1	2												15																											47										
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	<b>33</b>	<b>34</b>	<b>35</b>	<b>36</b>	<b>37</b>	<b>38</b>	<b>39</b>	<b>40</b>	<b>41</b>	<b>42</b>	<b>43</b>	<b>44</b>	<b>45</b>	<b>46</b>	<b>47</b>	<b>48</b>			
*68:20		2	4	6													17					22	23																							40					
*68:21:01	1	2																18					22	23									32														47				
*68:21:02	1	2																18					22	23												36											40				
*68:21:03	1	2																18					22	23								32					36										40				
*68:22	1	2																19					22	23													36										40				
*68:23	1	2																	20				22	23																							47				
*68:24	1	2																		21			22	23																							47				
*68:25	1	2		5					10													21	22	23														36									40				
*68:26	1	2							10									18				w															36	37													
*68:27:01-68:27:02	1																	17																			36														
*68:28	1	3																17				21																													
*68:29	1	2																					22	23		25																					40				
*68:30	1		4																				22	23																								47			
*68:31		3															16	17																																	
*68:32	1	2			7																		22	23																								47			
*68:33	1	2																					22	23		26																						47			
*68:34	1	3																17								27																									
*68:35:01:01-68:35:01:02	1	2																					22	23														36									40				
*68:36		2																					22	23																								40			
*68:37	1	2																					22	23																								47			
*68:38	1	2										12											22	23																								47			
*68:39	1	2	3																				22	23																								40			
*68:40	1	3															16	17																																	
*68:41, 68:265	1	2																					22	23																							45	47			
*68:42	1	2																					22	23															36									40			
*68:43:01-68:43:02	1	2																					22	23			27										36											40			
*68:44	1	3																17																																	
*68:45	1	2																					22	23																							40	41			
*68:46	w	2																					22	23																									40		
*68:47	1	2						9															22	23																								47			
*68:48	1	3																17																																	
*68:49N	1	3																17									26																								
*68:50:01, 68:230	1	2											13										22	23																								40			
*68:50:02	1	2											13										22	23																								47			
*68:51	1	3											13					17																																	
*68:52, 68:142N <sup>4</sup>	1	2																					22	23																								46	47		
*68:53	1	3																17																																	
*68:54, 68:61	1	3																17																																	
*68:55:01	1	2																					22	23																									40		
*68:55:02	1	2																					22	23																									47		
*68:57, 68:84, 68:90, 68:109, 68:136, 68:285	1	2																					22	23																										40	

Negative Control

















C*12:37															w																																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48