

Well No.	1	2	3	4	5	6
<b>HLA-A allele<sup>1,2</sup></b>						
<b>*69:01</b>	1	2	3			
*69:02	1			4		
*69:03	1	2	3		5	
*01:20, 01:66, 02:346, 02:427, 26:22, 66:09				4		
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:70, 02:01:72-02:01:81, 02:01:83-02:01:86, 02:01:88-02:01:109, 02:01:111-02:01:112, 02:03:01-02:03:08, 02:06:01-02:06:17, 02:09, 02:11:01-02:11:07, 02:16, 02:20:01-02:22:02, 02:24:01-02:26, 02:28-02:32N, 02:36, 02:40:01-02:43N, 02:45-02:46, 02:48-02:49, 02:51-02:52, 02:59-02:61, 02:64, 02:66-02:72, 02:74:01-02:77, 02:79:01-02:83N, 02:85-02:93, 02:95-02:97:02, 02:99, 02:101:01-02:101:02, 02:104-02:107, 02:109, 02:111, 02:113N-02:114, 02:116-02:121, 02:123-02:129, 02:131-02:134, 02:136-02:141, 02:143-02:147, 02:149-02:151, 02:153:01-02:153:02, 02:158-02:171:02, 02:173-02:178, 02:181-02:185, 02:187-02:190, 02:192-02:193, 02:195-02:198, 02:200-02:206, 02:208, 02:210-02:211:02, 02:214-02:218, 02:220-02:223N, 02:225N, 02:227N-02:228, 02:230, 02:233-02:241, 02:243:01-02:243:02, 02:246-02:254, 02:256-02:260, 02:262-02:264, 02:266-02:267, 02:269-02:270, 02:272-02:279, 02:281, 02:283-02:285, 02:287-02:299, 02:301N, 02:304-02:318, 02:321N-02:323, 02:325-02:333, 02:336, 02:339-02:343, 02:347-02:349, 02:351-02:358, 02:360-02:368, 02:370-02:372, 02:374-02:375, 02:377-02:397, 02:399-02:402, 02:404-02:407, 02:409-02:412, 02:414-02:416, 02:418-02:420, 02:422-02:425, 02:428, 02:430-02:431, 02:434-02:436, 02:438-02:448, 02:454-02:471, 02:475-02:478, 02:480-02:481, 02:483-02:484, 02:486-02:487, 02:489-02:490, 02:492-02:493, 02:495-02:496, 02:498-02:499, 02:501-02:502, 02:504-02:505, 02:507-02:508, 02:510-02:511, 02:513-02:514, 02:516-02:517, 02:519-02:520, 02:522-02:523, 02:525-02:526, 02:528-02:529, 02:531-02:532, 02:534-02:535, 02:537-02:538, 02:540-02:541, 02:543-02:544, 02:546-02:547, 02:549-02:550, 02:552-02:553, 02:555-02:556, 02:558-02:559, 02:561-02:562, 02:564-02:565, 02:567-02:568, 02:570-02:571, 02:573-02:574, 02:576-02:577, 02:579-02:580, 02:582-02:583, 02:585-02:586, 02:588-02:589, 02:591-02:592, 02:593-02:594, 02:596-02:597, 02:599-02:600, 02:602-02:603, 02:605-02:606, 02:608-02:609, 02:611-02:612, 02:614-02:615, 02:617-02:618, 02:620-02:621, 02:623-02:624, 02:626-02:627, 02:629-02:630, 02:632-02:633, 02:635-02:636, 02:638-02:639, 02:641-02:642, 02:644-02:645, 02:647-02:648, 02:650-02:651, 02:653-02:654, 02:656-02:657, 02:659-02:660, 02:662-02:663, 02:664-02:665, 02:667-02:668, 02:670-02:671, 02:673-02:674, 02:676-02:677, 02:679-02:680, 02:682-02:683, 02:684-02:685, 02:687-02:688, 02:689-02:690, 02:692-02:693, 02:694-02:695, 02:697-02:698, 02:699-02:700, 02:702-02:703, 02:704-02:705, 02:707-02:708, 02:710-02:711, 02:713-02:714, 02:716-02:717, 02:719-02:720, 02:722-02:723, 02:725-02:726, 02:728-02:729, 02:731-02:732, 02:734-02:735, 02:737-02:738, 02:740-02:741, 02:743-02:744, 02:746-02:747, 02:749-02:750, 02:752-02:753, 02:755-02:756, 02:758-02:759, 02:761-02:762, 02:764-02:765, 02:767-02:768, 02:770-02:771, 02:773-02:774, 02:776-02:777, 02:779-02:780, 02:782-02:783, 02:785-02:786, 02:788-02:789, 02:791-02:792, 02:793-02:794, 02:796-02:797, 02:799-02:800, 02:802-02:803, 02:804-02:805, 02:807-02:808, 02:810-02:811, 02:813-02:814, 02:816-02:817, 02:819-02:820, 02:822-02:823, 02:825-02:826, 02:828-02:829, 02:831-02:832, 02:834-02:835, 02:837-02:838, 02:840-02:841, 02:843-02:844, 02:846-02:847, 02:849-02:850, 02:852-02:853, 02:855-02:856, 02:858-02:859, 02:861-02:862, 02:864-02:865, 02:867-02:868, 02:870-02:871, 02:873-02:874, 02:876-02:877, 02:879-02:880, 02:882-02:883, 02:885-02:886, 02:888-02:889, 02:891-02:892, 02:894-02:895, 02:897-02:898, 02:899-02:900, 02:902-02:903, 02:904-02:905, 02:907-02:908, 02:910-02:911, 02:913-02:914, 02:916-02:917, 02:919-02:920, 02:922-02:923, 02:925-02:926, 02:928-02:929, 02:931-02:932, 02:934-02:935, 02:937-02:938, 02:940-02:941, 02:943-02:944, 02:946-02:947, 02:949-02:950, 02:952-02:953, 02:955-02:956, 02:958-02:959, 02:961-02:962, 02:964-02:965, 02:967-02:968, 02:970-02:971, 02:973-02:974, 02:976-02:977, 02:979-02:980, 02:982-02:983, 02:985-02:986, 02:988-02:989, 02:991-02:992, 02:994-02:995, 02:997-02:998, 02:999-02:1000, 03:01:01-03:07, 03:09-03:28, 03:31, 03:33, 03:35, 03:37-03:40, 03:42-03:61, 03:64-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:151, 03:153-03:171, 03:173-03:175, 03:177-03:197N, 03:199-03:200Q, 11:12, 24:24, 29:01:01-01-29:18, 29:21, 29:23-29:29, 29:31-29:33, 29:35-29:47, 29:49-29:65, 29:67-29:71, 30:01:01-30:01:10, 30:08, 30:11:01-30:11:02, 30:14L-30:20, 30:23-30:26, 30:30, 30:35-30:43, 30:48-30:49, 30:52-30:56, 30:58-30:60, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:89, 31:89, 32:17, 34:09, 66:02, 66:12, 66:16, 68:01:01-68:01:17, 68:01:19-68:02:07, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:39, 68:41-68:71, 68:73-68:84, 68:86-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:104:02, 68:106-68:108, 68:110-68:121, 74:13			3			
*02:12-02:13, 02:19, 02:27, 02:37-02:38, 02:44, 02:54, 02:142, 02:226N, 02:280, 02:417, 24:14			3		5	
*02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:78, 03:95, 29:22, 68:105		2	3			
<b>*02:55</b>	1		3			
*02:103, 03:01:01:01-03:07, 03:09-03:28, 03:31, 03:33, 03:35, 03:37-03:40, 03:42-03:61, 03:64-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:151, 03:153-03:171, 03:173-03:175, 03:177-03:197N, 03:199-03:200Q, 11:12, 24:24, 29:01:01-01-29:18, 29:21, 29:23-29:29, 29:31-29:33, 29:35-29:47, 29:49-29:65, 29:67-29:71, 30:01:01-30:01:10, 30:08, 30:11:01-30:11:02, 30:14L-30:20, 30:23-30:26, 30:30, 30:35-30:43, 30:48-30:49, 30:52-30:56, 30:58-30:60, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:89, 31:89, 32:17, 34:09, 66:02, 66:12, 66:16, 68:01:01-68:01:17, 68:01:19-68:02:07, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:39, 68:41-68:71, 68:73-68:84, 68:86-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:104:02, 68:106-68:108, 68:110-68:121, 74:13		2				
*24:93					5	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>

Negative Control