

HLA-A69 2019 3K7

Well No.	1	2	3	4	5	6
HLA-A allele^{1,2}						
*69:01:01:01-69:01:03, 69:04-69:06	1	2	3			
*69:02, 02:644, 02:741 ³	1			4		
*69:03	1	2	3		5	
*01:20, 01:66, 02:346, 02:427, 02:749, 26:22, 66:09				4		
*01:194, 02:103, 03:01:01:01-03:07:02, 03:09-03:29, 03:31, 03:33, 03:35, 03:37-03:40, 03:42-03:61, 03:64-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:151, 03:153-03:171, 03:173-03:175, 03:177-03:193, 03:195-03:197N, 03:199, 03:201-03:259, 03:261-03:262N, 03:264-03:272, 03:274-03:342N, 03:344-03:369, 11:12, 24:24, 24:290, 29:01:01:01-29:18, 29:21, 29:23-29:29, 29:31-29:33, 29:35-29:47, 29:49-29:65, 29:67-29:73, 29:75-29:97, 29:99-29:105, 29:107-29:136, 30:01:01:01-30:01:12, 30:01:14-30:01:15, 30:08:01-30:08:02, 30:11:01-30:11:02, 30:14L-30:20, 30:23-30:26, 30:30, 30:35-30:43, 30:48-30:49, 30:52-30:56, 30:58-30:60, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:89, 30:91-30:98, 30:102, 30:104, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:126, 30:128-30:132N, 30:134-30:138, 30:141-30:143, 30:145N, 30:147-30:148, 30:154, 31:89, 31:115, 32:17, 33:125, 33:131, 34:09, 66:02, 66:12, 66:16, 66:21, 66:25-66:26Q, 66:34, 68:01:01:01-68:01:17, 68:01:19-68:02:15, 68:06-68:14, 68:16-68:19, 68:21:01-68:30, 68:32-68:71, 68:73-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:104:02, 68:106-68:108, 68:110-68:179, 68:181N-68:205N, 68:207-68:211, 68:213N-68:221, 74:13		2				
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:70, 02:01:72-02:01:81, 02:01:83-02:01:86, 02:01:88-02:01:109, 02:01:111-02:01:156, 02:01:158-02:01:168, 02:03:01-02:03:09, 02:06:01:01-02:06:28, 02:09:01:01-02:09:01:02, 02:11:01:01-02:11:09, 02:16, 02:20:01-02:22:02, 02:24:01-02:26, 02:28-02:32N, 02:36, 02:40:01-02:43N, 02:45-02:46, 02:48-02:49, 02:51-02:52, 02:59-02:61, 02:64:01-02:64:02, 02:66-02:72, 02:74:01-02:77, 02:79:01-02:83N, 02:85-02:93:02, 02:95-02:97:02, 02:99:01-02:99:02, 02:101:01-02:101:02, 02:104-02:107, 02:109, 02:111, 02:113:01N-02:114, 02:116-02:121, 02:123-02:129, 02:131:01-02:134, 02:136-02:141, 02:143-02:147, 02:149-02:151, 02:153:01-02:153:02, 02:158-02:171:02, 02:173-02:178, 02:181-02:185, 02:187-02:190, 02:192-02:193, 02:195-02:198, 02:200-02:206, 02:208, 02:210-02:211:02, 02:214-02:218, 02:220-02:223N, 02:225N, 02:227N-02:228, 02:230, 02:233-02:241, 02:243:01-02:243:03, 02:246-02:254, 02:256-02:260, 02:262-02:264:01:02, 02:266-02:267, 02:269-02:270, 02:272-02:279, 02:281, 02:283-02:285, 02:287-02:299, 02:301N, 02:304-02:318, 02:321N-02:323, 02:325-02:333, 02:336, 02:339-02:343, 02:347-02:349, 02:351-02:358, 02:360-02:368, 02:370-02:372, 02:374-02:375, 02:377-02:397, 02:399-02:402, 02:404-02:407, 02:409-02:412, 02:414-02:416, 02:418-02:420, 02:422-02:425, 02:428, 02:430-02:431, 02:434-02:436, 02:438-02:448, 02:454-02:471, 02:475-02:476N, 02:479-02:483, 02:485-02:488, 02:490N-02:491, 02:493-02:494, 02:497-02:498, 02:500Q-02:501N, 02:503-02:506N, 02:508-02:510, 02:512, 02:514N-02:516N, 02:518-02:525N, 02:528:01-02:530, 02:533-02:539, 02:542, 02:545, 02:547, 02:549-02:557, 02:559-02:564, 02:568-02:571, 02:573-02:574, 02:576, 02:578-02:579, 02:581-02:582, 02:584-02:585, 02:587, 02:589-02:590, 02:592, 02:594-02:598, 02:600, 02:602-02:603, 02:605Q-02:616, 02:618Q-02:621, 02:623, 02:625, 02:627, 02:629, 02:632-02:635, 02:637-02:642, 02:645, 02:647-02:651, 02:654, 02:656, 02:658-02:661, 02:663, 02:665-02:666, 02:668-02:669, 02:671-02:675N, 02:677, 02:679, 02:681-02:708, 02:711-02:716, 02:718-02:720, 02:722, 02:724-02:730, 02:732-02:733, 02:735, 02:737-02:740, 02:742-02:748N, 02:750-02:763, 02:765, 02:767-02:770, 02:772-02:777, 02:779-02:786, 02:788N-02:789N, 02:792N-02:798, 02:800-02:803N, 02:805Q-02:806N, 02:808-02:814, 02:816, 02:819-02:821, 02:823-02:825, 02:827Q-02:828, 02:830-02:839, 02:841, 02:844-02:854, 02:856-02:858N, 02:860-02:866, 02:868-02:871N, 02:873-02:877, 31:99, 33:22			3			
*02:12-02:13, 02:19, 02:27, 02:37-02:38, 02:44, 02:54, 02:142, 02:226N, 02:280, 02:417, 02:543, 02:662, 02:723, 24:14:01:01-24:14:01:04, 24:324					5	

Negative Control

HLA-A69 2019 3K7

*02:34-02:35:03, 02:56:01-02:56:02, 02:62, 02:78, 02:580, 03:95, 03:343, 29:22, 68:105		2	3			
*02:55, 02:815	1		3			
*02:630, 02:840, 24:93					5	
*11:288				4	5	
Well No.	1	2	3	4	5	6