

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-A allele^{1,2,3}																
*74:01:01:01-74:01:08, 74:25, 74:28-74:29, 74:31, 74:36, 74:40, 74:43	1	2														
*74:02:01:01-74:02:01:03, 74:34, 74:37, 74:41	1		3													
*74:03:01:01-74:03:03, 74:38	1	2							9							
*74:04	1	?	?	4												
*74:05	1	?	?		5											
*74:06, 74:21	1		3			6										
*74:07	1		3				7									
*74:08, 74:16:01⁴	1	?	?					8								
*74:09	1	2								10						
*74:10	1	?	?								11					
*74:11	1	2										12				
*74:12N	1	?	?										13			
*74:13	1	2												14		
*74:14N	1	?	?						9		11					
*74:15	1	?	?													15
*74:16:02	1	2						8								
*74:17	1	?	?							10			13			
*74:18-74:20, 74:22-74:24, 74:26-74:27	1	?	?													
*74:30	1		3								11					
*74:32N	1	2			5				9							
*74:33	1	2					7		9							
*74:35	1		3		5											
*74:39	1	2				6										
*74:42	1		?													
*01:01:68, 01:69:02, 03:01:03:01-03:01:03:02, 11:01:106				w												
*01:83:02, 01:405, 02:01:09, 02:03:11, 02:05:05, 02:06:07, 02:07:12, 02:11:02, 02:35:03, 02:76:02-02:76:03, 02:243:02-02:243:03, 02:528:02, 03:23:01, 03:198, 11:70:02, 11:121, 23:01:21, 24:02:76, 24:310:02, 25:83, 26:07:02, 33:01:02, 33:08-33:09, 66:42, 68:01:41, 68:30				4												
*01:121, 02:65, 02:407, 02:449, 02:932, 03:246, 03:282, 11:07:02, 11:372, 68:25, 68:262	1															
*02:01:165, 29:159			3			6										
*02:1044, 34:25												12				
*02:1076										10						
*03:01:39, 29:172, 68:01:28, 68:01:36, 68:130:02			3											14		
*03:27, 33:81, 33:122										10			13			
*29:01:01:01-29:01:06, 29:01:08-29:02:22, 29:02:24, 29:02:26-29:02:32, 29:02:34-29:02:35, 29:02:37-29:04, 29:09-29:10:01, 29:11-29:12, 29:14, 29:23, 29:30, 29:32, 29:37, 29:40, 29:46, 29:56-29:58, 29:70-29:71, 29:84, 29:87-29:88, 29:90, 29:92-29:95, 29:100, 29:101:02, 29:103, 29:109-29:110, 29:114-29:124, 29:126Q-29:135, 29:137-29:138, 29:140-29:157, 29:160-29:171, 32:01:01:01-32:01:17, 32:01:19-32:01:21, 32:01:23-32:03:01:02, 32:05-32:08, 32:11Q, 32:15, 32:17, 32:19N, 32:24, 32:26:02-32:27N, 32:30:02-32:31, 32:33:03-32:34, 32:43:02-32:44, 32:46:02, 32:54-32:55:01, 32:55:03, 32:60-32:61, 32:65, 32:68-32:69, 32:86, 32:92N-32:94, 32:97-32:98, 32:101Q-32:105, 32:107, 32:109-32:114, 32:116-32:119, 32:121-32:122, 32:124-32:126N, 32:128-32:129, 32:131-32:152, 32:154-32:159, 32:161-32:171			3													
*29:02:23			3	4												
*29:05-29:08N, 29:10:02, 29:13, 29:15, 29:17-29:18, 29:20-29:22, 29:24-29:29, 29:31, 29:33-29:36, 29:38, 29:42-29:45, 29:47, 29:49-29:55, 29:59-29:65, 29:67-29:69, 29:72-29:83, 29:85-29:86, 29:89, 29:91, 29:96-29:99, 29:101:01, 29:102, 29:104-29:108, 29:111-29:113, 29:125, 29:136, 32:09-32:10, 32:12-32:14, 32:16, 32:18, 32:20-32:23, 32:25-32:26:01, 32:28-32:30:01, 32:32-32:33:02, 32:35-32:43:01, 32:45N-32:46:01, 32:47-32:53, 32:55:02, 32:56N-32:59, 32:63-32:64, 32:66-32:67, 32:70-32:74, 32:76-32:85, 32:87-32:91, 32:95-32:96, 32:99-32:100, 32:108, 32:120, 32:123, 32:127, 32:130N, 32:153			?													
*29:16, 32:75			?							10			13			
*29:19			?			6										
*29:39			3				7									15
*29:48			?	4		6										
*29:66			?								11					
*29:139			3							10			13			
*31:04:02																15
*31:138								8								
*32:62			3								11					
*32:106:01:01-32:106:01:02		2														
*32:160N			3			5										
*33:80N						5										
C*04:01:51, C*04:03:08, C*06:02:15, C*07:02:76											11					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control