

INTERPRETATION TABLE

HLA-B low resolution screening SSP

Amplification patterns of the B*07:02 to B*83:01 alleles¹⁴

	Well																							
	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.	110	215	140	130	185	180	290	165	165	180	100	280	195	105	115	80	150	140	60	210	170	110	395	160
PCR product(s)			235	265	235			330 285 220	390 190		195			435 395	150				400 245					425
Length of int. pos. control ¹	800	1070	1070	800	800	800	1070	1070	1070	800	1070	1070	1070	1070	1070	1070	800	1070	1070	800	800	1070	1070	1070
5'-primer(s) ²	5'-gAC ^{3'} 540	5'-TCg ^{3'} 97	5'-AgC ^{3'} 363	5'-CCg ^{3'} 103	5'-CCT ^{3'} 103	5'-CCg ^{3'} 103	5'-gga ^{3'} 45	5'-gga ^{3'} 45	5'-gga ^{3'} 45	5'-TTA ^{3'} 420	5'-AgA ^{3'} 206	5'-gTC ^{3'} 419	5'-TCT ^{3'} 142	5'-gTC ^{3'} 419	5'-gga ^{3'} 45	5'-Cgg ^{3'} 161	5'-gCT ^{3'} 167	5'-TCA ^{3'} 355	5'-gAC ^{3'} 206	5'-gTT ^{3'} 368	5'-TCT ^{3'} 142	5'-TTA ^{3'} 420	5'-gAA ^{3'} 246	5'-gTC ^{3'} 357
3'-primer(s) ³	5'-gCT ^{3'} 605	5'-TgA ^{3'} 272	5'-gCC ^{3'} 312	5'-CgT ^{3'} 193	5'-TAT ^{3'} 246	5'-TAT ^{3'} 246	5'-Tgg ^{3'} 165	5'-gTC ^{3'} 538	5'-CAG ^{3'} 559	5'-CTC ^{3'} 559	5'-Tgg ^{3'} 272	5'-TAT ^{3'} 3 rd 1	5'-gTC ^{3'} 301	5'-ATC ^{3'} 309	5'-TgA ^{3'} 272	5'-TCT ^{3'} 234	5'-TCT ^{3'} 204	5'-gCT ^{3'} 463	5'-AGT ^{3'} 312	5'-gTT ^{3'} 259	5'-ggg ^{3'} 311	5'-ggT ^{3'} 544	5'-gga ^{3'} 317	5'-ggT ^{3'} 272

*15:03:01-15:03:02, 15:03:04, 15:47:01- 15:47:02, 15:49, 15:61, 15:98, 15:103, 15:123, 15:127, 15:132, 15:151, 15:156, 15:158, 15:173, 15:210, 15:220, 15:235, 15:242, 15:266, 15:274, 15:281-15:282	B72, B70/blank, -							8			11	12						19							
*15:03:03	B72							7	8			11	12						19						
*15:04	B62							7					12						19						
*15:07:01-15:07:02, 15:126, 15:207	B62, -			3				7					12						19						
*15:08, 15:11:01- 15:11:03, 15:11:05, 15:15, 15:148, 15:191, 15:209N, 15:215	B75/B62, B75/62, Null, -							7					12						19						
*15:09	B70							8	9					14					19						
*15:10:01, 15:37, 15:90, 15:99, 15:133, 15:229, 15:252	B71, B70, -							8						14					19						
*15:10:02	B71							7	8					14					19						
*15:11:04	B75												12						19						
*15:12, 15:14, 15:19, 15:270	B76, -							7					12	13					19						
*15:13:01-15:13:02	B77, -							7					12	14			17		19						
*15:16:01-15:16:02, 15:67, 15:95, 15:254	B63, -									9			12	14					19						
*15:16:03	-									9			12	14					19						
*15:17:01:01-15:17:02, 15:162, 15:168, 15:177, 15:196, 15:208, 15:216, 15:273	B63, -							7	9					14					19						
*15:18:01-15:18:02, 15:18:04-15:18:05, 15:51- 15:52, 15:72, 15:93, 15:108, 15:114, 15:119, 15:134, 15:153, 15:176, 15:197-15:198, 15:200, 15:221, 15:226N, 15:238, 15:263, 15:275	B71, B70, Null, -							8					12	14					19						
*15:18:03	B71							8					12	14					19						
*15:20	B62							7					12				17		19						
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele⁴	ser.⁵																								
*15:21, 15:44	B75							7					12	14				17	19						

*15:23, 15:115	B70/B5/blank							8				12		14				19							
*15:24:01-15:24:02, 15:87, 15:157	B62, B15, -							7				12		14				19							
*15:25:01-15:25:03, 15:204, 15:240, 15:250, 15:271	B62, -							7				12				17		19							
*15:29	B70							8				12						19							
*15:30, 15:58, 15:63, 15:269	B62, -							7										19							
*15:31	B75							7				12						19							
*15:36	-		w					7				12				17		19							
*15:42	B15							7										19							
*15:43, 15:256	B15, -							7				12						19							
*15:45	B62		3					7										19							
*15:46	-			4				7	8			12						19							
*15:48, 15:150, 15:188, 15:248	B62, -							7										19							
*15:53, 15:212	-			4				8				12						19							
*15:54	B72							8			11	12						19							
*15:55	B15		3					7				12						19							
*15:57	B62							7				12						19							
*15:62	-							8			11	12				17		19							
*15:64:01-15:64:02	B71, -							8			11	12						19							
*15:66	B62							7				12		14				19							
*15:68	-		3								11	12													
*15:69	B72							8			11	12						19		21					
*15:73	B62							7										19							
*15:74	B72							8				12						19							
*15:76	-							7				12						19							
*15:77	B62					6	7										17	19							
*15:80	B70							8				12		14			17	19							
*15:83	B62							7	8									19							
*15:85	B62							7				12					17	19							
*15:86	-							7										19							
*15:89	B77		w					7				12					17	19							
*15:91, 15:131	-							8			11	12	13					19							
*15:101	-							7				12													
*15:106	-			4				7	8			12					17	19							
*15:116	B62							7				12						19							
*15:124	B71							8				12		14				19							
*15:137	-							7										19							
*15:138	-	1						7				12						19							
*15:142	-		2					7				12						19							
*15:143	-							7	8			12						19							
*15:144	-							7				12					17	19							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

HLA-B allele ⁴	ser. ⁵																			
*15:154, 15:165	-					7				12					17		19			
*15:161	-							8			12	13	14				19			
*15:180	-		2								12									
*15:186	-							8			12		14				19		21	
*15:189	-					6	7				12		14				19			
*15:194	-							7			12				17		19			
*15:202, 15:239	-							7			12					18	19			
*15:222	-								9			12		14			19			
*15:224	-							7									19			
*15:230	-		1					7	9				14				19			
*15:233	-					6	7										19			
*15:236	-							7					12							
*15:241	-		1					7					12				19			
*15:243	-							8	9				14				19			
*15:253	-							8		11							19			
*15:255	-							8			12									
*15:259	-							7	8		11	12					19			
*15:268	-							7	9			12		14			19			
*15:277	-							7	9			12					19			
*18:01:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:53, 18:55, 18:57-18:60, 18:62-18:65, 18:68, 18:70-18:71, 18:73-18:78, 18:80-18:91	B18, B18/blank, Null, -												12		15		18			
*18:04	B18												12		15		18			
*18:09	B18												12		15					
*18:12	B18												12		15		18			
*18:19	B18												12		15		18			
*18:22, 18:69	B18, -												12		15		17	18		
*18:26, 18:41, 18:61	B18, -														15		18			
*18:29, 18:72, 35:32:01-35:32:02, 35:37, 35:53N, 35:64:01-35:64:02, 35:66, 35:68:01-35:68:02, 35:99, 35:118-35:119, 35:174 ⁹	B35, Null, -													12			18			
*18:37	-										11	12			15					
*18:44:01-18:44:02	-			4									12		15		18			
*18:48	-												12		15					
*18:49, 18:79	-												12		15					
*18:54	-											12	13	15					20	
*18:56	-											12	13	15			18			

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ⁴	ser. ⁵																								
*35:01:01:01-35:01:09, 35:01:11-35:01:38, 35:07- 35:08:04, 35:08:06, 35:11:01-35:11:03, 35:15, 35:20:01-35:20:02, 35:23, 35:29:01-35:29:02, 35:40N-35:42:01, 35:48, 35:50, 35:52, 35:54, 35:57, 35:61:01-35:61:02, 35:77-35:78, 35:82, 35:90- 35:94, 35:100-35:101:02, 35:103-35:105, 35:107- 35:108:02, 35:110-35:112, 35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:158- 35:159, 35:161, 35:165N- 35:166, 35:168, 35:170- 35:171, 35:173N, 35:175- 35:178, 35:180, 35:187, 35:189, 35:192, 35:194, 35:196-35:197, 35:200, 35:202, 35:206, 35:208- 35:209, 35:212, 35:215- 35:216N, 35:219, 35:224- 35:225, 35:227-35:228	B35, Null, -											12					17	18							
*35:01:10	B35									9		12						17	18						
*35:02:01-35:03:02, 35:03:04-35:04:01, 35:04:03, 35:06, 35:12:01- 35:12:03, 35:33-35:34, 35:36, 35:39, 35:55, 35:59, 35:65Q, 35:70, 35:74, 35:83-35:85, 35:95, 35:98, 35:106, 35:128- 35:129N, 35:136, 35:149- 35:150, 35:152-35:153, 35:155-35:156, 35:160, 35:162, 35:167, 35:172, 35:179, 35:181-35:184, 35:193, 35:195, 35:201, 35:204, 35:211, 35:214, 35:220-35:221, 35:223, 35:231	B35, Null, -																	17	18						

*35:03:03, 35:56	B35																	17							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ⁴	ser. ⁵																								
*35:04:02	B35									9								17	18						
*35:05:01-35:05:03, 35:51, 35:58, 35:72, 35:89, 35:97, 35:114, 35:232	B35, -			3									12						18						
*35:08:05	-												12					17	18						
*35:09:01-35:09:03, 35:18, 35:31, 35:75, 35:88, 35:127, 35:151, 35:205	B35, -																		18						
*35:10, 35:28, 35:69, 35:80, 35:226	B35, -												12						17	18					
*35:13	B35																		17	18					
*35:14:01-35:14:02, 35:62	B35												12						17	18					
*35:16	B35			3									12						17	18					
*35:17, 35:30, 35:113, 35:125	B35, -			3									12						17	18					
*35:19	B35/blank												12						17						
*35:21, 35:24:01- 35:24:02, 35:188, 35:190	B35/B78, -												12						17	18					
*35:22, 35:199	B35, -			3															18						
*35:25, 35:49, 35:210	B35, -												12						17						
*35:26	B35						6						12						17						
*35:27, 35:203	B35, -												12						17						
*35:35	B35												12						17	18		21			
*35:38, 35:169	B35, -																		17	18					
*35:42:02	-												12						17	18					
*35:43:01, 35:67, 35:79, 35:86, 35:102, 35:117, 35:135	B35, -												12						18						
*35:44	-																		18						
*35:45, 35:71	B35												12	13					17	18					
*35:46	B35				4								12						17						
*35:47, 35:217	B35, -				4								12						17						
*35:60	B35																		18						
*35:63	-				4								12						17						
*35:76	B35/22												12						17	18					
*35:81, 35:96, 35:109, 35:157, 35:233	-																		17	18					
*35:87	-								8										18						
*35:115	-												12						17	18					
*35:154	-				4														17						

*35:163, 35:213	-																17	18							
*35:164	-			3													17	18							
*35:185	-											12						18							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele⁴	ser.⁵																								
*35:186	-												12					17	18						
*35:191	-												12					17	18						
*35:198	-																	17							
*35:207	-				4								12					17							
*35:218	-		2										12					17	18						
*35:222	-												12					17	18						
*35:229	-												12					17	18						
*37:01:01-37:01:07, 37:01:09, 37:03N, 37:06, 37:13, 37:15-37:21, 37:23- 37:27, 37:30N-37:33N, 37:36	B37, Null, -													13				17			20				
*37:01:08, 37:09	B37, -													13							20				
*37:02	B37/B27					5								13							20				
*37:04:01-37:04:02	B37, -													13				17			20				
*37:05	Blank													13				17			20				
*37:07	-	1												13							20				
*37:08	B37													13				17			20				
*37:10, 37:22, 37:29	B37, -																	17			20				
*37:11, 37:37	-																	17	18		20				
*37:12	B37								8					13							20				
*37:14	-																	17			20				
*37:28	-											11		13				17			20				
*37:34	-																	17			20				
*37:35	-												12	13				17			20				
*38:01:01-38:02:02, 38:02:04-38:02:05, 38:08- 38:09, 38:11-38:16, 38:18- 38:19, 38:21, 38:23-38:24, 38:27-38:29, 38:31-38:32, 38:34N, 38:38-38:43	B38, Null, -					5																	21	22	23
*38:02:03	-																					21		23	
*38:03	B38					5						11										21	22		
*38:04, 38:25	B38, -																					21		23	
*38:05, 38:33	B38, -						6															21	22	23	
*38:06	B38																					21			
*38:07	B38					5																21			
*38:10, 38:36-38:37	B38, -					5																	22	23	
*38:17	-					5								13							20	21	22	23	
*38:20	-					5												17				21	22	23	
*38:22	-					5											16						22	23	

*38:26	-					5																21	22		
*38:30	-					5		8															22	23	
*38:35	-					5																21		23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ⁴	ser. ⁵																								
*39:01:01:01-39:01:01:03, 39:01:03-39:01:08, 39:01:10-39:01:14, 39:01:16, 39:03, 39:05:01- 39:06:02, 39:09, 39:11, 39:14-39:15, 39:24:01, 39:26-39:29, 39:31, 39:35, 39:37-39:38Q, 39:40:01N- 39:41, 39:44, 39:46, 39:48, 39:51-39:57, 39:59- 39:62, 39:64-39:68, 39:70- 39:71, 39:73, 39:75, 39:77, 39:79-39:81	B3901/B 39, Low B39, Null, -					5																21		23	24
*39:01:09, 39:06:03, 39:12, 39:45, 39:72, 39:76, 39:78	B3901/B 39, -																					21		23	24
*39:01:15	-					5																21		23	24
*39:02:01, 39:08	B3902/B 39					5						11										21			24
*39:02:02, 39:13:01- 39:13:02, 39:23, 39:49	B3902/B 39, -											11										21			24
*39:04	B39						6															21		23	24
*39:07	B16					5							12									21		23	24
*39:10:01-39:10:02, 39:16, 39:20, 39:83	B39, -																					21			24
*39:17	B39																					21			24
*39:18	B39					5																21		23	24
*39:19:01	B39																		18			21		23	24
*39:19:02	B39					5													18			21		23	24
*39:22	B39					5																21		23	24
*39:24:02	-					5																21		23	
*39:36	-					5																		23	24
*39:39:01	B39					5						11										21			24
*39:39:02	-											11										21			24
*39:42	B39					5													17			21		23	24
*39:43	-					5							12											23	24
*39:58	-					5																21			24
*39:63	-																					21			24
*39:69	-					5																21		23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

*40:48	B60				4								10												?
*40:53, 40:72:02	B61, B60				4																				24
*40:58	B40				4																				?
*40:59-40:60	B40, -				4								12												24
*40:68, 40:232	-																								?
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele⁴	ser.⁵																								
*40:71	B61				3																				?
*40:74, 40:149, 40:210	B60, -																								?
*40:76	B60				4																				
*40:83, 40:177	-																								24
*40:93	-												11				16								24
*40:95	-																								?
*40:96	-																								24
*40:98	-	1																							?
*40:105	-												12												?
*40:109	-																								24
*40:110	-				4																				24
*40:117	-				4																				
*40:132, 40:178	-				4																				?
*40:136	-	1			4				8								14								?
*40:138	-																								24
*40:148, 40:161	-																								24
*40:157, 40:201	-																								
*40:158	-	1			4								12												?
*40:159	-												12												24
*40:166	-				4																				?
*40:170	-																								?
*40:174	-				3																				?
*40:183, 40:198	-	1			4																				?
*40:184	-																					21			?
*40:185	-																				18				?
*40:208	-				4																				?
*40:209	-																				18				?
*40:215	-				4																				?
*40:231	-	1			4				8								14								?
*41:01, 41:06-41:07, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25, 41:28	B41, -				4				8																24
*41:02:01-41:03:02, 41:11, 41:13, 41:15, 41:19, 41:23, 41:27	B41, -				4				8																24
*41:04, 41:10, 41:18	B41, -				4																				24
*41:05	B41				4				8																24
*41:08	B41	1			4				8								14								24

*47:01:01:01-47:01:02, 47:06-47:08	B47, -				4																				
*47:02	B47				4																				
*47:03	B47				4																				
*47:04-47:05	B17, -				4	5																			
*48:01:01-48:01:04, 48:04, 48:07, 48:09, 48:11, 48:19-48:20, 48:22, 48:24, 48:27, 48:29-48:30	B48, -											11									24				
*48:02:01-48:02:03, 48:25	B48/B70, -											11	12			17					24				
*48:03:01-48:03:02, 48:17	B48, -											11									24				
*48:05, 48:08	B48, -	1										11									24				
*48:06	-																				24				
*48:10, 48:12-48:13	B48, -											11									24				
*48:14	B48											11	12								24				
*48:15	B48	1										11									24				
*48:16	B48											11									24				
*48:18	B48											11													
*48:21, 48:26	-					5						11									24				
*48:23	-											11	12								24				
*48:28	-											11			15						24				
*48:31	-																				24				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ⁴	ser. ⁵																								
*49:01:01, 49:01:03- 49:01:04, 49:06, 49:08- 49:10, 49:12-49:17, 49:19N, 49:22-49:24	B49, Null, -				4																				
*49:01:02, 49:11	B49, -				4																				
*49:02	B49				4																				
*49:03	B49																								
*49:04-49:05	B49				4								12												
*49:07	-				4							10													
*49:18	-				4							9													
*49:20	-				4								11												
*49:21	-				4																				
*49:25	-		2																						
*49:26	-															16							22	23	
*50:01:01-50:01:03, 50:04- 50:05, 50:08, 50:10-50:11, 50:13, 50:16, 50:18-50:19, 50:32	B50, -				4																				24
*50:02	B45				4							11													24
*50:06	-				4																				24

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
																								ser. ⁵	HLA-B allele ⁴
			28																				47	B7, B7/blank , B7/B42, Null, -	*07:02:01-07:02:31, 07:02:33-07:02:39, 07:05:01-07:07, 07:10, 07:12-07:15, 07:18:01- 07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34- 07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55- 07:59, 07:61-07:64, 07:66- 07:68:03, 07:70-07:72, 07:74-07:77, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:121, 07:124- 07:130, 07:132, 07:135N- 07:138, 07:140-07:145, 07:147-07:148, 07:150- 07:154, 07:156-07:161N, 07:163, 07:166-07:173, 07:175-07:178, 07:181N, 07:183-07:191, 07:193- 07:194, 07:196, 07:198
			28																				47	-	*07:02:32, 07:197
																							47	B703/B7, B7/blank	*07:03, 07:08, 07:16, 07:32, 07:37
		27	28																				47	B7, -	*07:04, 07:25, 07:146
			28																				47	B7, -	*07:09, 07:11, 07:17, 07:162
			28																				47	B7/42, -	*07:19, 07:33:01- 07:33:02, 07:40, 07:48, 07:51, 07:53, 07:122, 07:139, 07:155, 07:164- 07:165
			28																				47	B7/blank , -	*07:20, 07:24, 07:131
																							46	-	*07:27
			28																				47	B7	*07:28
			28																				46	B7, -	*07:36, 07:81, 07:149
																							46	-	*07:38
			w																				47	B7	*07:47
																							47	-	*07:50
			28									38											47	B7, -	*07:54, 07:123
			28																				47	-	*07:60

			28				32									40			w							47	-	*07:65
																										47	-	*07:69, 07:85
			28																							47	-	*07:73
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48				Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48				Well No.	
																											ser. ⁵	HLA-B allele ⁴
			28							35																47	-	*07:78
			28							35																47	-	*07:84
			28																		45				47	-	*07:100	
			28									37													47	-	*07:133	
			28																w						47	-	*07:134	
			28					33																	47	-	*07:174	
			28																						47	-	*07:179, 07:195	
																									46	-	*07:180	
			28														43								47	Null	*07:182N	
			28																								-	*07:192
			27																						47	B8, Null, -	*08:01:01-08:01:21, 08:05, 08:08N, 08:10- 08:12:03, 08:15-08:16, 08:18-08:19N, 08:22- 08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41- 08:48, 08:51, 08:53:01- 08:53:02, 08:56:01- 08:59:02, 08:61, 08:63- 08:64, 08:66-08:69, 08:71- 08:73, 08:75, 08:80-08:83, 08:86N, 08:90-08:93, 08:95-08:100, 08:104- 08:106, 08:109	
			27																						46	B8, -	*08:02-08:03, 08:52, 08:78	
			27																						47	B8/blank , -	*08:04, 08:110	
			27																						47	B8, -	*08:07, 08:14, 08:74, 08:77, 08:108, 08:111	
			27			30																			47	B8, -	*08:09, 08:84	
																									47	B8, -	*08:13, 08:20, 08:40, 08:70, 08:103	
			27									38													47	-	*08:17, 08:54	
			27																						47	B8	*08:21	
																									47	B8	*08:25	
			27																						47	B8, -	*08:26, 08:50, 08:62, 08:85, 08:94, 42:07 ⁶	
			27																						47	B8, -	*08:28, 08:37, 08:89, 08:107	
			27																	44					47	B8	*08:32	
			27																						47	-	*08:35	

												ser. ⁵	HLA-B allele ⁴								
												38			45	47	-	*15:154, 15:165			
																47	-	*15:161			
															45	47	-	*15:180			
																47	-	*15:186			
												33			45	47	-	*15:189			
26												33				47	-	*15:194			
															38	45	47	-	*15:202, 15:239		
																46	-	*15:222			
												38	40	43		47	-	*15:224			
																46	-	*15:230			
												38			45	47	-	*15:233			
												38			45	47	-	*15:236			
												38			45	47	-	*15:241			
												38				47	-	*15:243			
												38				47	-	*15:253			
															38		47	-	*15:255		
												28			38		47	-	*15:255		
															38		47	-	*15:259		
																46	-	*15:268			
															38		45	-	*15:277		
																		Negative Control	*18:01:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:53, 18:55, 18:57-18:60, 18:62-18:65, 18:68, 18:70-18:71, 18:73-18:78, 18:80-18:91		
															44	47	B18, B18/blank, Null, -				
															40	44	47		B18	*18:04	
																44	46		B18	*18:09	
															38		47		B18	*18:12	
																44	45		47	B18	*18:19
																44	47		B18, -	*18:22, 18:69	
																44	47		B18, -	*18:26, 18:41, 18:61	
																44	47		B35, Null, -	*18:29, 18:72, 35:32:01-35:32:02, 35:37, 35:53N, 35:64:01-35:64:02, 35:66, 35:68:01-35:68:02, 35:99, 35:118-35:119, 35:174 ⁹	
																	47		-	*18:37	
															44	47	-	*18:44:01-18:44:02			
26												37				47	-	*18:48			
															44	47	-	*18:49, 18:79			
															44	46	-	*18:54			
															44	47	-	*18:56			

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
																									ser. ⁵	HLA-B allele ⁴
																									-	*18:66
											36								44						-	*18:67
			28				w						38												B27	*27:01
			28				32				36		38												B27, Null, -	*27:02:01-27:02:02, 27:53, 27:57, 27:65N, 27:83, 27:95, 27:102
			28				32						38												B27, Null, -	*27:03-27:05:15, 27:05:17- 27:05:19, 27:05:21-27:06, 27:09-27:10, 27:13, 27:17, 27:20, 27:27, 27:31, 27:35- 27:37, 27:45-27:46, 27:48- 27:51, 27:54-27:56, 27:58, 27:60-27:61, 27:64N, 27:66N, 27:68-27:69, 27:72-27:74, 27:78-27:80, 27:82, 27:84-27:88, 27:90:01-27:91, 27:93- 27:94N, 27:96-27:101, 27:103-27:106
			28				32						38												B27, -	*27:05:16, 27:05:20, 27:07:01-27:07:03, 27:11, 27:15, 27:21, 27:28, 27:32, 27:34, 27:38, 27:43, 27:47, 27:63, 27:70- 27:71, 27:76
			28				32						38												B2708/B 27/B7, -	*27:08, 27:26, 27:40, 27:42, 27:44, 27:89
													38												-	*27:12
			28				32						38												B27, -	*27:14, 27:81
													38												B27, Null, -	*27:16, 27:39, 27:59N
													38												-	*27:18
			28				32						38												B27	*27:19
																									-	*27:23, 27:92
			28				32	33					38												B27	*27:24
			28				32						38									45	46		B27	*27:25
													38												B27	*27:29
			28				32				36		38												B27	*27:30
			28				32						38										47		-	*27:33
			28				32						38												-	*27:41
			28				32						38												-	*27:52, 27:67
			28				32				36		38												-	*27:62
			28				32				36		38									45	46		-	*27:75
											36		38												-	*27:77

Negative Control

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
																								ser. ⁵	HLA-B allele ⁴
																			44			47		Negative Control	<p>*35:01:01:01-35:01:09, 35:01:11-35:01:38, 35:07- 35:08:04, 35:08:06, 35:11:01-35:11:03, 35:15, 35:20:01-35:20:02, 35:23, 35:29:01-35:29:02, 35:40N-35:42:01, 35:48, 35:50, 35:52, 35:54, 35:57, 35:61:01-35:61:02, 35:77-35:78, 35:82, 35:90- 35:94, 35:100-35:101:02, 35:103-35:105, 35:107- 35:108:02, 35:110-35:112, 35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:158- 35:159, 35:161, 35:165N- 35:166, 35:168, 35:170- 35:171, 35:173N, 35:175- 35:178, 35:180, 35:187, 35:189, 35:192, 35:194, 35:196-35:197, 35:200, 35:202, 35:206, 35:208- 35:209, 35:212, 35:215- 35:216N, 35:219, 35:224- 35:225, 35:227-35:228</p>
																		44			47		B35	*35:01:10	
																			44			47		B35, Null, -	<p>*35:02:01-35:03:02, 35:03:04-35:04:01, 35:04:03, 35:06, 35:12:01- 35:12:03, 35:33-35:34, 35:36, 35:39, 35:55, 35:59, 35:65Q, 35:70, 35:74, 35:83-35:85, 35:95, 35:98, 35:106, 35:128- 35:129N, 35:136, 35:149- 35:150, 35:152-35:153, 35:155-35:156, 35:160, 35:162, 35:167, 35:172, 35:179, 35:181-35:184, 35:193, 35:195, 35:201, 35:204, 35:211, 35:214, 35:220-35:221, 35:223, 35:231</p>

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
																			44			47		B35	*35:03:03, 35:56	
																									Well No.	
																									Well No.	
																									ser. ⁵	HLA-B allele ⁴
																				44			47		B35	*35:04:02
																				44			47		B35, -	*35:05:01-35:05:03, 35:51, 35:58, 35:72, 35:89, 35:97, 35:114, 35:232
													40										47		-	*35:08:05
																				44			47		B35, -	*35:09:01-35:09:03, 35:18, 35:31, 35:75, 35:88, 35:127, 35:151, 35:205
													38										47		B35, -	*35:10, 35:28, 35:69, 35:80, 35:226
													38										47		B35	*35:13
																			44	45		47		B35	*35:14:01-35:14:02, 35:62	
													38										47		B35	*35:16
																				44			47		B35, -	*35:17, 35:30, 35:113, 35:125
												37											47		B35/blank	*35:19
						31														44			47		B35/B78, -	*35:21, 35:24:01- 35:24:02, 35:188, 35:190
																				44			47		B35, -	*35:22, 35:199
																							47		B35, -	*35:25, 35:49, 35:210
																							47		B35	*35:26
																				44			47		B35, -	*35:27, 35:203
																				44			47		B35	*35:35
				29																44			47		B35, -	*35:38, 35:169
															40					44			47		-	*35:42:02
																				44	45		47		B35, -	*35:43:01, 35:67, 35:79, 35:86, 35:102, 35:117, 35:135
																				44	45		47		-	*35:44
				29																44			47		B35	*35:45, 35:71
	26							33															47		B35	*35:46
												37											47		B35, -	*35:47, 35:217
															40					44			47		B35	*35:60
												37	38										47		-	*35:63
			28				32															w	47		B35/22	*35:76
						31														44			47		-	*35:81, 35:96, 35:109, 35:157, 35:233
	27																			44			47		-	*35:87
				29																44			47		-	*35:115
												37											47		-	*35:154

Negative Control

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
																				44	45	47		-	*35:163, 35:213
																				44		47		-	*35:164
															40					44	45	47		-	*35:185
																									Well No.
																									Well No.
																								ser. ⁵	HLA-B allele ⁴
			28				32																47	-	*35:186
																							47	-	*35:191
																							47	-	*35:198
	26												38										47	-	*35:207
																				44			47	-	*35:218
																				44				-	*35:222
			28				32													44			47	-	*35:229
				29									38										46	B37, Null, -	*37:01:01-37:01:07, 37:01:09, 37:03N, 37:06, 37:13, 37:15-37:21, 37:23-37:27, 37:30N-37:33N, 37:36
													38										46	B37, -	*37:01:08, 37:09
													38										46	B37/B27	*37:02
				29		31							38										46	B37, -	*37:04:01-37:04:02
				29									38										47	Blank	*37:05
													38										46	-	*37:07
				29															44				46	B37	*37:08
				29									38										46	B37, -	*37:10, 37:22, 37:29
				29									38										47	-	*37:11, 37:37
				29									38										46	B37	*37:12
				29									38										46 w	-	*37:14
				29									38										46	-	*37:28
				29							36		38										46	-	*37:34
				29									38										46	-	*37:35
																							46	B38, Null, -	*38:01:01-38:02:02, 38:02:04-38:02:05, 38:08-38:09, 38:11-38:16, 38:18-38:19, 38:21, 38:23-38:24, 38:27-38:29, 38:31-38:32, 38:34N, 38:38-38:43
																							46	-	*38:02:03
													38										46	B38	*38:03
													38										46	B38, -	*38:04, 38:25
																							46	B38, -	*38:05, 38:33
																				44			46	B38	*38:06
																				44			46	B38	*38:07
																							46	B38, -	*38:10, 38:36-38:37
																							46	-	*38:17
																							46	-	*38:20
																							46	-	*38:22

Negative Control

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
																									ser. ⁵	HLA-B allele ⁴
25												37	38										47		B60, Null, -	*40:01:01-40:01:06, 40:01:08-40:01:10, 40:01:12-40:01:17, 40:01:19-40:01:29, 40:07, 40:51, 40:66, 40:73, 40:79, 40:124:01, 40:139- 40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N, 40:172, 40:186
												37	38										47		B60, -	*40:01:07, 40:01:18, 40:72:01, 40:77
25												37	38	40									47		-	*40:01:11
26												37	38										47		B61, B61/B41, Null, -	*40:02:01-40:02:17, 40:04, 40:06:05, 40:39, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N, 40:214, 40:229
26												37	38										47		B61	*40:03, 40:20
26												37	38										47		B4005/B 50	*40:05
26												37	38										47		B61, -	*40:06:01:01-40:06:04, 40:06:06-40:06:07, 40:70
26												37											47		B61/blan k	*40:08
?												37	38										47		B61, -	*40:09, 40:11:01- 40:11:02, 40:18, 40:24, 40:27:01-40:27:02, 40:29, 40:35, 40:37, 40:50, 40:56 40:57, 40:64, 40:78, 40:82, 40:85, 40:90-40:91, 40:99, 40:104, 40:107, 40:111, 40:115, 40:119- 40:120, 40:133Q, 40:145, 40:169, 40:173, 40:176, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:211, 40:220, 40:224-40:226, 40:246
25												37	38										47		B60	*40:10:01
25	?											37	38										47		-	*40:10:02, 40:135
25												38											47		B60/B48	*40:12
26								34	36			38											46		-	*40:13

25	?													37	38				43									47	B60	*40:48	
	?													37	38													47	B61, B60	*40:53, 40:72:02	
	?													37	38	40												47	B40	*40:58	
25														37	38													47	B40, -	*40:59-40:60	
	?													37														47	-	*40:68, 40:232	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48							Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48							Well No.	
																														ser. ⁵	HLA-B allele ⁴
	?													37	38		41	43										47	B61	*40:71	
25	?													37	38													47	B60, -	*40:74, 40:149, 40:210	
25	?													38															B60	*40:76	
														37	38														-	*40:83, 40:177	
														38															-	*40:93	
	?										35	37	38																-	*40:95	
	?													38													46		-	*40:96	
	?													37	38													47	-	*40:98	
	?													37	38													47	-	*40:105	
	?										34	36	38														46	-	*40:109		
	?													38													46	-	*40:110		
25	?										34	36	38														46	-	*40:117		
25	?			29										37	38														-	*40:132, 40:178	
	?	27												37	38														-	*40:136	
25														37	38														-	*40:138	
	?													37	38														-	*40:148, 40:161	
	?													38													46	-	*40:157, 40:201		
	?													37	38													47	-	*40:158	
	?													37	38													47	-	*40:159	
25	?		28											37														47	-	*40:166	
25														37	38														-	*40:170	
	?													37	38														-	*40:174	
25	?													37	38														-	*40:183, 40:198	
	?													37	38														-	*40:184	
25														38															-	*40:185	
25	?													38															-	*40:208	
														38															-	*40:209	
25														37	38														-	*40:215	
25	?	27		29										37	38														-	*40:231	
		27			30									37	38														B41, -	*41:01, 41:06-41:07, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25, 41:28	
		27												37	38														B41, -	*41:02:01-41:03:02, 41:11, 41:13, 41:15, 41:19, 41:23, 41:27	
		27												37	38														B41, -	*41:04, 41:10, 41:18	
		27			30									?	38														B41	*41:05	
		27												37	38														B41	*41:08	

Negative Control

										35								44	46		-	*51:01:40, 51:54, 51:78:01-51:78:02				
					30	31				36									44	46		B51, -	*51:04, 51:46, 51:56:01- 51:56:02, 51:139			
					30					36									44	46		B51, -	*51:06:01-51:06:02, 51:59, 51:135			
					30					35	36								44	46		B51, Null, -	*51:08, 51:20, 51:44N, 51:141, 51:153			
					30					36									44	46		B51	*51:10			
										36							41		44	46		-	*51:15			
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
																									ser. ⁵	HLA-B allele ⁴
					30					35	36								44	46		B51		*51:24:02		
					30					35	36								44	46		B51		*51:36		
					30					35	36								44	46		B51, -		*51:37, 51:63		
				29	30						36								44	46		-		*51:42		
					30						36								44	46		-		*51:45		
					30					35	36								44	45	46		-	*51:61:01-51:61:02		
					30						36				41	43	44		44	46		-		*51:62		
					30						36								44	46		-		*51:64		
					30					35	36								44	46		-		*51:68		
					30						36								44	46		-		*51:81		
										35	36								44	46		-		*51:93, 51:144		
					30					35	36								44	46		-		*51:97		
					30					35	36								44	46		-		*51:101		
					30					35	36										46		Null, -	*51:104, 51:118N, 51:147		
					30					35	36				41			44	46				-	*51:106		
					30				34	35	36										46		-	*51:112		
										36									44	46		-		*51:148		
										35	36		38								46		B52, -	*52:01:01:01-52:01:01:02, 52:01:03, 52:01:06- 52:01:08, 52:01:11- 52:01:14, 52:01:16- 52:01:17, 52:04-52:05, 52:07-52:08, 52:10:01- 52:13, 52:15, 52:17-52:18, 52:22-52:24, 52:27-52:31		
					30					35	36		38								46		B52, -	*52:01:02, 52:01:04, 52:01:09, 52:02:01-52:03, 52:06:01-52:06:02, 52:09		
										35	36		38								46		B52, -	*52:01:05, 52:01:15, 52:14		
										35	36		38								46		-	*52:01:10		
										35	36		38								46		-	*52:16		
					30					35	36		38								46		-	*52:19		
										35			38								46		-	*52:20		

Negative Control

				30					35	36		38					45	46		-	*52:21					
									35	36		38		41				46		-	*52:25					
									35	36		38						46		-	*52:26					
												36					44	46		B53, -	*53:01:01-53:01:08, 53:05, 53:08:01-53:08:02, 53:10, 53:15-53:16, 53:18, 53:20-53:21, 53:23-53:27, 53:29					
					31					36							44	46		B53	*53:02					
																	44	46		B53/B35	*53:03, 53:09, 53:11- 53:13					
										36							44	46		B53, -	*53:04, 53:07, 53:19					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
																									ser. ⁵	HLA-B allele ⁴
					30	31					36							44	46		B53/B51	*53:06				
											36							44	46		-	*53:14				
											36	38							46		-	*53:17:01-53:17:02				
				29							36							44	46		-	*53:22				
						31					36	38							46		-	*53:28				
										34	36								46		-	*53:30, 57:45, 57:51				
																		44	46		-	*53:31				
			28				32							39	40						47			B54, Null, -	*54:01:01, 54:01:03- 54:02, 54:05N, 54:07- 54:08N, 54:10, 54:13, 54:16-54:19, 54:21-54:25, 54:27-54:29	
			28				32								40						47			B55, B54, -	*54:01:02, 55:01:07, 55:02:01-55:02:06, 55:07, 55:10, 55:12, 55:16, 55:19, 55:26, 55:30, 55:35, 55:37, 55:39, 55:41- 55:43, 55:47-55:48, 55:50, 55:57 ¹¹	
			28				32						39	41							47			B22	*54:03	
			28				32						39								47			B54, -	*54:04, 54:15	
			28				32						39						45		47			-	*54:06	
			28				32						39								47			-	*54:09, 54:14	
			28				32						39								47			B54	*54:11	
			28				32			34			39	40						46				-	*54:12	
			28				32			35			39	40							47			-	*54:20	
			28				32						39								47			-	*54:26	
													39	40							47			-	*54:30	

Negative Control

			28				32			35					41					47	-	*56:25, 56:37			
			28				32	w							41					47	-	*56:39			
																							B57, -	*57:01:01-57:01:04, 57:01:06-57:01:15, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:22-57:23, 57:25-57:27, 57:29-57:30, 57:33-57:38, 57:40-57:41, 57:43-57:44, 57:48-57:50, 57:52-57:56, 57:58, 57:60- 57:62, 57:64-57:65	
																							B57, -	*57:01:05, 57:11, 57:13, 57:21, 57:31, 57:47	
																							B57, Null, -	*57:02:01-57:03:02, 57:17, 57:28N, 57:32, 57:39, 57:42, 57:46, 57:57, 57:63	
																							B57	*57:04	
																							B57	*57:05	
																								Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	ser. ⁵	HLA-B allele⁴
				29							36						42					46		B57	*57:07
				29							36						42					46		B57	*57:09
																	42					47		B57	*57:12
						31					36						42					46		-	*57:14
				29							36						42					46		-	*57:24
25											36						42					46		-	*57:59
																								B58, Null, -	*58:01:01-58:01:02, 58:01:04-58:01:12, 58:04- 58:05, 58:10N-58:15, 58:19, 58:21-58:24, 58:29, 58:31N-58:33, 58:35, 58:37, 58:39N-58:42, 58:45
																								B58, -	*58:01:03, 58:02, 58:06, 58:16:01-58:16:02, 58:25- 58:26, 58:38, 58:43, 58:46
																								B58	*58:07
																								B17/B5	*58:08:01
																								-	*58:08:02
																								B17	*58:09
																								Null	*58:17N
																								-	*58:18
																								-	*58:20
																								-	*58:27

negative Control

										36					43			46	Z	-	*58:28, 58:34					
26										36					42			46		-	*58:36					
										36								46		-	*58:44					
										34					40			46		B59, -	*59:01:01:01-59:01:01:02, 59:05-59:06					
										34								46		B59, -	*59:02-59:03					
										34					41			46		-	*59:04					
	28																			B67, -	*67:01:01, 67:03					
	28																			B67, -	*67:01:02-67:02					
	28																	45	47	B73, -	*73:01-73:02					
										35					40			44	47	B78, -	*78:01:01-78:01:02, 78:02:02-78:03, 78:07					
										35								44	47	B78, B35	*78:02:01, 78:04					
										35	38								47	-	*78:05					
										35	38								47	-	*78:06					
	28			32	33															B81	*81:01					
	28				33															B81	*81:02					
	28				33															Null, -	*81:03-81:06					
	28				32										41					B82, -	*82:01-82:03					
	28	29			32								39							-	*83:01					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.	
																									ser. ⁵	HLA-B allele ⁴
																									-	A*23:31, A*24:106, C*07:231, C*16:10
																									-	A*24:174
																				45					-	A*26:68, A*68:56, C*02:56, C*06:20, C*12:50
																									-	C*01:30, C*12:87
26						31																			-	C*01:73
																		w							Cw2, -	C*02:06, C*02:47
																									-	C*02:23, C*04:77
																									Cw3, Cw10, -	C*03:05, C*03:25, C*03:27, C*03:143, C*03:167
																41									Cw3	C*03:19
																41									-	C*03:102
25																									-	C*03:129, C*17:16
																									-	C*03:179
																									-	C*06:72
																									-	C*07:02:30, C*08:16:02
																									Cw7	C*07:46
																									-	C*07:302
																									-	C*15:02:04
																40									-	C*15:25
				29																					-	C*15:39
						31																			-	C*15:51

Negative Control

