

HLA-B low screening 2018 1H8 R01

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48											
*15:154, 15:165							7					12					17	19																																									
*15:161								8				12	13	14					19																																								
*15:180		2										12																																															
*15:186								8				12	14						19		21																																						
*15:189						6	7					12	14						19														33																										
*15:194								7				12					17	19								26							33																										
*15:197:02			3	4				8				12	14						19																																								
*15:202, 15:239								7				12						18	19																																								
*15:222									9			12	14						19																																								
*15:224								7											19																																								
*15:230		1						7	9				14						19																																								
*15:233						6	7												19																																								
*15:236								7				12																																															
*15:241		1						7				12							19																																								
*15:243								8	9				14						19																																								
*15:253								8		11									19																																								
*15:255								8				12	14															28																															
*15:259							7	8		11	12								19																																								
*15:268							7	9				12	14						19																																								
*15:277							7	9				12							19																																								
*15:289							7					12						17	19																																								
*15:293								8				12	14						19																																								
*15:297		1						7				12						17	19																																								
*15:303								7											19																																								
*15:308								7				12						17	18	19																																							
*15:310								7				12							19																																								
*15:318								7				12							19																																								
*15:329			3	4				8					14						19																																								
*15:336				4								12							19																																								
*15:339								7											19																																								
*15:360, 15:383			3	4				7				12							19																																								
*15:376												12	14						18	19																																							
*15:377Q, 15:430								7				12							19																																								
*15:399				4				7				12						17	19																																								
*15:412								7				12							19							25																																	
*15:418												12	14					17	19																																								
*15:420								7					14					17	19																																								
*15:434, 15:436												12	14						19																																								
*15:449								7				12							19																																								

HLA-B low screening 2018 1H8 R01

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*35:84, 35:231	-																	17	18								26																		44		47					
*35:87	-								8									17	18									27																			44		47			
*35:114	-			3									12					17	18																												44		47			
*35:115	-												12					17	18											29																	44		47			
*35:154	-				4													17																			37											44		47		
*35:163, 35:213	-																	17	18																												44	45	47			
*35:164, 35:242	-			3														17	18																													44		47		
*35:185	-												12					17	18																													44	45	47		
*35:186	-												12					17	18										28				32																	44		47
*35:191	-												12					17	18																																47	
*35:198	-																	17																																	47	
*35:207	-				4								12					17									26																							47		
*35:218	-		2										12					17	18																														44		47	
*35:229	-												12					17	18											28				32															44		47	
*35:230	-																	17	18							25																								44		47
*35:251	-								9									17	18																															44		47
*35:256	-		2															17	18																															44		47
*35:260	-												12					17	18												29																			44		47
*35:265	-												12					17	18																		w												44	45	47	
*35:274	-																	17																					37											47		
*35:276	-																	17	18			21																											44		47	
*35:289	-												12		15			17	18																															44		47
*35:311	-																	17																																	47	
*35:329	-																	17	18																															44	46	47
*35:331	-												12					17	18							25																								44		47
*35:361	-											11						17																																	47	
*35:368	-			3	4								12					17	18											28				32															44		47	
*37:01:01-37:01:07, 37:01:09, 37:01:11, 37:01:13, 37:03N, 37:06:01-37:06:02, 37:13, 37:15-37:21, 37:23-37:27, 37:30N-37:33N, 37:36, 37:38, 37:41-37:43, 37:45-37:50, 37:52-37:55, 37:57-37:59, 37:61, 37:63-37:65, 37:68-37:71	B37, Null, -													13				17			20																														46	
*37:01:08, 37:01:10, 37:09, 37:44, 37:51	B37, -													13							20																													46		
*37:01:12, 37:10, 37:22	B37, -																	17			20																														46	
*37:02	B37, B27					5								13							20																													46		
*37:04:01-37:04:02	B37, -													13				17	18		20																														46	
*37:05	-													13				17	18		20																														47	
*37:07	-		1											13							20																														46	
*37:08	B37													13				17			20																												44	46		
*37:11, 37:37	-																	17	18		20																														47	
*37:12	B37								8					13				17	18		20																													46		
*37:14	-																	17	18		20																													46	w	
*37:28	-												11	13				17			20																														46	
*37:29	-																	17	18		20																														46	
*37:34	-																	17			20																		36											46		
*37:35, 37:56	-												12	13				17			20																													46		
*37:39-37:40	-													13				17			20																														46	

Negative Control

HLA-B low screening 2018 1H8 R01

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48								
*37:60	-													13				17			20								28	29		32														45	46										
*37:62	-		2											13				17			20									29																			46								
*37:66	-													13				17			20																												45	46							
*37:67	-																	17	18		20									29																			44			47					
*38:01:01:01-38:01:07, 38:01:09-38:02:02, 38:02:04-38:02:05, 38:02:07, 38:08-38:09, 38:11-38:16, 38:18, 38:21, 38:23-38:24, 38:27-38:29, 38:31-38:32, 38:34N, 38:38-38:44, 38:46-38:49, 38:51, 38:53, 38:55Q-38:62, 38:64, 38:66-38:69, 38:71	B38, Null, -					5																21	22	23																									46								
*38:01:08, 38:02:03	-																					21		23																										46							
*38:02:06, 38:10, 38:36-38:37, 38:45, 38:50, 38:65, 38:70	B38, -					5																		22	23																										46						
*38:03	B38					5						11										21	22																											46							
*38:04, 38:25	B38, -																					21	22	23																											46						
*38:05, 38:33	B38, -						6															21	22	23																											46						
*38:06	B38																					21																													44		46				
*38:07	B38					5																21																												44		46					
*38:17	-					5								13							20	21	22	23																												46					
*38:19	-				4	5																21	22	23																												46					
*38:20:01-38:20:02	-					5													17			21	22	23																												46					
*38:22	-					5											16						22	23																												46					
*38:26	-					5																21	22							28																						46					
*38:30	-					5			8														22	23																												46					
*38:35	-					5																21		23																													46				
*38:52	-					5							12										22	23																												46					
*38:54	-					5																21	22																														46				
*38:63	-					5																	22	23			25																										46				
*39:01:01:01-39:01:01:07, 39:01:03:01-39:01:08, 39:01:10-39:01:14, 39:01:16-39:01:18, 39:01:20-39:01:24, 39:05:01:01-39:06:02:04, 39:06:04, 39:09:01:01-39:09:03, 39:11, 39:15, 39:26-39:28, 39:31:01:01-39:31:01:02, 39:35, 39:38Q, 39:40:01N-39:41, 39:44, 39:46, 39:48, 39:51-39:57, 39:59-39:62, 39:64-39:68, 39:70-39:71, 39:73, 39:75, 39:77, 39:80-39:81, 39:84-39:87N, 39:90, 39:93-39:95N, 39:97N-39:98, 39:100, 39:103, 39:106, 39:108-39:109, 39:111, 39:113-39:119, 39:121-39:125, 39:127, 39:130, 39:132, 39:135	B39, B3901, Low B39, Null, -					5																21		23	24																											47					
*39:01:09, 39:06:03, 39:12, 39:45, 39:72, 39:78, 39:92, 39:133N	B39, B3901, Null, -																					21		23	24																													47			
*39:01:15	-					5																21		23	24																														47		
*39:02:01, 39:08	B39, B3902					5						11										21		24																													47				
*39:02:02:01-39:02:03, 39:13:01-39:13:02, 39:23, 39:49, 39:101, 39:105, 39:110	B39, B3902, -											11										21		24																														47			
*39:03:01:01-39:03:01:02, 39:14:01:01-39:14:01:02, 39:24:01, 39:29, 39:37, 39:79, 39:120	B39, -				4	5																21		23	24																														47		

Negative Control

HLA-B low screening 2018 1H8 R01

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*44:20, 44:100	B44, -				4				8																					29	30																	46				
*44:25, 44:50:01	B44				4													17												29																			46			
*44:31, 44:182	B44, -				4																							25																					46			
*44:37:01-44:37:02, 44:64:01-44:64:02, 44:268	B44, -				4		6											17																														46				
*44:40, 44:130, 44:210:01-44:210:02	B44, -											11					16	17												29																			46			
*44:44	B44										11					16	17													29																		46				
*44:46, 44:75, 44:129, 44:277	B12, -				4													17												29																			47			
*44:47, 44:65, 44:192:01-44:193, 44:197	B44, -				4																									29																			46			
*44:49	B44		2		4													17												29																		46				
*44:54	B44			3	4																									29																		46				
*44:57	-			w														17												29																		46				
*44:60, 44:110	-			3	4													17												29																		46				
*44:62	-			4	4				8	9								17												29																			46			
*44:90	-			4														17											28	29			32																47			
*44:95	-			4														17												29						34		36											46			
*44:97	-			4														17											28	29			32																46			
*44:103	-			4														17											26		29																		46			
*44:106	-			3	4																									29																			46			
*44:117, 44:251	-			4								11						17												29																			46			
*44:131	-			4														17												29																			47			
*44:139	-			4														17												29																			46			
*44:146	-			4														17												29																			45	46		
*44:150	-		1		4		6											17													31																		46			
*44:156	-		2		4						11							17												29																				46		
*44:166	-			4				8																																										46		
*44:169	-			4														17										25			29																			46		
*44:190	-			4		6												17												29			31																46			
*44:203	-			4														17												29			31																46			
*44:211	-			3	4													17												29																				46		
*44:212	-			4														17												26		29																		46		
*44:213	-			3	4				8																																								46			
*44:230	-			4									12	13																																			46			
*44:231	-			w														17												29																			46			
*44:236	-			4																																													46			
*44:275	-			4		6												17																															45	46		
*44:276	-			4									12					17																31																46		
*45:01:01-45:01:03, 45:05, 45:07, 45:11-45:15, 45:17-45:21, 45:23	B45, -			4					8		11																																						41			47
*45:02-45:03	B45			4					8		11																																							47		
*45:04	B21			4					8																																									47		
*45:06	B45			4					8		11																																							47		
*45:08	-			4					8																																									47		
*45:09	-			4					8				12	13																																				47		
*45:10	-			4					8		10																																							47		
*45:16	-			4					8		11																																							47		
*45:22	-			4					8		11																																								47	
																			18																																47	

Negative Control

