

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28
HLA-B allele ^{1,2,3}																												
*13:01:01-13:01:08, 13:51, 13:57, 13:60, 13:73	1	2	3																									
*13:02:01-13:02:16, 13:54, 13:65-13:66, 13:68	1	2		4																								
*13:03	1				5																							
*13:04	1					6					11																	
*13:06, 13:53	1						7																					
*13:07N	1	2	3					8																				
*13:08	1	2		4					9																			
*13:09, 13:67	1	2		4						10																		
*13:10	1	2									11																	
*13:11	1	2										12																
*13:12:01, 13:52 ⁴	1	2	3										13															
*13:12:02		2	3										13															
*13:13	1	2	3											14														
*13:14, 13:55	1	2		4											15													
*13:15, 13:33	1	2		4												16												
*13:16	1	2		4	5																				24			
*13:17	1	2	3														16											
*13:18	1	2		4				8																				
*13:19	1	2		4										13														
*13:20	1	2	3		5																					24		
*13:21	1	2	3							10				14														
*13:22:01	1		3		5										15													
*13:22:02	1		3		5																							
*13:23	1	2	3												15													
*13:25	1	2	3																			22						
*13:26:01-13:26:02	1	2	3								11																	
*13:27, 13:49N ⁴	1	2		4													17											
*13:28	1	2	3															18										
*13:29	1	2	3																			21						
*13:30	1	2		4																		21						
*13:31	1	2		4	5			8																		24		
*13:32	1	2		4														18										
*13:34, 13:38 ⁴	1	2		4															19									
*13:35, 13:59	1	2		4										14														
*13:36	1	2	3																	20								
*13:37	1	2		4																					23			
*13:39		2	3								10																	
*13:40	1	2		4																			21	22				
*13:41	1	2		4	5			8																				
*13:42	1	2		4						9															23			
*13:43	1	2	3															17										
*13:44	1	2		4														18							23			
*13:45	1			4	5																							
*13:46	1					6																						
*13:47	1	2		4																						26		
*13:48	1				5																			24		26		
*13:50	1	2	3																									
*13:56N	1	2		4		6																						
*13:58		2		4																								
*13:61	1	2	3																								27	
*13:62	1																								24			
*13:63N	1	2	3																									28
*13:64	1	2																									25	
*13:69	1	2		4																							27	
*13:70	1	2		4																						25		
*13:71	1	2								10				14														
*13:72	1	2		4							11																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28

*40:95, 40:148, 50:31				4						10																			24
*40:96				4						10				15															
*40:159				4		6				10	11																		
*40:161				4						10																			24
*41:24										10	11																		
*41:30						6	7			10	11																		
*44:10, 46:11, 54:03, 56:01:01-56:01:02, 56:01:04, 56:01:06-56:01:07, 56:07-56:08, 56:13-56:14, 56:16-56:17, 56:19N-56:20:01, 56:24, 56:26-56:30, 56:33, 56:35, 56:38N-56:42, 59:04, 82:01-82:03					5																								
*44:60	1																												
*45:02						6				10																			
*45:09						6				10	11																		
*45:14, 56:36					5					10																			
*47:03										w																			
*49:07, 51:16, 51:34, 52:08				4																									24
*50:09						6				10																			24
*50:33						6				10	11																		24
*51:05, 51:54												8																	24
*51:13:01-51:13:02, 51:92, 52:14, 55:58						6																							24
*51:29, 51:61:01-51:61:02, 52:21, 52:24												8																	24
*51:38										9																	23	24	24
*51:62					5							12							16										24
*51:93	2		4											14															24
*51:106, 52:25					5													16											24
*51:122														14															
*54:10, 55:37					5		7	8																					
*54:20, 55:21, 55:52					5		7	8																					24
*54:22					5		7							15															
*54:26, 55:22				4	5																								
*55:09				4	5			8																					24
*55:24				4	5																								24
*55:32						6	7			11																			
*56:01:03					5													15											
*56:02, 56:04					5						12																		
*56:20:02					5					10								15											
*56:25, 56:37					5																								24
*56:34					5													16											
*56:43					5			8																					
C*01:90					5		7			10																			
C*07:159													13																
C*17:08																					19								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	

Negative Control

29

29

Negative Control

Negative Control	29	29	
------------------	----	----	--

Negative Control	29	29
------------------	----	----

29 Negative Control