

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31		
<b>HLA-B allele<sup>1,2,3</sup></b>																																	
*13:01:01:01-13:01:10, 13:01:12-13:01:13, 13:51, 13:60, 13:73, 13:77-13:80, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109-13:110	1	2	3																														
*13:01:11	1	2	3						10																								
*13:02:01:01-13:02:22, 13:54, 13:65-13:66, 13:68, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:99-13:102, 13:104-13:105	1	2	4																														
*13:03	1			5																													
*13:04	1				6					11																							
*13:06, 13:53	1					7																											
*13:07N	1	2	3																												30		
*13:08	1	2	4						9																								
*13:09, 13:67	1	2	4							10																							
*13:10	1	2									11																						
*13:11	1	2										12																					
*13:12:01	1	2	3										13																				
*13:12:02	1	2	3										13																				
*13:13:01-13:13:02	1	2	3											14																			
*13:14	1	2	4												15																		
*13:15, 13:33 <sup>4</sup>	1	2	4													16																	
*13:16	1	2	4	5																				24									
*13:17	1	2	3													16																	
*13:18	1	2	4				8																										
*13:19	1	2	4										13																				
*13:20	1	2	3	5																					24								
*13:21	1	2	3							10				14																			
*13:22:01	1	3	5												15																		
*13:22:02	1	3	5																														
*13:23	1	2	3																												29		
*13:25	1	2	3																				22										
*13:26:01-13:26:02	1	2	3							11																							
*13:27, 13:49N	1	2	4													17																	
*13:28	1	2	3															18															
*13:29	1	2	3																			21											
*13:30	1	2	4																			21											
*13:31	1	2	4	5			8																	24									
*13:32	1	2	4															18															
*13:34, 13:38 <sup>4</sup>	1	2	4																19														
*13:35, 13:59	1	2	4											14																			
*13:36	1	2	3																			20											
*13:37	1	2	4																					23									
*13:39	1	2	3							10																							
*13:40	1	2	4																			21	22										
*13:41	1	2	4	5			8																										
*13:42	1	2	4						9															23									
*13:43	1	2	3															17															
*13:44	1	2	4															18						23									
*13:45	1		4	5																													
*13:46	1				6																												
*13:47	1	2	4																												26		
*13:48	1			5																					24								
*13:50	1	2	3																													26	
*13:52	1	2	3																	19												27	
*13:55	1	2	4																													29	
*13:56:01N-13:56:02N	1	2	4		6																												
*13:57	1	2	3										13		16																		
*13:58, 13:97-13:98	1	2	4																														
*13:61	1	2	3																													27	
*13:62	1																								24								
*13:63N	1	2	3																													28	
*13:64	1	2																								25							
*13:69	1	2	4																													27	
*13:70	1	2	4																														
*13:71	1	2								10				14																			
*13:72	1	2	4		6						11																						
*13:76N	1	2	3																			19											
*13:83	1	2	3												15																		
*13:86	1																																
*13:96	1	2	4																			19										27	
*13:103N	1	2	4																													26	
*13:108	1	2												14																			
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>		

Negative Control











