

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19
HLA-B allele^{1,2,3}																			
*14:01:01-14:01:05, 14:26, 14:40	1	2						8			11								
*14:02:01-14:02:12, 14:24-14:25, 14:30-14:31, 14:33, 14:35-14:36, 14:38-14:39	1		3								11								
*14:03	1		3	4							11								
*14:04	1		w		5						11								
*14:05			3			6					11								
*14:06:01-14:06:02			3				7				11								
*14:07N	1	2			5			8			11								
*14:08:01-14:08:02, 39:43 ⁴		2					7	8			11								
*14:09	1		3						9		11								
*14:10		2						8			11				15				
*14:11	1		3							10	11								
*14:12	1	2						8			11	12							
*14:13			3								11		13						
*14:14	1	2						8			11			14					
*14:15	1		3								11				15				
*14:16, 14:23 ⁵	1		3								11					16			
*14:17	1		3			6					11								
*14:18	1		3								11			14					
*14:19	1	2						8				12							
*14:20			3	4							11								
*14:21	1										11								
*14:22	1		3								11							17	
*14:27	1		3									12							
*14:28	1									10	11								
*14:29	1		3			6				10	11								
*14:32	1	2						8	9		11					16			
*14:34	1		3								11								18
*14:37			3								11								17
*07:02:01-07:02:15, 07:02:17-07:02:31, 07:02:33-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 15:71, 15:175, 27:08, 27:12, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 35:50, 35:84, 35:197, 35:231, 40:02:01-40:02:14, 40:02:16-40:06:11, 40:08-40:09, 40:11:01-40:11:02, 40:18, 40:20, 40:24, 40:26-40:27:02, 40:29, 40:35:01-40:35:02, 40:37, 40:40, 40:44, 40:50, 40:56, 40:64, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:243-40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:295, 44:131, 48:02:02, 56:35, 73:01-73:02, 81:01-81:07, C*03:137, C*03:182, C*07:351, C*12:82								8											
*07:02:32, 39:33, 39:47, 39:50, 39:74, 39:80		2						8											
*07:16, 07:37, 07:50, 07:75, 07:91, 07:207, 15:68, 18:10, 18:21, 18:35, 18:56, 18:58, 18:101, 35:162, 40:149, 48:01:01-48:02:01, 48:02:03-48:06, 48:08-48:17, 48:19-48:21, 48:23-48:36								8						14					
*07:19, 07:31, 07:34, 07:43, 08:94, 37:37, 39:01:09, 39:02:02, 39:06:03, 39:10:01-39:10:02, 39:16-39:17, 39:19:01, 39:23, 39:39:02, 39:45, 39:78, 39:83, 39:89, 39:92, 39:96, 40:39, 40:184, 42:01:01-42:02, 42:04-42:05:02, 42:07-42:10, 42:12-42:19, 54:02, 55:10, 67:01:02-67:02, 67:05								8			11								
*07:27, 07:38, 07:180, 07:219, 15:03:01-15:03:05, 15:29, 15:47:01-15:47:02, 15:49, 15:54, 15:62, 15:64:01-15:64:02, 15:74, 15:91, 15:98, 15:103, 15:123, 15:127, 15:131-15:132, 15:151, 15:156, 15:158, 15:173, 15:210, 15:220, 15:235, 15:242, 15:251, 15:253, 15:266, 15:274, 15:281-15:282, 35:25, 35:124-35:125, 35:142, 37:01:05, 37:02, 37:07, 37:26, 37:39, 37:48, 40:12, 44:130, 44:156, 48:18, 48:22, 49:25, 57:58, 58:12														14					
*07:115						6		8											
*07:186, C*07:102								w											
*07:218, 07:220				4				8											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19

Negative Control

