

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
HLA-B allele^{1,2,3}																						
*14:01:01-14:01:06, 14:26, 14:40, 14:47, 14:49	1	2						8			11											
*14:02:01:01-14:02:13, 14:24-14:25, 14:30-14:31, 14:35-14:36, 14:38-14:39, 14:43, 14:45, 14:48, 14:50	1		3								11											
*14:03	1		3	4							11											
*14:04	1		w		5						11											
*14:05			3			6					11											
*14:06:01-14:06:02			3				7				11											
*14:07N	1	2			5			8			11											
*14:08:01-14:08:02, 39:43 ⁴		2					7	8			11											
*14:09	1		3						9		11											
*14:10		2						8			11				15							
*14:11	1		3							10	11											
*14:12	1	2						8			11	12										
*14:13			3								11		13									
*14:14	1	2						8			11			14								
*14:15	1		3								11				15							
*14:16, 14:23 ⁵	1		3								11					16						
*14:17	1		3			6					11											
*14:18	1		3								11			14								
*14:19	1	2						8				12										
*14:20			3	4							11											
*14:21, 14:42	1										11											
*14:22	1		3								11						17					
*14:27	1		3									12										
*14:28	1									10	11											
*14:29	1		3			6				10	11											
*14:32	1	2						8	9		11					16						
*14:33	1		3								11									20		
*14:34	1		3								11							18				
*14:37			3								11						17					
*14:41N	1		3								11								19			
*14:44	1		3																			
*14:46		2						8														21
*07:02:01-07:02:15, 07:02:17-07:02:31, 07:02:33-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 07:229-07:235, 07:237-07:238, 07:240-07:243, 07:245, 07:247-07:256, 15:71, 15:175, 27:08, 27:12, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 35:03:21, 35:50, 35:84, 35:197, 35:231, 35:269, 40:02:01-40:02:14, 40:02:16-40:06:14, 40:08-40:09, 40:11:01-40:11:02, 40:18, 40:20, 40:24, 40:26-40:27:02, 40:29, 40:35:01-40:35:02, 40:37, 40:40, 40:44, 40:50, 40:56, 40:64, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:297, 40:302-40:307, 40:309, 40:311, 40:313, 40:317, 44:131, 48:02:02, 50:43, 56:35, 73:01-73:02, 81:01-81:07, C*03:137, C*03:182, C*07:351, C*12:82								8														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*07:02:32, 39:33, 39:47, 39:50, 39:74, 39:80, 39:102		2						8														
*07:16, 07:37, 07:50, 07:75, 07:91, 07:207, 07:228, 15:68, 18:10, 18:21, 18:35, 18:56, 18:58, 18:101, 35:162, 35:267, 40:149, 48:01:01-48:02:01, 48:02:03-48:06, 48:08-48:17, 48:19-48:21, 48:23-48:36, 81:08								8						14								

Negative Control

*18:01:01:01-18:01:01:02, 18:01:03-18:01:12, 18:01:14-18:01:15, 18:01:17-18:01:22, 18:05-18:08, 18:12:01-18:12:02, 18:15, 18:17N-18:20, 18:23N-18:24, 18:28, 18:31-18:34, 18:37-18:38, 18:40-18:43, 18:45-18:49, 18:51, 18:53, 18:55, 18:59-18:60, 18:62-18:66, 18:71-18:79, 18:81-18:83, 18:85, 18:87-18:93, 18:96, 18:99-18:100, 18:103-18:104, 18:106, 18:108-18:109, 18:111-18:116							7	8				11			14										
*18:01:02, 18:01:23, 18:03, 18:29, 18:80, 18:94N, 18:98, 18:102							7	8				11													
*18:01:13, 18:09, 18:25, 18:54, 18:67-18:68, 18:70, 18:84, 18:95							7					11			14										
*18:01:16, 18:02, 18:11, 18:14, 18:27, 18:39, 18:110, 37:05, 37:11, 39:13:01-39:13:02, 39:20, 39:49, 42:11, 48:37								8				11			14										
*18:13, 18:30			4				7	8				11			14										
*18:22, 18:105								8				11			14	15		17							
*18:26, 18:61, 18:97							7	8				11			13	14									
*18:36, 18:50, 18:57, 18:86							7	8							14										
*18:44:01-18:44:02	1							8				11			14										
*18:52							7	8				11			14							17			
*18:69								8				11			14	15									
*27:07:04		2																							
*35:03:17, 49:26											10														
*35:16-35:17:02, 35:32:01, 35:61:02, 35:98, 44:140, 46:01:07, 50:04, 58:59, C*03:102, C*03:263																								20	
*35:21, 35:24:01-35:24:02, 35:81, 35:109, 35:287, 53:02, 53:06, 53:28, 58:09, C*15:39																15		17							
*35:96, 35:157, 35:188, 35:190												11				15		17							
*35:125															14								20		
*35:233, 51:04, 51:46, 51:139															13	15									
*37:04:01-37:04:02												11			14	15									
*38:01:01-38:01:07, 38:01:09-38:02:02, 38:02:04-38:02:07, 38:07, 38:09-38:12, 38:15, 38:17-38:18, 38:20-38:21, 38:23-38:24, 38:27, 38:29, 38:31-38:32, 38:34N-38:40, 38:42-38:45, 38:47-38:58		2										11			14										
*38:03, 38:13-38:14, 38:16, 38:26, 38:28, 38:41, 38:46, 39:01:15, 39:104		2										11													
*38:08, 38:22, 38:30		2													14										
*38:19		2				6						11			14										
*38:33			3								10	11			14										
*39:01:01:01-39:01:01:04, 39:01:03-39:01:08, 39:01:10-39:01:14, 39:01:16-39:02:01, 39:06:01-39:06:02, 39:06:04-39:06:05, 39:09:01-39:09:03, 39:14-39:15, 39:18, 39:19:02, 39:22, 39:25N-39:31, 39:34-39:36, 39:38Q-39:39:01, 39:41-39:42, 39:44, 39:46, 39:51-39:54, 39:57-39:62, 39:64-39:71, 39:73, 39:75, 39:77, 39:79, 39:81-39:82, 39:84-39:88, 39:90-39:91, 39:93-39:95N, 39:97N-39:98, 39:100, 39:103, 39:106-39:107, 67:01:01, 67:03-67:04		2						8				11													
*39:03, 39:24:01-39:24:02		2				6		8				11													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22			
*39:04			3								10	11													
*39:05:01-39:05:02, 39:07-39:08, 39:11, 39:55-39:56		2						8				11			14										
*39:32		2					7	8							13										
*39:37		2				6		8				11			14										
*39:40:01N-39:40:02N		2			5			8				11													
*39:48		2					7	8				11			13										
*39:99								w				11													
*40:28								8							13	15		17							
*40:51, 40:163							7					11			13										
*40:113, 51:06:01-51:06:03, 51:62, 51:194							7								13										
*40:174								8							13										
*40:243								8															21		
*42:06, 42:20						6		8				11													
*44:190												11				15									
*44:203, 57:14:01-57:14:02																15									
*46:48																						18			
*48:07						6		8							14										
*49:09, A*33:40																						17			
*51:22, 78:03			3												13										
*51:29, 51:82, 52:24				4											13										
*51:56:01-51:56:03															13	15		17							
*51:116, 51:176, 51:179, 52:16, 52:27															13	14									

Negative Control

Negative Control

*57:11, 58:02, 58:06-58:07, 58:25, 58:38, 58:43, 58:46, 58:60, C*01:35, C*01:107, C*03:17, C*03:71, C*03:249, C*03:271, C*03:302, C*06:96, C*12:122, C*14:20												12										
C*01:10, C*02:05:01-02:05:03, C*02:17, C*06:08, C*12:119, C*14:25, C*16:29, C*16:50, C*17:21																				21		
C*01:30	1			4															17			
C*01:101, C*01:113, C*15:55																			16			
C*05:16, C*05:85, C*05:107, C*06:129, C*07:364, C*08:12								9										16				
C*08:08:01-08:08:02						6																
C*08:51, C*08:114				4			7												17			
C*12:87	1																		17			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22