

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	
HLA-B allele^{1,2,3}																							
*14:01:01:01-14:01:06, 14:26, 14:40, 14:47, 14:49, 14:54	1	2						8			11												
*14:02:01:01-14:02:15, 14:24-14:25, 14:30-14:31, 14:35-14:36, 14:38-14:39, 14:43, 14:45, 14:48, 14:50-14:52, 14:58-14:61	1		3								11												
*14:03	1		3	4							11												
*14:04	1		w		5						11												
*14:05			3			6					11												
*14:06:01-14:06:02, 14:56			3				7				11												
*14:07N	1	2			5			8			11												
*14:08:01-14:08:02, 39:43 ⁴	1	2					7	8			11												
*14:09	1		3						9		11												
*14:10		2						8			11				15								
*14:11	1		3							10	11												
*14:12	1	2						8			11	12											
*14:13			3								11		13										
*14:14	1	2						8			11			14									
*14:15	1		3								11				15								
*14:16, 14:23 ⁵	1		3								11					16							
*14:17	1		3			6					11												
*14:18	1		3								11			14									
*14:19	1	2						8				12											
*14:20			3	4							11												
*14:21, 14:42, 14:57	1										11												
*14:22	1		3								11							17					
*14:27	1		3									12											
*14:28	1									10	11												
*14:29	1		3			6				10	11												
*14:32	1	2						8	9		11				16								
*14:33	1		3								11										20		
*14:34	1		3								11								18				
*14:37			3								11						17						
*14:41N	1		3								11								19				
*14:44	1		3																				
*14:46		2						8															21
*14:53		2						8			11		13										
*14:55		2					7	8			11						17						
*07:02:01:01-07:02:14, 07:02:17-07:02:31, 07:02:33-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44N-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 07:229-07:235, 07:237-07:238, 07:240-07:243, 07:245, 07:247-07:272N, 07:274-07:296, 07:298-07:305, 07:307-07:311, 15:71, 15:175, 15:381, 27:08, 27:12:01:01-27:12:01:02, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 35:03:21, 35:50, 35:84, 35:197, 35:231, 35:269, 40:02:01:01-40:02:14, 40:02:16-40:06:14, 40:06:16, 40:08-40:09, 40:11:01-40:11:02, 40:18, 40:20:01:01-40:20:01:02, 40:24, 40:26-40:27:02, 40:29, 40:35:01-40:35:02, 40:37, 40:40, 40:44, 40:50, 40:56, 40:64:01:01-40:64:01:02, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:297, 40:302-40:307, 40:309, 40:311, 40:313, 40:317, 40:320, 40:322, 40:330-40:331, 40:334, 40:337N, 40:341, 40:343, 40:345N, 40:347-40:351, 40:354, 40:356, 40:359, 40:361N-40:364, 40:369, 44:131, 48:02:02, 50:43, 56:35, 73:01-73:02, 81:01-81:07, C*03:137, C*03:182, C*07:351, C*12:82, C*16:116								8															
*07:02:15, 07:19, 07:31, 07:34, 07:43, 08:94, 37:37, 39:01:09, 39:02:02:01-39:02:03, 39:06:03, 39:10:01-39:10:02, 39:16-39:17, 39:19:01, 39:23, 39:39:02, 39:45, 39:78, 39:83, 39:89, 39:92, 39:96, 39:101, 39:105, 39:110, 39:126, 39:131, 39:133N, 40:39, 40:184, 42:01:01-42:02:01:02, 42:04-42:05:02, 42:07-42:10, 42:12-42:19, 42:21, 42:23-42:25, 54:02, 55:10, 55:82, 67:01:02-67:02, 67:05								8			11												
*07:02:32, 39:33, 39:47, 39:50, 39:74, 39:80, 39:102		2						8															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	
*07:16, 07:37:01-07:37:02, 07:50, 07:75:01:01-07:75:01:02, 07:91, 07:207, 07:228:01-07:228:02, 07:306, 15:68, 18:10, 18:21, 18:35, 18:56, 18:58, 18:101, 35:162, 35:267, 40:149, 48:01:01:01-48:02:01, 48:02:03-48:06, 48:08-48:17, 48:19-48:21, 48:23-48:36, 48:38-48:46, 81:08								8						14									

Negative Control

*18:01:01:01-18:01:01:09, 18:01:03-18:01:12, 18:01:14-18:01:15, 18:01:17-18:01:22, 18:01:24-18:01:25, 18:05:01:01-18:08, 18:12:01-18:12:02, 18:15, 18:17N-18:20, 18:23N-18:24, 18:28, 18:31-18:34, 18:37:01-18:38, 18:40-18:43, 18:45-18:49, 18:51, 18:53, 18:55, 18:59-18:60, 18:62-18:66, 18:71-18:79, 18:81-18:83, 18:85, 18:87-18:93, 18:96, 18:99-18:100, 18:103-18:104, 18:106, 18:108-18:109, 18:111-18:124, 18:126-18:131:01:02, 18:133-18:135, 18:137-18:141, 18:143-18:145								7	8		11		14									
*18:01:02, 18:01:23, 18:03:01-18:03:02, 18:29, 18:80, 18:94N, 18:98, 18:102, 18:132, 18:142								7	8		11											
*18:01:13, 18:09, 18:25, 18:54, 18:67-18:68, 18:70, 18:84, 18:95, 18:136								7			11		14									
*18:01:16, 18:02, 18:11, 18:14, 18:27, 18:39, 18:110, 37:05, 37:11, 37:67, 39:13:01-39:13:02, 39:20, 39:49, 42:11, 48:37									8		11		14									
*18:13, 18:30				4				7	8		11		14									
*18:22, 18:105									8		11		14	15	17							
*18:26, 18:61, 18:97								7	8		11		13	14								
*18:36, 18:50, 18:57, 18:86								7	8				14									
*18:44:01-18:44:02	1								8		11		14									
*18:52								7	8		11		14				17					
*18:69									8		11		14	15								
*27:07:04	2																					
*27:153, 40:174									8				13									
*35:03:17, 49:26										10												
*35:12:04, 35:16-35:17:02, 35:32:01, 35:61:02, 35:98, 35:346, 44:140, 46:01:07, 50:04, 58:59:01, C*03:102, C*03:263:01-03:263:02																			20			
*35:21, 35:24:01-35:24:02, 35:81, 35:109, 35:287, 53:02, 53:06, 53:28, 58:09, C*15:39, C*15:130														15	17							
*35:96, 35:157, 35:188, 35:190, 58:76											11		15	17								
*35:125													14						20			
*35:233, 51:04, 51:46, 51:139													13	15								
*37:04:01-37:04:02											11		14	15								
*38:01:01:01-38:01:07, 38:01:09-38:02:02, 38:02:04-38:02:07, 38:07, 38:09-38:12, 38:15, 38:17-38:18, 38:20-38:21, 38:23-38:24, 38:27, 38:29, 38:31-38:32, 38:34N-38:40, 38:42-38:45, 38:47-38:62, 38:64, 38:66-38:70	2										11		14									
*38:03, 38:13-38:14, 38:16, 38:26, 38:28, 38:41, 38:46, 39:01:15, 39:104	2										11											
*38:08, 38:22, 38:30, 38:63, 38:65	2												14									
*38:19	2					6					11		14									
*38:33			3								10	11	14									
*39:01:01:01-39:01:01:07, 39:01:03:01-39:01:08, 39:01:10-39:01:14, 39:01:16-39:02:01, 39:06:01-39:06:02:04, 39:06:04-39:06:05, 39:09:01:01-39:09:03, 39:14:01:01-39:15, 39:18, 39:19:02, 39:22, 39:25N-39:31:01:02, 39:34-39:36, 39:38Q-39:39:01, 39:41-39:42, 39:44, 39:46, 39:51-39:54, 39:57-39:62, 39:64-39:71, 39:73, 39:75, 39:77, 39:79, 39:81-39:82, 39:84-39:88, 39:90-39:91, 39:93-39:95N, 39:97N-39:98, 39:100, 39:103, 39:106-39:109, 39:111-39:112, 39:114-39:117, 39:119, 39:121-39:125, 39:127-39:128, 39:130, 39:132, 39:134, 67:01:01, 67:03-67:04, 67:07	2						8				11											
*39:03:01:01-39:03:01:02, 39:24:01-39:24:02, 39:120	2					6	8				11											
*39:04				3							10	11										
*39:05:01:01-39:05:03, 39:07-39:08, 39:11, 39:55-39:56, 39:113, 39:118	2							8			11		14									
*39:32	2						7	8					13									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*39:37		2				6	8				11		14									
*39:40:01N-39:40:02N		2			5		8				11											
*39:48, 67:06		2					7	8			11		13									
*40:28								8					13	15	17							
*40:51, 40:163							7				11		13									
*40:113, 51:06:01-51:06:04, 51:62, 51:194							7						13									
*40:243								8														21
*42:06, 42:20						6	8				11											
*44:190, 55:77											11			15								
*44:203, 44:275, 51:227, 57:14:01-57:14:02, C*03:384														15								
*46:48																						18
*48:07							6	8					14									
*49:09, A*33:40																						17
*50:51, 57:11, 58:02:01-58:02:02, 58:06-58:07, 58:25, 58:38, 58:43, 58:46, 58:60, C*01:35, C*01:107, C*03:17:01-03:17:02, C*03:71, C*03:249, C*03:271, C*03:302, C*06:96, C*06:197, C*12:122, C*14:20												12										
*51:22, 78:03			3										13									
*51:29, 51:82, 51:217, 52:24				4									13									
*51:56:01-51:56:03													13	15	17							
*51:116, 51:176, 51:179, 51:218, 52:16, 52:27													13	14								
C*01:10, C*02:05:01-02:05:03, C*02:17, C*06:08, C*12:119, C*14:25, C*16:29, C*16:50, C*17:21																						21
C*01:30, C*14:76	1			4																	17	
C*01:101, C*01:113, C*15:55																						16
C*05:16, C*05:85, C*05:107, C*06:129, C*07:364, C*08:12									9													16

Negative Control

Negative Control

C*06:207, C*12:87	1																	17					
C*07:516, C*07:521							7											17					
C*08:08:01-08:08:02						6																	
C*08:51, C*08:114				4			7											17					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	