

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
HLA-B Allele ^{1,2,3}																							
*14:01:01:01-14:01:13, 14:26, 14:40, 14:47, 14:49, 14:54, 14:65, 14:77-14:78, 14:80-14:83, 14:86-14:87, 14:97, 14:103-14:105, 14:108Q, 14:113N	1	2						8			11											22	
*14:02:01:01-14:02:08, 14:02:10-14:02:16, 14:02:18-14:02:30, 14:24-14:25, 14:30-14:31, 14:35-14:36, 14:38-14:39, 14:43, 14:45, 14:48, 14:50-14:52, 14:58-14:61, 14:63-14:64, 14:66-14:70Q, 14:72N, 14:74-14:76N, 14:79N, 14:84-14:85N, 14:88-14:94, 14:98-14:101N, 14:106-14:107, 14:109-14:110	1	3									11											22	
*14:02:09	1	3									11												
*14:02:17, 14:33	1	3									11									20		22	
*14:03	1	3	4								11											22	
*14:04	1	w			5						11											22	
*14:05	1					6					11											22	
*14:06:01-14:06:02, 14:56	1	3					7				11											22	
*14:07N	1	2			5			8			11											22	
*14:08:01-14:08:02, 39:43 ⁵	1	2					7	8			11											22	
*14:09	1	3							9		11											22	
*14:10	1	2						8			11				15							22	
*14:11	1	3								10	11		12									22	
*14:12	1	2						8			11		12									22	
*14:13	1	3									11			13								22	
*14:14	1	2	3					8			11				14							22	
*14:15	1	3									11				15							22	
*14:16, 14:23 ⁴	1	3									11					16						22	
*14:17	1	3			6						11											22	
*14:18	1	3									11			14								22	
*14:19	1	2						8			12											22	
*14:20	1	3	4								11											22	
*14:21, 14:42:01-14:42:02, 14:57	1										11											22	
*14:22	1	3									11						17					22	
*14:27	1	3									12											22	
*14:28	1									10	11											22	
*14:29	1	3			6					10	11											22	
*14:32	1	2						8	9		11					16						22	
*14:34	1	3									11						18					22	
*14:37	1	3									11						17					22	
*14:41N	1	3									11							19				22	
*14:44	1	3									11											22	
*14:46	2					8															21	22	
*14:53	2					8				11			13										
*14:55	2				7	8										17							
*14:62	2	3									11											22	
*14:71	1	2				8					11											22	
*14:73	1										11			14								22	
*14:95, 39:79 ⁵	2					8					11											22	
*14:96	2					7				11		13				15		17					
*14:102	1	3				8				11											21	22	
*14:111	1	2	4				8			11												22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
*07:02:01:01-07:02:05, 07:02:07-07:02:14, 07:02:17-07:02:31, 07:02:33-07:02:77, 07:02:79-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44N-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85:01, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 07:229:01-07:235, 07:237-07:238, 07:240-07:243, 07:245, 07:247-07:272N, 07:274-07:296, 07:298-07:305, 07:307-07:312, 07:314-07:326, 07:328-07:351N, 07:353-07:362, 07:364-07:414, 07:416-07:452, 07:454-07:469N, 15:71, 15:75, 15:381, 27:08, 27:12:01:01-27:12:01:03, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:49, 27:204:01:01-27:204:01:02, 27:242, 35:03:21, 35:08:12, 35:50, 35:84, 35:197, 35:231, 35:269, 35:411, 35:423, 40:02:01:01-40:02:14, 40:02:16-40:06:14, 40:06:16-40:06:22, 40:06:24-40:06:30, 40:08:01:01-40:09, 40:11:01:01-40:11:02, 40:18, 40:20:01:01-40:20:01:02, 40:24, 40:26-40:27:03, 40:29, 40:35:01-40:35:03, 40:37, 40:40, 40:44, 40:50:01:01-40:50:01:02, 40:56, 40:64:01:01-40:64:02, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:170, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:297, 40:302-40:307, 40:309, 40:311, 40:313, 40:317, 40:320, 40:322, 40:330-40:331, 40:334:01-40:334:02, 40:337N, 40:341, 40:343, 40:345N, 40:347-40:351, 40:354, 40:356, 40:359, 40:361N-40:364, 40:366, 40:369, 40:371, 40:373-40:376, 40:378, 40:384, 40:387-40:390, 40:399N, 40:405, 40:408-40:415, 40:425, 40:427, 40:429, 40:434-40:435, 40:437-40:438N, 40:440-40:442, 40:444-40:445, 40:451, 40:453-40:455, 40:458-40:460, 40:464-40:465, 40:467-40:468, 40:471-40:473, 40:476, 40:478, 40:481N-40:482, 40:484, 40:487N-40:489, 40:492-40:494, 40:500, 40:505, 40:511N, 40:514N-40:515, 40:518-40:522, 40:526, 44:31, 48:01:10, 48:02:02, 48:48, 50:43, 56:35, 56:84, 73:01:01:01-73:03, 81:01:01:01-81:07, 81:09-81:10, A*68:233, C*03:03:51, C*03:137, C*03:182, C*03:459, C*03:504, C*07:351, C*08:02:22, C*12:82, C*16:116																							
*07:02:15, 07:19, 07:31:01-07:31:02, 07:34, 07:43, 08:94, 08:226, 37:37, 39:01:09, 39:01:25, 39:01:30, 39:02:01-07:31:02, 39:06:03, 39:10:01-39:10:02, 39:16-39:17, 39:19:01, 39:23, 39:39:02, 39:45, 39:78, 39:83:01, 39:83:03, 39:89, 39:92, 39:96, 39:101, 39:105, 39:110, 39:126, 39:131, 39:133N, 39:143, 39:145, 39:162, 39:164, 39:175N, 39:180, 40:39, 40:184, 42:01:01:01-42:01:02, 42:04-42:05:02, 42:07-42:10, 42:12-42:19, 42:21, 42:23-42:32, 54:02, 55:10, 55:82, 67:01:07-67:02:01:02, 67:05																							
*07:02:32, 07:85:02, 39:33, 39:47, 39:50, 39:74, 39:80, 39:102	2							8															
*07:16, 07:37:01-07:37:02, 07:50, 07:75:01:01-07:75:01:02, 07:91, 07:207, 07:228:01-07:228:02, 07:306, 15:68, 15:556, 15:571, 15:582, 18:10, 18:21, 18:35, 18:56, 18:58, 19:101, 18:164, 18:165, 18:189, 35:162, 35:267, 35:494, 40:149, 48:01:01:01-48:01:01:09, 48:01:11-48:02:01, 48:02:03-48:06, 48:08-48:17, 48:19-48:21, 48:23-48:36, 48:38-48:47, 48:49-48:55, 81:08													14										
*07:27, 07:38, 07:180, 07:219, 07:453, 08:255, 15:03:01:01-15:03:13, 15:29:01:01-15:29:01:02, 15:47:01-15:47:02, 15:49, 15:54, 15:62, 15:64:01-15:64:02, 15:74, 15:91, 15:98, 15:103, 15:123, 15:127, 15:131-15:132, 15:151, 15:156, 15:158, 15:173, 15:210, 15:220:01:01-15:220:04, 15:235, 15:242:01-15:242:02, 15:251:01-15:251:03, 15:253, 15:266, 15:274, 15:281-15:282, 15:369, 15:379, 15:389, 15:426, 15:429, 15:444, 15:447-15:448, 15:485, 15:487, 15:502, 15:509, 15:517, 15:527, 15:539, 15:545, 15:560, 15:588, 15:614:01:01-15:614:01:02, 15:616Q-15:617, 15:619, 15:628, 35:25, 35:124, 35:142, 35:320, 35:487, 35:557, 37:01:05, 37:02, 37:07, 37:26, 37:39, 37:48, 37:94, 37:97, 40:12:01-40:12:02, 44:130, 44:156, 44:221, 44:224, 44:258, 48:18, 48:22, 49:25, 50:77, 53:43:01-53:43:02, 57:58, 57:80, 58:12, C*06:147													14										
Negative Control																							

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
*07:115, 07:246, 07:327, 40:314						6		8															
*07:186, 07:313, C*07:102								w															
*07:218, 07:220				4				8															
*07:239								8								16					21		
*08:01:01:01-08:01:23, 08:01:25-08:02, 08:04:01-08:05, 08:07-08:10, 08:12:01-08:20:02, 08:22, 08:24, 08:26:01-08:39, 08:41-08:53:02, 08:55-08:58, 08:70-08:78, 08:80-08:93, 08:95-08:109, 08:111-08:117, 08:119-08:124, 08:126-08:132, 08:134-08:138, 08:140-08:155, 08:157-08:164, 08:166-08:170, 08:172-08:174, 08:176-08:185, 08:187-08:189:02, 08:191, 08:193-08:199, 08:201-08:204, 08:206-08:209, 08:211-08:225, 08:227-08:252N, 08:254, 08:256-08:270Q, 08:272-08:294, 08:296-08:303, 13:06, 13:46, 13:53, 13:131, 15:42, 15:50, 15:83:01:01-15:83:01:02, 15:86, 15:121, 15:188, 15:199, 15:218Q, 15:224, 15:265, 15:348, 15:449, 15:485, 15:494, 15:497, 15:566, 15:592, 27:15, 27:28, 27:62, 27:71, 27:249, 35:35, 35:60, 35:87, 35:93, 35:213, 35:262, 35:276-35:277, 35:343, 35:350, 35:468, 35:471, 37:01:12, 37:22:01-37:22:03, 37:60, 37:95, 39:12:01:01-39:12:01:02, 39:63, 39:72, 39:181, 40:01:69, 40:199, 40:424, 41:01:01:01-41:16, 41:20-41:21, 41:23-41:61, 41:63-41:79, 42:22, 44:03:28, 44:04, 44:56N, 44:132, 44:163:01-44:163:02, 44:166, 44:268, 44:446, 45:08, 46:03, 46:18, 46:21:01-46:21:02, 46:29, 46:83, 47:08, 49:76, 50:45, 50:49, 51:36, 51:143, 51:157, 51:197, 51:295, 51:300, 51:376, 53:69, 54:01:01:01-54:01:10, 54:04-54:05N, 54:07-54:25, 54:27-54:32, 54:34-54:37, 54:39, 54:41-54:46, 55:01:01:01-55:01:20, 55:01:22-55:05, 55:07, 55:11-55:21, 55:23, 55:25-55:50, 55:52-55:76, 55:78-55:81, 55:83N-55:132N, 56:10, 56:12, 56:18, 56:23, 56:32, 56:54, 56:66, 57:26, 58:05, 58:20, 58:35, 58:84, 59:01:01:01-59:03, 59:05-59:07, 59:10N-59:11										11													
*08:01:24										10	11												
*08:03, 08:23, 08:54:01-08:54:02, 08:110, 08:133, 08:139, 08:271, 08:295, 15:69, 18:179, 37:01:01:01-37:01:04, 37:01:06-37:01:11, 37:01:13-37:01:29, 37:03N, 37:06:01-37:06:02, 37:08-37:10, 37:12-37:21, 37:23-37:25, 37:27-37:36, 37:38, 37:40-37:47, 37:49-37:59, 37:61-37:66, 37:68-37:93, 37:96, 37:98-37:107, 38:01:08, 38:01:19, 38:02:03, 38:04, 38:06, 38:25, 38:89, 41:22											11			14									
*08:69, 08:200, 39:76, 41:18						6					11												
*08:118, 51:21, 51:101, 51:153:01-51:153:02, 51:280, 51:340, 52:37, 52:71, 52:84, 78:07											11		13										
*08:190, 39:99								w			11												
*13:15, 51:158:02, 51:225, 51:372, 56:34													13			17							
*13:62, 44:459, 51:01:01:01-51:01:18, 51:01:20-51:01:30, 51:01:32-51:01:49, 51:01:51-51:01:67, 51:01:69-51:01:73, 51:01:75-51:01:101, 51:03, 51:07:01-51:08:03, 51:08:05-51:09:03, 51:11N-51:14, 51:16, 51:19-51:20, 51:24:01-51:24:05, 51:26-51:28, 51:30-51:33, 51:35, 51:38-51:39, 51:41N, 51:43-51:44N, 51:48-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:63:01-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:95-51:100, 51:102-51:107, 51:109-51:115, 51:117-51:134, 51:136-51:138, 51:140-51:142, 51:145, 51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159-51:161, 51:162:02-51:171, 51:173Q-51:174:02, 51:177-51:178N, 51:181-51:193:02, 51:195-51:196, 51:198, 51:200-51:205, 51:208-51:209, 51:211-51:215, 51:219, 51:221-51:222, 51:224, 51:226, 51:228-51:253, 51:255-51:260, 51:262-51:273N, 51:275-51:278, 51:282-51:292, 51:294, 51:296-51:297, 51:299, 51:301-51:309, 51:311-51:318N, 51:320-51:339, 51:342-51:344N, 51:346Q-51:351, 51:353, 51:355-51:358, 51:360-51:371, 51:373-51:375, 52:01:01:01-52:01:10, 52:01:12-52:01:38, 52:01:40-52:02:02, 52:04-52:10:03, 52:13-52:15, 52:17-52:23, 52:25-01-52:25:02, 52:28-52:36, 52:38-52:56, 52:58-52:62, 52:64-52:66, 52:68-52:70, 52:72-52:74, 52:76, 52:78-52:83, 52:85-52:87, 52:89N-52:110N, 56:05:01-56:06, 56:21, 58:08:01-58:08:02, 78:01:01:02-78:02:02, 78:05-78:06, 78:08-78:10												13											
*15:09:01-15:10:08, 15:18:01:01-15:18:10, 15:23, 15:51-15:52, 15:72, 15:80, 15:90, 15:99, 15:108, 15:114-15:115, 15:119, 15:124, 15:133-15:134, 15:153, 15:161, 15:176, 15:197:01-15:198, 15:200, 15:221, 15:226N, 15:229, 15:238, 15:243, 15:263, 15:275:01-15:275:02, 15:290, 15:292-15:293, 15:306-15:307, 15:311-15:314, 15:329, 15:337-15:338, 15:351, 15:354, 15:376, 15:380N, 15:388, 15:414, 15:427, 15:433-15:434, 15:436, 15:439, 15:445, 15:451, 15:455, 15:487N, 15:498, 15:503, 15:506, 15:526, 15:530, 15:548, 15:570, 15:573, 15:607, 15:611, 15:620, 15:643				3										14									
*15:21:01:01-15:21:01:02, 15:66, 15:294N, 15:394, 15:404, 15:420, 15:504:01-15:504:02, 15:563, 15:612, 15:615, 15:623, 15:646, 35:82, 35:85, 35:135, 35:294, 44:263, 56:33				3																			
*15:37, 15:335				3				7					14										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
*15:38:01-15:38:02, 15:185, 15:364, 51:45, 51:135							7																
*15:44, 39:129, 59:08				3							11												
*15:93, 15:186, 15:252, 38:05				3							11			14									
*15:189:01-15:189:02, 35:26				3						10													
*15:323				3				7			11			14									
*15:368, 18:04:01-18:04:02, 18:107, 18:125, 56:31							7				11												
*15:382				w										14									
*15:435				3			7						13	14									22
*15:510, 35:475, 44:440, 46:84, 50:51, 57:11, 58:02:01:01-58:02:02, 58:06-58:07, 58:25, 58:38, 58:43, 58:60, 58:104, 58:132												12											
*15:610							7							14									
*15:645, 44:203:01-44:203:02, 44:275, 44:290, 51:227, 51:281, 52:88, 56:81, 57:14:01-57:14:02, C*03:384														15									
*18:01:01:01-18:01:01:71, 18:01:03-18:01:12, 18:01:14-18:01:15, 18:01:17-18:01:22, 18:01:24-18:01:36, 18:01:38-18:01:40, 18:01:42-18:01:49, 18:01:52, 18:05:01:01-18:05:08, 18:12:01-18:12:02, 18:15, 18:17N-18:20, 18:23N-18:24, 18:28, 18:31-18:34, 18:37:01-18:38, 18:40-18:43, 18:45-18:49, 18:51, 18:53, 18:55, 18:59-18:60, 18:62-18:66, 18:71-18:79, 18:81-18:83, 18:85, 18:87-18:93, 18:96, 18:99-18:100, 18:103-18:104, 18:106Q, 18:108-18:109, 18:111-18:124, 18:126-18:131:01:02, 18:133-18:135:01:02, 18:137-18:141, 18:143-18:146, 18:148, 18:151, 18:154N-18:163, 18:166-18:167, 18:173, 18:175-18:178, 18:180-18:188, 18:192-18:193, 18:196, 18:199-18:202, 18:205-18:209, 18:211-18:226								7	8		11												
*18:01:02, 18:01:23, 18:03:01:01-18:03:02, 18:29, 18:80, 18:94N, 18:98:01-18:98:02, 18:102, 18:132, 18:142, 18:152, 18:171-18:172, 18:174, 18:198, 18:203								7	8		11												
*18:01:13, 18:01:51, 18:09, 18:25, 18:54, 18:67-18:68, 18:70, 18:84, 18:95, 18:136, 18:147, 18:190, 18:194:01:01-18:195, 18:197								7			11			14									
*18:01:16, 18:01:41, 18:01:50, 18:02:01-18:02:03, 18:11, 18:14, 18:27, 18:39, 18:110, 18:168, 18:170, 18:210, 37:05, 37:11, 37:67, 39:13:01-39:13:02, 39:20, 39:49, 39:170, 39:186, 42:11, 48:37								8			11			14									
*18:01:37				2				7	8		11			14									
*18:13, 18:30					4			7	8		11			14									
*18:22, 18:105								8			11			14	15		17						
*18:26, 18:61, 18:97, 18:191								7	8		11		13	14									
*18:36, 18:50, 18:57:01-18:57:02, 18:86, 18:153, 18:169								7	8					14									22
*18:44:01-18:44:02	1							8			11			14									
*18:52								7	8		11			14			17						
*18:69, 18:149-18:150, 18:204								8			11			14	15								22
*27:07:04				2																			
*27:153, 40:174								8					13										
*35:01:64, 35:03:17, 35:29:03, 49:26											10												
*35:12:04, 35:16-35:17:05, 35:32:01, 35:51:02, 35:61:02, 35:98, 35:346, 35:484:01-35:484:02, 35:525, 44:140, 46:01:07, 50:04:01, 58:59:01, C*03:102, C*03:263:01-03:263:03, C*03:514																					20		
*35:21, 35:24:01-35:24:02, 35:81, 35:109, 35:287, 53:02, 53:06, 53:28, 58:09, C*15:39, C*15:130, C*15:195														15		17							

Negative Control

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
*35:96, 35:157, 35:188, 35:190, 58:76											11				15		17			20			
*35:125														14									
*35:233, 51:04:01-51:04:02, 51:46, 51:139, 51:310													13	15									
*35:440, C*04:443, C*08:08:01-08:08:02						6																	
*35:519, 49:09, 50:58, A*33:40:01																	17						
*37:04:01-37:04:02																							
*38:01:01:01-38:01:07, 38:01:09-38:01:18, 38:01:20-38:02:02:02, 38:02:04-38:02:11, 38:07, 38:09-38:12, 38:15, 38:17-38:18, 38:20:01-38:21, 38:23-38:24, 38:27, 38:29, 38:31-38:32, 38:34N-38:40, 38:42-38:45, 38:47-38:62, 38:64, 38:66-38:63N, 38:65-38:68, 38:90-38:102, 38:160-38:173Q		2									11			14	15								
*38:03, 38:13-38:14, 38:16, 38:26, 38:28, 38:41, 38:46, 39:01:15, 39:104, 39:192		2									11												
*38:08, 38:22, 38:30, 38:63, 38:65		2												14									
*38:19		2				6					11			14									
*38:33			3							10	11			14									
*38:84		w									11			14									
*39:01:01:01-39:01:29, 39:01:03:01-39:01:08, 39:01:10-39:01:14, 39:01:16-39:01:25, 39:01:27-39:01:29, 39:01:31-39:02:01:02, 39:06:01:01-39:06:02:08, 39:06:04-39:06:08, 39:09:01:01-39:09:05, 39:14:01:01-39:15, 39:18, 39:19:02, 39:22, 39:25N-39:31:01:02, 39:34-39:36, 39:38Q-39:39:01, 39:41-39:42, 39:44, 39:46, 39:51-39:54, 39:57-39:62, 39:64-39:71, 39:73, 39:75, 39:77, 39:81-39:82, 39:83:02, 39:84-39:88, 39:90-39:91, 39:93-39:95N, 39:97N-39:98, 39:100, 39:103, 39:106-39:109, 39:111-39:112, 39:114-39:117, 39:119, 39:121-39:125, 39:127-39:128, 39:130, 39:132, 39:134-39:142N, 39:146N, 39:148, 39:150Q-39:151, 39:153-39:160, 39:169, 39:171, 39:173, 39:176-39:177, 39:179, 39:183, 39:187-39:191, 39:193, 67:01:01, 67:03-67:04, 67:07		2						8			11												
*39:03:01:01-39:03:02, 39:24:01-39:24:02, 39:120, 39:144, 39:152, 39:182, 39:185		2				6		8			11												
*39:04			3							10	11												
*39:05:01:01-39:05:05, 39:07-39:08, 39:11, 39:55-39:56, 39:113, 39:118, 39:147N, 39:149, 39:161N, 39:163, 39:165-39:168, 39:172		2						8			11			14									
*39:24:03, 39:184, 42:06, 42:20						6		8			11												
*39:32		2				7	8						13										
*39:37:01:01-39:37:01:02		2				6	8				11			14									
*39:40:01N-39:40:02N		2			5			8			11												
*39:48, 67:06		2				7	8				11			13									
*39:174		2					8							14									
*39:178		w						11															
*40:28							8						13	15		17						22	
*40:51, 40:163						7					11			13									
*40:113, 51:06:01:01-51:06:04, 51:62, 51:194, 51:293, 51:319, 51:341						7							13										
*40:243							8															21	
*44:190, 55:77											11			15									
*44:533, 46:48																	18						
*48:07						6	8							14									
*51:01:68										10				13									
*51:22, 78:03			3											13									
*51:29, 51:82, 51:217, 51:279, 52:24				4										13									
*51:56:01-51:56:03														13		15		17					
*51:116, 51:176, 51:179, 51:218, 51:354, 52:16, 52:27, 52:77														13	14							22	
*54:38											11												
*58:46, C*01:35, C*01:107, C*03:17:01-03:17:02, C*03:71, C*03:249, C*03:271, C*03:302, C*06:96, C*06:197, C*12:122, C*14:20, C*16:149, C*16:183, C*16:202												12											
C*01:10, C*02:05:01-02:05:03, C*02:17, C*06:08, C*12:119, C*14:25, C*16:29, C*16:50, C*17:21																						21	
C*01:30, C*14:76		1		4													17					22	
C*01:32:01-01:32:02, C*06:20, C*14:82, C*16:98, C*16:102, C*16:110																							
C*01:101, C*01:113, C*15:55																16							
C*02:142, C*07:516, C*07:521:01-07:521:02, C*08:243, C*08:246						7											17						
C*05:16, C*05:85, C*05:107, C*05:241, C*06:129, C*07:364, C*08:12:01:01-08:12:01:02								9								16							
C*05:216:01:01-05:216:01:02, C*08:51, C*08:114				4		7											17						
C*06:207, C*12:87		1															17					22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23

Negative Control