

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
HLA-B allele^{1,2,3}																										
*37:01:01-37:01:11, 37:31-37:32, 37:38, 37:41, 37:49-37:51, 37:53-37:54, 37:57	1				5							12														
*37:02		2			5							12														25
*37:03N	1		3		5							12														
*37:04:01-37:04:02	1			4	5							12														
*37:05	1					6						12														
*37:06:01-37:06:02	1				5		7					12														
*37:07					5			8				12														
*37:08	1				5				9																	
*37:09					5						11	12				16										
*37:10	1											12	13													
*37:11, 37:37	1					6						12		14	15			18								
*37:12, 37:56					5							12				16										
*37:13	1		3	4	5							12														
*37:14	1					w				10		12						18								
*37:15	1				5							12	13													
*37:16Q	1				5							12									20					
*37:17	1				5							12										22				
*37:18	1				5					11		12														
*37:19:01					5							12											23			
*37:19:02, 37:40, 37:44					5							12														
*37:20	1				5							12				16	17									
*37:21	1				5							12							19							
*37:22	1								9			12														
*37:23	1				5							12												23		
*37:24, 37:27 ⁴	1				5							12										21				
*37:25	1				5							12					17									
*37:26, 37:46 ⁴	1				5					10		12														
*37:28	1	2			5							12														
*37:29	1				5							12						18								
*37:30N	1				5				9			12														
*37:33N	1				5							12	13									21				
*37:34	1											12														
*37:35	1				5							12				16										
*37:36	1				5		7					12									20					
*37:39				4	5							12									20					
*37:42N	1				5		7		9			12														
*37:43	1				5						11	12											23			
*37:45	1				5							12			15				19							
*37:47	1				5							12		14												
*37:48	1			4	5							12									20					
*37:52	1				5							12													24	
*37:55	1				5																					
*07:02:01-07:03, 07:05:01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:237-07:256, 08:79, 08:156, 15:138, 15:230, 15:241, 15:297, 44:150, 48:05, 48:08, 48:15, C*02:23, C*03:260, C*04:77								8																		
*07:04, 07:146								8			11					16										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
*07:19, 07:25, 08:01:01-08:01:22, 08:01:24-08:05, 08:07-08:08N, 08:10-08:11, 08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:27, 08:29-08:31, 08:33-08:39, 08:41-08:48, 08:50-08:54, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:71-08:73, 08:75-08:78, 08:80-08:83, 08:85-08:86N, 08:88, 08:90-08:106, 08:108-08:110, 08:112-08:125, 08:127-08:128, 08:130, 08:132, 08:134-08:138, 08:140-08:142, 08:144-08:154, 42:01:01-42:01:03, 42:05:01-42:08, 42:10-42:12, 42:14-42:16, 42:21, 44:166, 44:213											11					16										
*07:27, 07:236		2			5			8	9																	
*07:65, 07:134						6		8							15			18								

Negative Control

*07:133				6			8																			
*07:143							8							14												
*07:197			2				8																			
*08:01:23, 08:14, 08:28, 08:74, 08:87, 08:111, 42:20, 44:106, 44:158												11														
*08:09, 08:12:01-08:12:03, 08:16, 08:60, 08:84, 08:89, 08:107, 08:129, 15:51, 15:83, 15:179:01-15:179:02, 15:199, 15:218Q, 38:30, 42:04, 42:13, 44:14, 44:17, 44:51, 44:62, 44:123, 44:230, 51:08:01-51:08:02, 51:20, 51:36, 51:44N, 51:97, 51:141, 51:153, 52:19, 53:22, 55:20, 55:56, 56:13, 57:09, 57:24, 82:01-82:03																	16									
*08:32, 08:133, 35:87							6					11		15	16	18										
*08:49, C*07:68, C*07:302	1																									
*08:62												11		14	16	19										
*08:126			2				5				9	11			16											
*08:131, 08:139			2								9	11			16											
*08:155							6					11			16											
*13:23, 13:55, 40:47, 40:96, 40:157, 40:201, 44:15, 44:55, 44:103, 44:188, 44:212, 49:02													12	13												
*14:10, 38:20, 39:42, 39:106, 44:117, 51:32, 58:52, C*15:05:08, C*17:01:10																							23			
*15:32:01-15:32:02, 15:299, 44:209, 53:07, C*03:90, C*03:103, C*03:178, C*03:300, C*07:03, C*07:186, C*07:242, C*07:265				4													20									
*15:46, 15:53, 15:106, 15:143, 18:01:21, 18:49, 18:107-18:108, 35:03:03, 35:19, 35:27, 35:47, 35:56, 35:154, 35:203, 35:274, 35:299, 39:69, 40:10:01-40:10:02, 40:40, 40:49, 40:58, 40:129, 40:164, 40:191, 40:210, 40:219, 40:229, 44:09, 44:46, 44:75, 44:90, 44:129, 47:02, 50:13, 50:43, 78:06, 78:09							6																			
*15:71, 15:175, 15:204, 15:210, 15:225, 15:344, 27:06, 27:24, 27:33, 27:138, 39:101, 40:46, 40:93, 40:208, 40:265N, 45:16, 48:22, 48:30, 50:06												12	14													
*15:129, 48:21																									25	
*15:202, 15:239, 15:308, 18:01:13, 18:04, 18:25, 18:43, 35:01:01-35:01:38, 35:01:40-35:02:02, 35:02:04-35:03:02, 35:03:04-35:08:01, 35:08:03-35:08:04, 35:08:06-35:18, 35:20:01-35:24:02, 35:28-35:37, 35:39-35:45, 35:51, 35:53N-35:55, 35:57-35:58, 35:59:02-35:62, 35:64:01-35:72, 35:74-35:83, 35:85-35:86, 35:88-35:114, 35:116-35:153, 35:155-35:161, 35:163-35:168, 35:170-35:197, 35:199-35:202, 35:204-35:206, 35:208-35:209, 35:211-35:216N, 35:218-35:221, 35:223-35:230, 35:232-35:241, 35:243-35:247, 35:249-35:253, 35:255-35:259, 35:261-35:270, 35:272-35:273, 35:275-35:279, 35:281-35:292, 35:294-35:298, 35:300, 39:19:01-39:19:02, 46:40:01-46:40:02, 56:06, 56:45, 58:64, 78:01:01-78:05, 78:07-78:08							6							15	18											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
*15:209N, 53:03, A*68:112:02									9																	
*15:212, 35:63, 40:01:01-40:01:04, 40:01:06-40:06:14, 40:09, 40:11:01-40:11:02, 40:14:01-40:14:03, 40:18, 40:20, 40:22N, 40:24, 40:26-40:31, 40:33-40:36, 40:38-40:39, 40:42-40:45, 40:48, 40:50-40:57, 40:59-40:67, 40:70:01-40:75, 40:77-40:92, 40:94-40:95, 40:97, 40:99-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:128, 40:130:01-40:134, 40:138-40:151, 40:154-40:156, 40:159-40:161, 40:163, 40:165, 40:167-40:182, 40:184, 40:186-40:187, 40:189-40:190, 40:192-40:197, 40:199-40:200, 40:202-40:207, 40:211-40:212, 40:214-40:218, 40:221-40:228, 40:230, 40:233-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:278, 40:280, 40:282-40:291N, 40:293-40:305, 40:307, 40:309-40:312, 40:314-40:317, 44:131, 45:01:01-45:02, 45:04-45:08, 45:10-45:13, 45:15, 45:17, 50:01:01-50:02, 50:04-50:05, 50:07-50:12, 50:14, 50:16-50:20, 50:31-50:42							6					12	14													
*15:342, 47:04, 48:26		2																							25	

Negative Control

*41:05					w						12				16										
*41:10					6					11	12		14		16										
*41:26											12		14		16										
*41:33					6										16										
*42:02:01:01-42:02:01:02, 42:09, 42:17-42:18										11			14		16										
*45:03					6						12		14			17									
*47:01:01:01-47:01:02, 47:06-47:10						7																			
*47:03						w																			
*47:05	2					7																			25
*51:103, 51:116, 51:151																19									
*54:02, C*03:278, C*16:86													14												24
C*06:77																								24	
Allele	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25