

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
HLA-B allele^{1,2,3}																														
*37:01:01-37:01:11, 37:31-37:32, 37:38, 37:41, 37:49-37:51, 37:53-37:54, 37:57, 37:59-37:61, 37:63-37:65	1				5							12																		
*37:02		2			5							12													25					
*37:03N	1		3		5							12																		
*37:04:01-37:04:02	1			4	5							12																		
*37:05	1					6						12																		
*37:06:01-37:06:02	1				5		7					12																		
*37:07					5			8				12																		
*37:08	1				5				9																					
*37:09					5						11	12				16														
*37:10	1											12	13																	
*37:11, 37:37	1					6						12		14	15		18													
*37:12, 37:56					5							12				16														
*37:13	1		3	4	5							12																		
*37:14	1					w				10		12					18													
*37:15	1				5							12																28		
*37:16Q	1				5							12								20										
*37:17	1				5							12									22									
*37:18	1				5						11	12																		
*37:19:01					5							12										23								
*37:19:02, 37:40, 37:44, 37:66					5							12																		
*37:20	1				5							12				16	17		19											
*37:21	1				5							12																		
*37:22	1							9				12																		
*37:23	1				5							12										23								
*37:24, 37:27 ⁴	1				5							12									21									
*37:25	1				5							12					17													
*37:26, 37:46 ⁴	1				5					10		12																		
*37:28	1	2			5							12																		
*37:29	1				5							12						18												
*37:30N	1				5				9			12																		
*37:33N	1				5							12									21								29	
*37:34	1											12																		
*37:35	1				5							12				16														
*37:36	1				5		7					12								20										
*37:39				4	5							12									20									
*37:42N	1				5		7		9			12																		
*37:43	1				5						11	12										23								
*37:45	1				5							12			15			19												
*37:47	1				5							12		14																
*37:48	1			4	5							12									20									
*37:52	1				5							12												24						
*37:55, 37:62	1				5							12																		
*37:58	1				5							12																26		
*37:67	1					6						12		14	15			18	19											
*37:68	1				5							12																27		
*07:02:01-01-07:03, 07:05:01-01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:237-07:260, 07:262-07:272N, 07:274-07:296, 08:79, 08:156, 15:138, 15:230, 15:241, 15:297, 44:150, 48:05, 48:08, 48:15, C*02:23, C*03:260, C*04:77								8																						
*07:04, 07:146								8			11				16															
*07:19, 07:25, 08:01:01-01-08:01:22, 08:01:24-08:05, 08:07-08:08N, 08:10-08:11, 08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:27, 08:29-08:31, 08:33-08:39, 08:41-08:48, 08:50-08:54:02, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:71-08:73, 08:75-08:78, 08:80-08:83, 08:85-08:86N, 08:88, 08:90-08:106, 08:108-08:110, 08:112-08:125, 08:127-08:128, 08:130, 08:132, 08:134-08:138, 08:140-08:142, 08:144-08:154, 08:157-08:170, 08:172-08:179, 08:182-08:183, 42:01-01-42:01:03, 42:05:01-42:08, 42:10-42:12, 42:14-42:16, 42:21-42:24, 44:166, 44:213										11					16															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
*07:27, 07:236		2			5			8	9																					
*07:65, 07:134						6		8						15			18													
*07:133						6		8																						
*07:143								8						14																
*07:197		2						8																						

Negative Control

