

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
HLA-B allele^{1,2,3}																																
*37:01:01-01-37:01:27, 37:31-37:32, 37:38, 37:41, 37:49, 37:51, 37:53-37:54, 37:57, 37:59-37:61, 37:63-37:65, 37:69-37:76, 37:78-37:82N, 37:86N-37:93, 37:96	1				5							12																				
*37:02		2			5							12														25						
*37:03N	1		3		5							12																				
*37:04:01-37:04:02	1			4	5							12																				
*37:05	1					6						12																				
*37:06:01-37:06:02	1				5		7					12																				
*37:07					5			8				12																				
*37:08	1				5				9																							
*37:09					5						11	12				16																
*37:10	1											12	13																			
*37:11, 37:37	1					6						12		14	15			18														
*37:12, 37:56					5							12				16																
*37:13	1		3	4	5							12																				
*37:14	1				w				10			12						18														
*37:15	1				5							12																28				
*37:16Q	1				5							12								20												
*37:17	1				5							12										22										
*37:18	1				5						11	12																				
*37:19:01					5							12											23								30	
*37:19:02					5							12																			30	
*37:20	1				5							12				16	17															
*37:21	1				5							12							19													
*37:22:01-37:22:02	1								9			12																				
*37:23	1				5							12												23								
*37:24, 37:27 ⁴	1				5							12									21											
*37:25	1				5							12					17															
*37:26, 37:46 ⁴	1				5					10		12																				
*37:28	1	2			5							12																				
*37:29	1				5							12						18														
*37:30N	1				5				9			12																				
*37:33N	1				5							12										21									29	
*37:34, 37:95	1											12																				
*37:35, 37:77, 37:84	1				5							12				16																
*37:36	1				5		7					12											20									
*37:39, 37:97				4	5							12										20										
*37:40, 37:44, 37:66, 37:83, 37:94, 18:195 ⁵					5							12																				
*37:42N	1				5		7		9			12																				
*37:43	1				5						11	12											23									
*37:45	1				5							12			15				19													
*37:47	1				5							12		14																		
*37:48	1			4	5							12										20										
*37:50	1				5							12																			w	
*37:52	1				5							12												24								
*37:55, 37:62, 37:85	1				5							12																				
*37:58	1				5							12															26					
*37:67	1					6						12		14	15			18	19													
*37:68	1				5							12																27				
*07:02:01-01-07:02:84, 07:02:86-07:03, 07:05:01-01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:237-07:260, 07:262-07:272N, 07:274-07:300, 07:302-07:328, 07:330N-07:331, 07:333-07:342, 07:344-07:345, 07:347-07:348, 07:350-07:358, 07:360Q-07:362, 07:364-07:385, 07:387, 07:389-07:399, 07:401-07:406, 07:409-07:410, 07:412-07:419, 07:421-07:435N, 08:79, 08:156, 15:138, 15:230, 15:241, 15:297, 44:150, 46:87, 48:05, 48:08, 48:15, C*02:23, C*03:260, C*04:77								8																								
*07:04:01-07:04:02, 07:146, 07:349									8			11				16																

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31		
*07:19, 07:25, 08:01:01:01-08:01:22, 08:01:24-08:05, 08:07-08:08N, 08:10-08:11, 08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:27, 08:29-08:31, 08:33-08:39, 08:41-08:48, 08:50-08:54:02, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:71-08:73, 08:75-08:78, 08:80-08:83, 08:85-08:86N, 08:88, 08:90-08:106, 08:108-08:110, 08:112-08:125, 08:127-08:128, 08:130, 08:132, 08:134-08:138, 08:140-08:142, 08:144-08:154, 08:157-08:170, 08:172-08:179, 08:182-08:183, 08:185-08:199, 08:201-08:204, 08:206-08:225, 08:228-08:231, 08:233-08:238, 08:240-08:241, 08:243-08:244, 08:246-08:250, 08:252N-08:253, 08:256-08:263, 08:265-08:280, 42:01:01:01-42:01:05, 42:05:01-42:08, 42:10-42:12, 42:14-42:16, 42:21-42:25, 42:29-42:31, 44:166, 44:213											11				16																		
*07:27, 07:236		2			5			8	9																								
*07:65, 07:134, 07:400						6		8							15		18																
*07:133						6		8																									
*07:143								8						14																			
*07:197		2						8																									
*07:273					5			8																									
*07:329								8															23										
*08:01:23, 08:14, 08:28, 08:74, 08:87, 08:111, 08:200, 08:227, 08:245, 08:251, 08:264, 42:20, 44:106, 44:158											11																						
*08:09, 08:12:01-08:12:03, 08:16, 08:60, 08:84, 08:89, 08:107, 08:129, 08:181, 08:184, 08:232, 08:254, 13:154, 15:51, 15:83:01:01-15:83:01:02, 15:179:01-15:179:02, 15:199, 15:218Q, 15:518, 38:30, 39:156, 42:04, 42:13, 44:14, 44:17, 44:51, 44:62, 44:123, 44:230, 51:08:01:01-51:08:05, 51:20, 51:36, 51:44N, 51:97, 51:141, 51:153, 51:263, 51:275, 51:342, 52:19, 53:22, 55:20, 55:56, 56:13, 57:09, 57:24, 57:119, 82:01:01:01-82:04, C*07:626																16																	
*08:32, 08:133, 08:180, 08:239, 35:87						6					11				15	16		18															
*08:49, C*07:68, C*07:302, C*07:929, C*07:941	1																																
*08:62, 08:242, 42:26											11			14		16			19														
*08:126		2			5				9		11					16																	
*08:131, 08:139		2							9		11					16																	
*08:155, 41:47						6					11					16																	
*08:226, 42:02:01:01-42:02:02, 42:09, 42:17-42:18, 42:27											11			14		16																	
*13:23, 13:55, 40:47, 40:96, 40:157, 40:201, 44:15:01:01-44:15:01:02, 44:55, 44:103, 44:188, 44:212, 49:02, 49:70												12	13																				
*13:151, 14:10, 15:227, 15:460, 38:20:01, 39:42, 39:106, 44:117, 44:251, 44:338, 51:32, 58:52, C*03:26:02, C*15:140, C*15:152:03, C*17:01:10																							23										
*15:32:01-15:32:02, 15:299, 15:583, 44:209, 53:07, 53:65, 58:121, C*03:90, C*03:103, C*03:178, C*03:300, C*03:474, C*07:03, C*07:186, C*07:242, C*07:265, C*07:748				4																		20											
*15:46, 15:53, 15:106, 15:143, 15:525, 18:01:21, 18:49, 18:107-18:108, 18:132, 18:142, 18:198, 35:01:52, 35:02:14, 35:03:03, 35:19, 35:27, 35:47, 35:56, 35:154, 35:274, 35:299, 35:450, 39:69, 40:06:15, 40:10:01:01-40:10:02, 40:40, 40:49, 40:58, 40:129, 40:164, 40:191, 40:210, 40:219, 40:229, 40:318, 40:325, 40:328, 40:342, 40:399N, 40:449, 40:457, 40:462, 44:09, 44:46:01-44:46:02, 44:75, 44:90, 44:129, 44:254, 44:277, 44:438N, 47:02, 48:41, 50:13, 50:43, 56:67, 78:06, 78:09-78:10						6																											
*15:71, 15:175, 15:204, 15:210, 15:225, 15:344, 15:381, 15:559, 15:571, 18:178, 27:06:01:01-27:06:01:02, 27:20, 27:21:02, 27:24, 27:33, 27:91, 27:138, 27:153, 27:208, 27:222, 39:101, 40:46, 40:93, 40:208, 40:265N, 40:400, 40:430, 45:01:09, 45:16, 48:22, 48:30, 50:06, 50:75												12		14																			
*15:129, 15:395, 48:21																											25						

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31					
*15:202, 15:239, 15:308, 15:376, 18:01:13, 18:04:01-18:04:02, 18:25, 18:43, 35:01:01:01-35:01:38, 35:01:40-35:01:51, 35:01:53-35:02:02, 35:02:04-35:02:08, 35:02:10-35:02:13, 35:02:15-35:03:02, 35:03:04-35:08:01:10, 35:08:03-35:08:04, 35:08:06-35:18, 35:20:01-35:24:02, 35:28:01:01-35:37, 35:39-35:43:01, 35:43:03-35:45, 35:51:01-35:51:02, 35:53N-35:55, 35:57-35:58, 35:59:02-35:62, 35:64:01-35:72, 35:74-35:83, 35:85-35:86, 35:88-35:114, 35:116-35:153, 35:155-35:161, 35:163-35:168, 35:170-35:197, 35:199-35:202, 35:204-35:206, 35:208:01-35:209, 35:211-35:216N, 35:218-35:221, 35:223-35:230, 35:232-35:241, 35:243-35:247, 35:249-35:253, 35:255-35:259, 35:261-35:270, 35:272-35:273, 35:275-35:279, 35:281-35:292, 35:294-35:298, 35:300-35:309, 35:312-35:316, 35:318-35:322, 35:324-35:328, 35:330-35:354, 35:356-35:360, 35:362-35:423, 35:425-35:432, 35:434-35:440, 35:442-35:449, 35:451-35:470, 35:472-35:521, 35:523-35:524, 35:526-35:529, 39:19:01-39:19:02, 40:365, 46:40:01-46:40:02, 56:06, 56:45, 58:64, 78:01:01:02-78:05, 78:07-78:08						6									15		18																			
*15:209N, 39:139N, A*68:112:02									9																											
*15:212, 35:63, 35:317, 40:01:01-40:01:04, 40:01:06-40:06:14, 40:06:16-40:06:27, 40:09, 40:11:01:01-40:11:02, 40:14:01-40:14:03, 40:18, 40:20:01:01-40:20:01:02, 40:22N, 40:24, 40:26-40:31, 40:33-40:36, 40:38-40:39, 40:42-40:45, 40:48, 40:50:01:01-40:57, 40:59-40:67, 40:70:01-40:75, 40:77-40:92, 40:94-40:95, 40:97, 40:99-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:128, 40:130:01-40:134, 40:138-40:151, 40:154-40:156, 40:159-40:161, 40:163, 40:165, 40:167-40:182, 40:184, 40:186:01-40:187, 40:189-40:190, 40:192-40:197, 40:199-40:200, 40:202-40:207, 40:211-40:212, 40:214-40:218, 40:221-40:225:01, 40:226-40:228, 40:230, 40:233-40:243, 40:245:01-40:255, 40:257-40:264, 40:266-40:278, 40:280, 40:282-40:291N, 40:293-40:305, 40:307, 40:309-40:312, 40:314-40:317, 40:319-40:324, 40:326-40:327, 40:329-40:334, 40:336-40:339, 40:341, 40:343, 40:345N-40:349, 40:351-40:354, 40:356, 40:358-40:364, 40:366-40:393, 40:395-40:398, 40:401-40:422, 40:424-40:428N, 40:431-40:448, 40:450-40:456, 40:458-40:461, 40:463-40:465, 40:467-40:468, 40:470-40:480, 40:482-40:484, 40:486-40:495, 41:62, 44:131, 45:01:01:01-45:01:08, 45:01:10-45:02, 45:04-45:08, 45:10-45:13, 45:15, 45:17-45:21, 45:23-45:25, 45:27, 50:01:01:01-50:01:17, 50:01:19-50:02:01:02, 50:04-50:05, 50:07-50:12, 50:14, 50:16-50:20, 50:31-50:42, 50:44-50:74						6					12		14																							
*15:342, 47:04, 48:26		2																								25										
*15:426, 18:03:02, 18:37:01-18:37:02, 18:66, 39:31:01:01-39:31:01:02, 39:128, 46:14, 55:16, 56:35, 73:01:01:01-73:03														14																						
*18:01:01:01-18:01:12, 18:01:14-18:01:20, 18:01:22-18:03:01:03, 18:05:01:01-18:06, 18:08, 18:10-18:11, 18:13-18:15, 18:17N, 18:19-18:24, 18:26-18:28, 18:30-18:36, 18:38-18:42, 18:44:01-18:47, 18:50-18:53, 18:55, 18:57:01-18:65, 18:68-18:78, 18:80-18:106Q, 18:109-18:112, 18:114-18:124, 18:126-18:131:01:02, 18:133-18:135:01:02, 18:137-18:141, 18:143-18:146, 18:148-18:164, 18:166-18:177, 18:180-18:182N, 18:184-18:193, 18:196-18:197, 18:199-18:203, 18:205-18:212, 45:22						6								14	15			18																		
*18:07:01-18:07:03:02, 18:29, 35:50, 35:84, 35:162, 35:231, 35:280, 35:323, 35:441						6								14	15			18	19																	
*18:09													13																							
*18:12:01-18:12:02, 40:185, 40:209, 40:308, 40:350												12		14	15			18																		
*18:18:01:01-18:18:01:02, 18:204			4			6								14	15			18																		
*18:48, 40:25, 40:69, 40:166, 40:244, 40:313, 40:357, 40:481N, 50:15						6								14																						
*18:54					5				9																											

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
*18:56, 18:113						6								14	15	16		18													
*18:79						6												19													
*18:136, 49:67, 51:103, 51:116, 51:151																			19												
*18:147, 35:02:03						6												18													
*18:165		2				6								14	15			18													
*18:179														14	15			18													
*18:183						6								14	15			18					23								
*27:01, 27:142		2										12	13																		
*27:02:01:01-27:03, 27:05:02:01-27:05:17, 27:05:19-27:05:55, 27:07:01-27:07:06, 27:09-27:10, 27:13:01-27:14, 27:16-27:17, 27:27-27:28, 27:31-27:32, 27:34-27:36, 27:38-27:39, 27:41-27:43, 27:45-27:56, 27:58-27:65N, 27:67, 27:70-27:90:04, 27:93, 27:95-27:99, 27:101, 27:104, 27:108, 27:110-27:111, 27:116-27:121, 27:123-27:124, 27:126, 27:128-27:137, 27:139-27:141, 27:143-27:146, 27:148, 27:150-27:152, 27:155-27:156, 27:158-27:164, 27:166-27:167, 27:169-27:172, 27:174-27:191, 27:193-27:202, 27:205-27:207, 27:209, 27:212N-27:213, 27:216-27:221, 27:223N, 27:225N, 27:227-27:228, 27:230-27:239, 27:243N-27:245, 27:248		2										12																			
*27:04:01-27:04:06, 27:08, 27:12:01:01-27:12:01:03, 27:15, 27:25-27:26, 27:40, 27:44, 27:66N, 27:68, 27:100, 27:103, 27:105, 27:112-27:115, 27:147, 27:149, 27:165, 27:173, 27:204, 27:210-27:211, 27:214, 27:226, 27:229, 27:240, 27:247		2										12		14																	
*27:05:18:01-27:05:18:02		2							9			12																			
*27:11, 27:21:01, 27:30, 27:102, 27:106-27:107, 27:125, 27:154, 27:168, 40:19, 40:76, 40:109, 40:117, 40:292, 40:340, 40:394, 44:18:01:01-44:18:01:02, 49:01:01:01-49:01:19, 49:03-49:09, 49:11-49:14, 49:16-49:23, 49:27-49:30, 49:32-49:62, 49:64-49:66, 49:68-49:69, 49:71-49:73, 52:51												12																			
*27:18		2										12		14											25						
*27:19:01:01-27:19:01:02, 27:127		2										12																		30	
*27:23		2										12							19												
*27:29		2										12													25						
*27:37, 27:157, 27:224, 27:246N, A*32:89		2										12																			
*27:57, 27:122		2										12								21											
*27:69		2										12		14										23							
*27:92, 55:90		2							9			12																			
*27:94N		2										12									21									29	
*27:109, 27:242		2												14																	
*27:192												12		14										23							
*27:203, 27:241		2										12																		28	
*35:01:39, 35:248, 35:424, 35:522															15			18													
*35:38, 35:115, 35:169, 35:260						6									15	16		18													
*35:48, 35:433			4			6									15			18		20											
*35:52, 35:310						6									15																
*35:59:01, 35:242						6									15			18					23								
*35:203, 35:355						6																									30
*35:217, 40:08, 40:68, 40:106, 40:232						6								14					19												
*35:222, 35:329, 35:525																		18													
*35:293															15																
*38:17		2				5			9																						
*38:61																										26					
*40:01:05, 40:37, 40:135, 40:256N, 40:281, 40:306, 40:335, 40:344, 40:423, 50:01:18						6						12																			
*40:07, 40:162, 40:220						6						12		14					19												
*40:13												12							19												
*40:15-40:16:01:02, 40:23, 40:32, 40:98, 40:158, 40:183, 40:198, 40:355, 40:466						6		8				12		14																	
*40:136, 40:231, 40:279, 41:01:01:01-41:01:07, 41:03:01-41:03:02, 41:06-41:09, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25, 41:28-41:30, 41:32, 41:34-41:35, 41:37, 41:44, 41:48, 41:50, 41:53-41:54, 41:56Q-41:57, 41:59, 41:61, 41:63-41:64, 41:66-41:67, 41:70, 41:73, 45:09, 45:14						6						12		14	16																
*40:152						6						12		14							21										
*40:153						6						12		14								22									
*40:188							7					12																			
*40:213, 40:485						6						12		14									23								
*40:225:02, 40:469						6						12		14											25						
*40:429		2				6						12		14											25						

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
*41:02:01:01-41:02:09, 41:04, 41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:43, 41:45N, 41:49, 41:51, 41:58, 41:65, 41:68-41:69, 41:71-41:72, 41:74						6					11	12		14		16																
*41:05						w						12				16																
*41:10, 41:52						6					11	12		14																		
*41:26												12		14		16																
*41:33						6										16																
*41:46											11	12				16			19													
*41:55						6					11	12		14		16									24							
*41:60												12				16																
*42:28											11					16																
*44:257							7																		23							
*45:03						6						12		14			17															30
*47:01:01:02-47:01:05, 47:06-47:10, 47:12							7																									
*47:03						w																										
*47:05		2					7																									
*53:03, 53:55									9																							30
*54:02, C*03:278, C*03:504, C*07:714, C*07:724, C*16:86														14											24							
C*06:77																									24							
C*07:494Q																					20											
C*07:516				4																												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	

Negative Control