

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
HLA-B allele ^{1,2,3}																																
*37:01:01-01:37:01:31, 37:31-37:32, 37:38, 37:41, 37:49, 37:51, 37:53-37:54, 37:57, 37:59-37:61, 37:63-37:65, 37:69-37:76, 37:78-37:82N, 37:86N-37:93, 37:96, 37:98-37:108	1				5							12																				
*37:02		2			5							12													25							
*37:03N	1		3		5							12																				
*37:04:01-37:04:02	1			4	5							12																				
*37:05	1					6						12																				
*37:06:01-37:06:02	1				5		7					12																				
*37:07					5			8				12																				
*37:08	1				5				9																							
*37:09					5						11	12				16																
*37:10	1											12	13																			
*37:11, 37:37	1					6						12		14	15			18														
*37:12, 37:56					5							12				16																
*37:13	1		3	4	5							12																				
*37:14	1					w				10		12						18														
*37:15	1				5							12																28				
*37:16Q	1				5							12								20												
*37:17	1				5							12									22											
*37:18	1				5						11	12																				
*37:19:01					5							12											23							30		
*37:19:02					5							12																			30	
*37:20	1				5							12				16	17															
*37:21	1				5							12							19													
*37:22:01-37:22:02	1							9				12																				
*37:22:03, 37:34, 37:95	1											12																				
*37:23	1				5							12											23									
*37:24, 37:27 ⁴	1				5							12								21												
*37:25	1				5							12					17															
*37:26, 37:46 ⁴	1				5					10		12																				
*37:28	1	2			5							12																				
*37:29	1				5							12						18														
*37:30N	1				5			9				12																				
*37:33N	1				5							12								21									29			
*37:35, 37:77, 37:84	1				5							12				16					21											
*37:36	1				5		7					12								20												
*37:39, 37:97				4	5							12								20												
*37:40, 37:44, 37:66, 37:83, 37:94, 18:195 ⁵					5							12																				
*37:42N	1				5		7		9			12																				
*37:43	1				5						11	12											23									
*37:45	1				5							12			15				19													
*37:47	1				5							12		14																		
*37:48	1			4	5							12								20												
*37:50	1				5							12																		w		
*37:52	1				5							12												24								
*37:55, 37:62, 37:85	1				5							12																				
*37:58	1				5							12															26					
*37:67	1					6						12		14	15			18	19													
*37:68	1				5							12																27				
*07:02:01:01-07:02:05, 07:02:07-07:02:84, 07:02:86-07:03, 07:05:01:01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:237-07:260, 07:262-01-07:272N, 07:274-07:300, 07:302-07:328, 07:330N-07:331, 07:333-07:342, 07:344-07:345, 07:347-07:348, 07:350-07:358, 07:360Q-07:362, 07:364-07:385, 07:387, 07:389-07:399, 07:401-07:406, 07:409-07:410, 07:412-07:419, 07:421-07:436, 07:438-07:441, 07:443-07:445, 07:447-07:449, 07:451-07:467Q, 07:469N-07:483, 08:79:01-08:79:02, 08:156, 13:174, 15:138, 15:230, 15:241, 15:297, 44:150, 46:87, 48:05, 48:08, 48:15, C*02:23, C*03:260, C*04:77								8																								
*07:04:01-07:04:02, 07:146, 07:349											11					16																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	

Negative Control

[illegible]

[illegible]

[illegible]

*41:33					6										16																	Negative Contr
*41:46										11	12				16			19														
*41:55					6						11	12			14	16									24							
*41:60												12				16																
*41:82					6						11	12			14	16			19													
*42:28												11				16																
*44:257						7																									30	
*45:03					6							12			14		17															
*47:01:01:02-47:01:06, 47:06-47:10, 47:12-47:13																																
*47:03						7																										
*47:05			2				7																									
*51:354																16			19											25		
*53:03, 53:55										9																					30	
*54:02, C*03:278, C*03:504, C*07:714, C*07:724, C*16:86																																
C*06:77															14															24		
C*07:494Q																													24			
C*07:516				4																20												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	