

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	
HLA-B allele^{1,2,3}																												
*38:01:01-38:01:07, 38:01:09-38:01:12, 38:38-38:40, 38:42, 38:51, 38:53, 38:56-38:61	1	2	3	4									13	15														
*38:01:08		2	3										13	15														
*38:02:01-38:02:02, 38:02:04-38:02:07, 38:43-38:44, 38:47-38:49, 38:62	1	2	3		5								13	15														
*38:02:03		2	3		5								13	15														
*38:03	1				5	6	7						13															
*38:04		2	3		5		7						13	15														
*38:05		2	3	4				8					13															
*38:06		2							9				13	15													26	
*38:07	1	2							9	10			13	15													26	
*38:08	1	2	3		5						11			15														
*38:09	1	2	3	4								12	13	15														
*38:10	1	2	3	4										14	15				19				23					
*38:11, 38:19 [†]	1	2	3	4									13	15	16													
*38:12	1	2	3	4						10			13	15														
*38:13	1		3	4							11		13															
*38:14, 38:41	1		3	4								12	13															
*38:15	1	2	3		5			8					13	15														
*38:16	1			4						10			13	15														
*38:17	1	2	3										13	15	16									24				
*38:18	1	2	3		5						11		13	15														
*38:20	1	2	3	4				8					13	15														
*38:21	1	2	3	4			7		9				13	15														
*38:22	1	2	3	4				8						15														
*38:23	1	2	3		5								13	15		17												
*38:24	1	2	3	4									13	15			18											
*38:25		2	3				7						13	15														
*38:26	1			4								12	13															
*38:27	1	2	3	4							11		13	15														
*38:28	1			4									13	15					20									
*38:29	1	2	3		5									15								22						
*38:30	1	2	3	4										15				19		21								
*38:31	1	2	3	4									13	15												25		
*38:32	1	2	3	4									13	14	15								23					
*38:33		2	3	4				8		10			13															
*38:34N	1	2	3	4		w	w						13	15														
*38:35	1	2	3		5	6							13	15														
*38:36, 38:63	1	2	3	4										15					19									
*38:37	1	2	3	4										15				18										
*38:45	1	2	3		5									15														
*38:46	1								10				13	15														
*38:50	1	2	3		5									15					19									
*38:52	1	2	3	4										15														
*38:54	1	2		4									13	15														
*38:55Q	1	2	3	4									13	15									22					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	
*07:02:01-07:02:31, 07:02:33-07:04, 07:07-07:12, 07:14-07:15, 07:17-07:24, 07:26, 07:28-07:31, 07:33:01-07:33:03, 07:35-07:36, 07:39, 07:41-07:47, 07:49N, 07:52, 07:54-07:55, 07:57-07:68:03, 07:70-07:71, 07:73-07:74, 07:76-07:77, 07:79, 07:81-07:84, 07:86-07:89, 07:92-07:96:02, 07:98-07:104, 07:106-07:109, 07:111N, 07:113-07:122, 07:124-07:136:02, 07:139, 07:141-07:148, 07:150-07:154, 07:156-07:163, 07:165-07:175, 07:177-07:179, 07:181N, 07:183-07:200, 07:203-07:205, 07:208, 07:211-07:212, 07:215-07:218, 07:221, 07:224-07:225, 07:227, 07:229-07:234, 07:237-07:239, 07:241-07:248, 07:250-07:257, 07:259-07:260, 07:263, 07:265-07:268, 07:272N-07:275, 08:156, 14:21, 14:42, 15:01:01:01-15:01:04, 15:01:06-15:02:11, 15:04:01:01-15:04:04, 15:06-15:08:02																												

Negative Control

	2				6 7				15 16				19														
*48:36																											
*49:03									9				14								20						
*49:26				4						10			14														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
*51:01:01:01-51:01:18, 51:01:20-51:01:30, 51:01:32-51:01:38, 51:01:41-51:01:49, 51:01:51-51:01:52, 51:02:01-51:02:02, 51:02:04-51:04, 51:06:01-51:07:02, 51:11N-51:14, 51:16, 51:21-51:24:05, 51:26-51:30, 51:32-51:35, 51:38-51:39, 51:41N, 51:43, 51:46, 51:48-51:52, 51:55-51:56:03, 51:58-51:60, 51:62-51:71, 51:74-51:77, 51:79-51:80, 51:82-51:84, 51:86, 51:88-51:89, 51:95-51:96, 51:98N-51:100, 51:102, 51:105-51:107, 51:109-51:111, 51:113-51:114, 51:117-51:119, 51:121, 51:123-51:134, 51:136-51:138, 51:140, 51:142-51:145, 51:149N-51:152, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:175, 51:177-51:178N, 51:180, 51:182-51:193, 51:195-51:196, 51:198-51:203, 51:205, 51:207-51:208, 52:01:01:01-52:01:09, 52:01:12-52:01:28, 52:04-52:09, 52:11, 52:13-52:15, 52:17-52:18, 52:22-52:25:02, 52:30-52:31:02, 52:33-52:43, 52:45-52:56, 52:58-52:61, 58:08:01-58:08:02									9				14						19				23				
*51:01:19, 51:206, 52:57									9				14						w				w				
*51:01:31, 51:01:50, 51:02:03, 51:17-51:18, 51:37, 51:45, 51:85, 51:90, 51:92:01-51:92:02, 51:94, 51:135, 51:146, 51:172, 51:197, 52:01:11, 52:12, 52:26, 53:06, 53:08:01-53:08:02, 53:28, 58:06, 58:19, 58:46, 58:78									9				14														
*51:01:39, 51:204									9				14										23				
*51:01:53									9				14						19				23			26	
*51:05, 51:09:01-51:10, 51:15, 51:31, 51:40, 51:73, 51:81, 51:93, 51:108, 51:115, 51:122, 51:139, 51:148, 51:157, 51:181, 51:194, 52:03, 52:10:01-52:10:03, 58:27-58:28:02, 58:73									9										19								
*51:08:01-51:08:02, 51:20, 51:44N, 51:97, 51:141, 51:153, 52:19									9										19	21							
*51:19, 51:42, 53:01:01-53:02, 53:04-53:05, 53:07, 53:10, 53:14-53:18, 53:20, 53:23-53:27, 53:29, 53:32-53:35, 53:37, 53:40-53:42, 58:01:01:01-58:02:02, 58:04-58:05, 58:07, 58:09-58:11, 58:13-58:16:02, 58:18, 58:21-58:26, 58:29, 58:31N-58:35, 58:37-58:43, 58:45:01-58:45:02, 58:47-58:63, 58:66-58:72N, 58:74-58:77, 58:79-58:81									9																		
*51:36									9				14						19	21							
*51:53, 51:120, 52:28-52:29									9										19				23				
*51:57, 51:103									9				14	15					19				23				
*51:61:01-51:61:02, 52:21, 52:44									9				14						19				w				
*51:72, 51:87, 51:91									9				14						19								
*51:101									9				13	14									23				
*51:116		2							9				14	15					19				23				
*51:176, 51:179		2							9				14						19				23				
*52:01:10				4					9				14						19				23				
*52:16		2				6	7		9				14						19				23				
*52:27		2				6	7		9				14						19				23				
*53:19, 58:65									9										w								
*53:21									9				15														
*53:43, 58:12		2							9																		
*55:54, C*01:90, C*01:116, C*03:317, C*15:24, C*15:128																			19				23				
*55:56, 57:09													14						19	21							
*58:20									9				13														
A*03:67, C*07:209																										26	
A*23:01:18, A*25:01:08				4								12															
C*02:02:07, C*05:01:06								8																			
C*02:68																			19				w				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27

Negative Control