

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	
HLA-B allele^{1,2,3}																															
*38:01:01-38:01:02, 38:01:04-38:01:07, 38:01:09-38:01:12, 38:38-38:40, 38:42, 38:51, 38:53, 38:56-38:60, 38:66, 38:69, 38:71, 38:73, 38:77-38:78	1	2	3	4									13	15													27				
*38:01:03, 38:01:13, 38:67	1	2	3	4									13	15																	
*38:01:08			2	3									13	15													27				
*38:02:01-38:02:02, 38:02:04-38:02:08, 38:43-38:44, 38:47-38:49, 38:62, 38:72, 38:75-38:76, 38:79	1	2	3		5								13	15																	
*38:02:03			2	3	5								13	15																	
*38:03	1				5	6	7						13																		
*38:04			2	3	5		7						13	15																	
*38:05			2	3	4			8					13															27			
*38:06			2						9				13	15													26	27			
*38:07	1	2							9	10			13	15													26	27			
*38:08	1	2	3		5						11		15																		
*38:09	1	2	3	4								12	13	15																	
*38:10	1	2	3	4									14	15				19				23					27				
*38:11, 38:19 ⁴	1	2	3	4									13	15	16												27				
*38:12	1	2	3	4					10				13	15														27			
*38:13	1		3	4							11		13															27			
*38:14	1		3	4								12	13															27			
*38:15	1	2	3		5			8					13	15														27			
*38:16	1			4						10			13	15														27			
*38:17	1	2	3										13	15	16									24							
*38:18	1	2	3		5					11			13	15																	
*38:20:01-38:20:02	1	2	3	4				8					13	15														27			
*38:21	1	2	3	4			7		9				13	15														27			
*38:22	1	2	3	4				8					15															27			
*38:23	1	2	3		5								13	15		17															
*38:24	1	2	3	4									13	15			18											27			
*38:25			2	3			7						13	15														27			
*38:26	1			4								12	13															27			
*38:27	1	2	3	4							11		13	15														27			
*38:28	1			4									13	15					20									27			
*38:29	1	2	3		5								15							22								27			
*38:30	1	2	3	4									15						19	21								27			
*38:31	1	2	3	4									13	15											25			27			
*38:32	1	2	3	4									13	14	15							23						27			
*38:33	1	2	3	4				8		10			13															27			
*38:34N	1	2	3	4		6	7						13	15														27			
*38:35	1	2	3		5	6							13	15																	
*38:36	1	2	3	4									15					19										27			
*38:37	1	2	3	4									15					18										27			
*38:41	1		3	4								12	13																		
*38:45	1	2	3		5								15																		
*38:46	1									10			13	15																	
*38:50	1	2	3		5								15					19													
*38:52, 38:65, 38:70	1	2	3	4									15															27			
*38:54	1	2		4									13	15														27			
*38:55Q	1	2	3	4									13	15							22							27			
*38:61	1	2	3	4									13	15														27	28		
*38:63	1	2	3	4									15					19													
*38:64	1	2	3		5								13	15														28			
*38:68Q	1	2	3	4									13	15														27	29		
*38:74	1	2	3		5								13	15																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	
*07:02:01-01-07:02:31, 07:02:33-07:04:02, 07:07-07:12, 07:14-07:15, 07:17-07:24, 07:26, 07:28-07:31, 07:33:01-07:33:03, 07:35-07:36, 07:39, 07:41-07:47, 07:49N, 07:52, 07:54-07:55, 07:57-07:68:03, 07:70-07:71, 07:73-07:74, 07:76-07:77, 07:79, 07:81-07:84, 07:86-07:89, 07:92-07:96:02, 07:98-07:104, 07:106-07:109, 07:111N, 07:113-07:122, 07:124-07:136:02, 07:139:01-07:139:02, 07:141-07:148, 07:150-07:154, 07:156-07:163, 07:165-07:175, 07:177-07:179, 07:181N, 07:183-07:200, 07:203-07:205, 07:208, 07:211-07:212, 07:215-07:218, 07:221, 07:224-07:225, 07:227, 07:229-07:234, 07:237-07:239, 07:241-07:248, 07:250-07:257, 07:259-07:260, 07:263, 07:265-07:268, 07:271-07:277, 07:279-07:282, 07:284-07:286, 07:288-07:292, 07:294-07:300, 07:302-07:303, 07:305, 07:307-07:316N, 07:318N-07:323, 07:325N-07:326, 07:328-07:331, 07:334-07:335, 08:156, 14:21, 14:42, 15:01:01-15:01:04, 15:01:06-15:02:12, 15:04:01-15:04:04, 15:06-15:08:02, 15:11:01-15:13:01, 15:14-15:17:04, 15:17:06-15:17:07, 15:19, 15:21:01-01-15:21:01:02, 15:24-01-15:28, 15:32:01-15:36, 15:38:01-15:40:02, 15:43-15:46, 15:50, 15:53, 15:56-15:57, 15:60, 15:63, 15:65-15:67, 15:70, 15:76-15:79N, 15:81-15:82, 15:85, 15:87, 15:89, 15:92, 15:94N-15:97, 15:101-15:102, 15:104, 15:106, 15:109-15:110, 15:112-15:113																															

Negative Control

*51:101								9				13	14					19			23				27					
*51:116, 51:218	2							9				14	15					19			23				27					
*51:176, 51:179	2							9				14						19			23				27					
*51:220, 53:08:02								9				14																		
*52:01:10			4					9				14						19			23				27					
*52:16	2					6	7	9				14						19			23				27					
*52:27, 52:77	2					6	7	9				14						19			23				27					
*52:71, 58:20								9				13														27				
*53:01:03, 53:05:01-53:05:02, 53:16, 53:33, 58:01:05, 58:01:12, 58:75, 58:82								9																						
*53:19, 58:65								9										w								27				
*53:21								9				15														27				
*53:43, 58:12	2							9																		27				
*55:54, C*01:90, C*01:116, C*03:317, C*14:92, C*15:24, C*15:128																		19			23									
*55:56, 57:09												14						19		21										
*58:84							8	9													22					27				
A*23:01:18, A*25:01:08			4								12																			
C*02:02:07, C*05:01:06							8																							
C*02:68																		19			w									
C*04:01:80											12																			
C*07:01:59											w																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30